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Drive DeVilbiss Healthcare  
99 Seaview Blvd.  
Port Washington, NY 11050  
1-877-224-0946  
www.drivemedical.com

## Knee Support Cushion

**COMFORT TOUCH™** Cushions by Drive™ are designed to help satisfy your therapeutic needs while immersing you in comfort with an ergonomic design, high-quality foam and luxurious look and feel.

### Beneficial Uses

The comfortable memory foam and super soft velour cover of this Comfort Touch Knee Support Cushion help to alleviate discomfort for side sleepers by promoting proper alignment of the spine, knees and hips so that you can sleep easy and wake refreshed. The Comfort Touch Knee Support Cushion may be especially beneficial for women who are pregnant or people who have poor circulation or lower back pain.

### How to Use

Place the Comfort Touch Knee Support Cushion between your knees or legs while lying down and wrap the strap around one leg. Secure the straps with hook and loop fastener to help keep in place. Slip any loose ends through the loops on the strap.

Drive Comfort Touch Knee Support Cushion is intended to alleviate discomfort and pressure by providing support for alignment of the spine, knees and hips. **Seek medical attention immediately if symptoms persist or condition worsens.**

### Care and Cleaning Instructions

**Cover:** Remove the velour cover by unzipping and removing the memory foam. Do not wash with other articles and attach the hook and loop fastener straps together before washing. Wash it on a cold gentle cycle with mild soap as needed. Do not bleach. Tumble dry on low heat or allow to air dry.

**Memory Foam:** If the memory foam has any odor or stains, surface clean it with a moist cloth and air out until completely dry. No industrial solvents should be used at any time. Any lingering odor will dissipate over time. Never put foam in washer or dryer.

### One Year Limited Warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for 1 year from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited 1 Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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