B5082 / B5083 Toilet Seat Riser with Arms

Included in box:	
	 x 2
 x 2	 x 2
 x 2	 x 2

WARNING:
Before using your raised toilet seat, ensure the unit is firmly attached to the toilet.
USER WEIGHT LIMIT: 300lbs.

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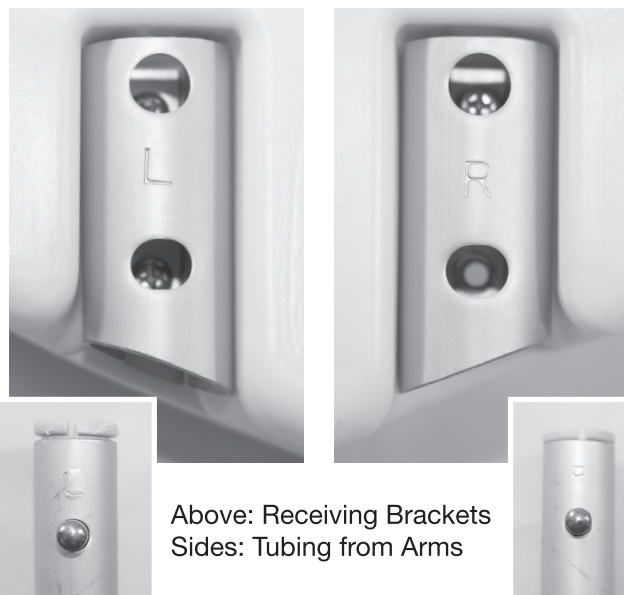
ASSEMBLY

- 1** Locate the toilet seat riser and the two arms.

- 2** Turn the toilet seat riser upside down. Inspect the underside, noting that each side has a “R” and “L” on the receiving brackets, denoting the Right and Left sides. The arms have the same letters, one for Right and one for Left.

Above: Receiving Brackets
Sides: Tubing from Arms

- 3** Align the Right side arm with the Right side of the toilet seat riser (see image). Insert both ends of the arm into the receiving brackets until the gold push buttons are touching the receiving brackets. Push down gold buttons using your thumbs one at a time while sliding arm forward with other hand until buttons click and arm is secure. Both buttons should now fully protrude through holes and arm is secure.

- 4** Repeat Step 3 for the Left side. Your Toilet Seat Riser is now assembled. Proceed to INSTALLATION.

INSTALLATION

- 1** Remove your toilet seat from the toilet base. *(May require screwdriver and pliers, not included.)*

- 2** Place your toilet seat riser on top of the toilet base. Next, place your toilet seat on top of the riser, and align all holes for bolt installation.

- 3** Insert the enclosed bolts thru the holes, attaching all pieces to toilet base using the wing nuts and washers provided. Fully tighten the wingnuts. *(Requires screwdriver and pliers, not included.)*

- 4** **Installation is now complete!**

WARNING:

Before using your raised toilet seat, ensure the unit is firmly attached to the toilet.

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Para usuarios de 300 lb ó menos

ESSENTIAL MEDICAL SUPPLY, INC. LIMITED WARRANTY

Essential Medical Supply, Inc. ("Essential") warrants its products free from defects in materials and workmanship ("Warranty") for a period of three (3) years from the date of purchase ("Warranty Period"). The Warranty extends only to the original purchaser of Essential products. If within the Warranty Period there is a breach of Warranty as to an Essential product, such product shall be repaired or replaced at Essential's sole discretion. This Warranty does not include any labor or shipping costs incurred in replacement, part installation, or repair of any Essential product. ESSENTIAL'S SOLE OBLIGATION AND YOUR EXCLUSIVE REMEDY UNDER THIS WARRANTY SHALL BE LIMITED TO REPAIR AND/OR REPLACEMENT OF THE DEFECTIVE ESSENTIAL PRODUCT.

For Warranty service, please contact the dealer from whom you purchased your Essential product. In the event you do not receive satisfactory Warranty service, please write Essential directly at:

Essential Medical Supply, Inc.
Customer Service Department
6420 Hazeltine National Drive
Orlando, FL 32822-5121

Please provide the dealer's name, address, and date of purchase, and indicate the nature of the problem and the Essential product serial number.

Upon receipt of written notification, Essential will issue a return authorization. The defective unit or parts must be returned to Essential for Warranty inspection using the serial number as identification within thirty (30) days of the date of return authorization. DO NOT return products to Essential without Essential's prior written consent. Cash on delivery shipments will be refused and shipping charges must be prepaid.

Limitations and Exclusions: The Warranty shall not apply to products if 1) the Essential serial number has been removed or defaced; 2) the product has been damaged by negligence, accident, improper operation, maintenance or storage; and/or 3) the product has been modified without Essential's express written consent or repaired by an unauthorized third-party ("Warranty Disqualifying Events") include but are not limited to the addition of unauthorized parts or attachments or damage caused by circumstances beyond Essential's control, the occurrence of which shall be determined exclusively by Essential. The Warranty also shall not apply to problems or defects arising from normal wear and tear on Essential products and parts, including but not limited to cane feet, handles, straps and failure to adhere to user instructions and safety precautions.

THE WARRANTY STATED HEREIN IS THE SOLE AND EXCLUSIVE WARRANTY OFFERED BY ESSENTIAL. THERE ARE NO OTHER WARRANTIES RESPECTING ESSENTIAL'S PRODUCTS, EITHER EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO ANY WARRANTY OF DESIGN, MERCHANTABILITY, OR FITNESS FOR A PARTICULAR PURPOSE, EVEN IF ESSENTIAL HAS BEEN INFORMED OF SUCH PURPOSE. NO AGENT OF ESSENTIAL IS AUTHORIZED TO ALTER OR EXCEED ESSENTIAL'S WARRANTY OBLIGATIONS AS SET FORTH HEREIN. ESSENTIAL SHALL NOT BE RESPONSIBLE FOR ANY INDIRECT, INCIDENTAL, SPECIAL, OR CONSEQUENTIAL DAMAGES ARISING OUT OF USE OF ESSENTIAL PRODUCTS.

Product Serial No. _____

NOTE: Warranty voids if Serial No. label is removed or destroyed.

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