## The FAWSsit<sup>™</sup> FOLD AWAY SHOWER

## **QUICK SET-UP GUIDE**

1. STAND THE UNIT UPRIGHT





2.
PULL THE
RIGHT SHOWER
WALL OPEN

PULL THE LEFT SHOWER WALL OPEN





4.
LOWER THE
DARK BLUE
WATER PAN

## The FAWSsit™ FOLD AWAY SHOWER

## **QUICK SET-UP GUIDE**

CONTINUED

WHEEL BACKWARDS INTO SHOWER





LOWER THE
SUPPORT BAR
AND FASTEN ALL
VELCRO STRIPS
TO THE FRAME

CONNECT SHOWER HEAD
TO SINK'S FAUCET



PLUG THE POWER CORD
INTO A 110V OUTLET



9.
PLACE THE WASTE WATER
TUBE INTO A DRAIN



10.
CONGRATULATIONS!
TURN ON THE PUMP AND
SHOWER AWAY!



FOR MORE DETAILED INSTRUCTIONS
REFER TO THE USER MANUAL