




HOW TO FIT **DEPEND**[®] REAL FIT[®] UNDERWEAR FOR WOMEN, FOR MEN

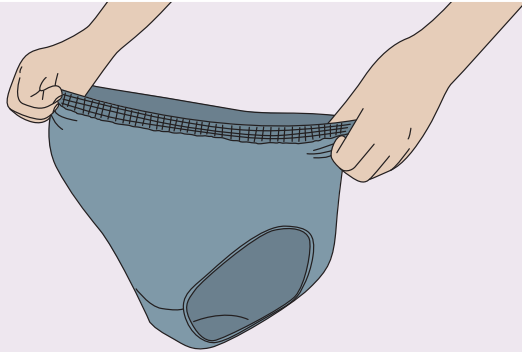


1. Begin by gathering from the waistband down to the leg elastic

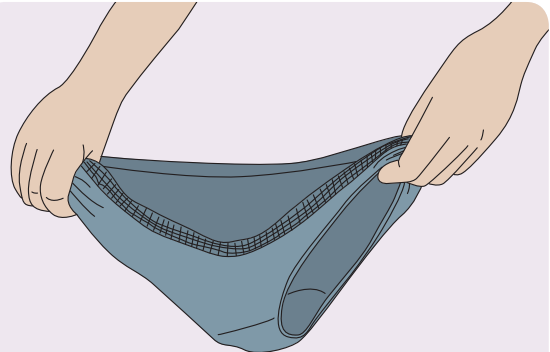


2. Bunch the gathering within the palm of your hand, holding firm the gathering

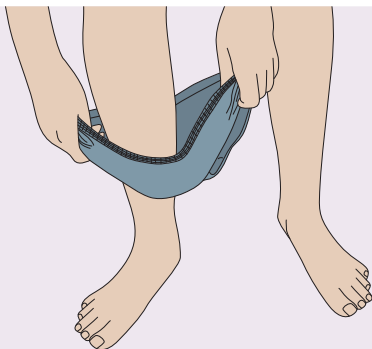

3. Feed the feet through each leg opening, one by one

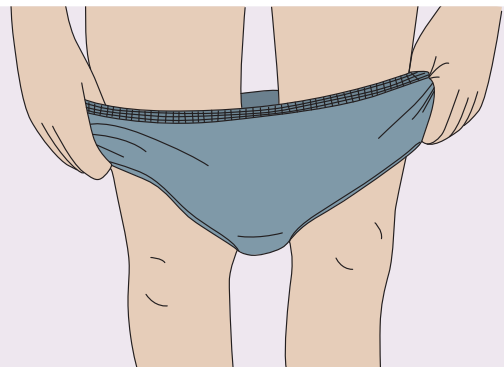

4. Pull the underwear up while still keeping the gathering within your palms

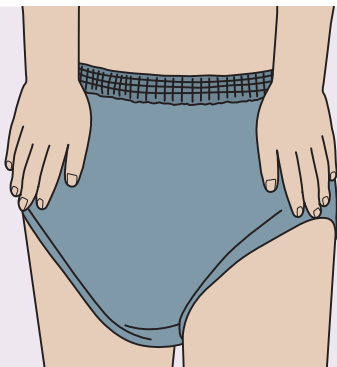


5. Pull over the pelvic area to fit and smooth the underwear out for comfort

Tip: Due to the soft cotton-like fabric, take care not to put your fingers through the body of the underwear




DEPEND® UNDERWEAR RANGE

DEPEND® REAL FIT® Underwear for Men, for Women

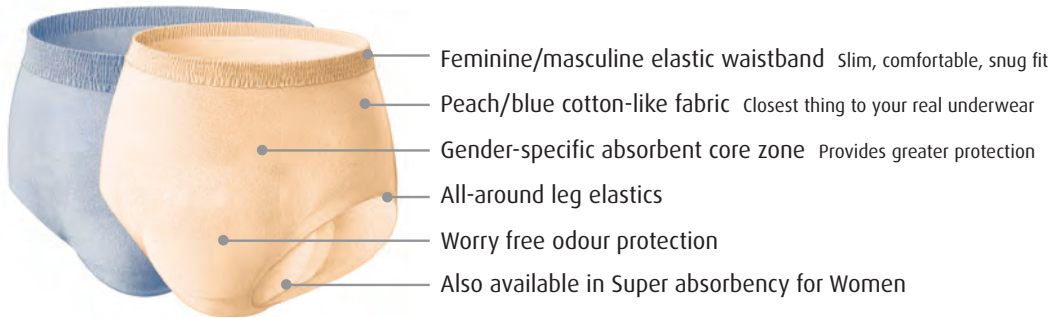

DEPEND® Underwear, Unisex

- Underwear-like waistband Soft & comfortable fabric
- Stretchy waistband Easy to fit
- Unisex design
- Maximum protection Absorbent core traps moisture & locks in odour
- Full length leak guards For extra protection
- All around leg elastics Protection while active
- Available in Super & Super Plus absorbencies

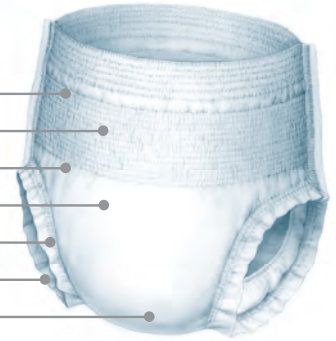

PROD. CODE	DESCRIPTION	SIZE	HIP SIZE (CM)	WAIST SIZE (CM)	WEIGHT (KG)	TOTAL CAPACITY*	PACKS/CASE	APN	TUN
19602	DEPEND® REAL FIT® Underwear for Women	S/M	86-117	71-102	52-86	920ml	10 x 4	9310088007909	19310088007890
19603	DEPEND® REAL FIT® Underwear for Women	L	112-137	97-127	77-117	920ml	9 x 4	9310088007916	19310088007906
19604	DEPEND® REAL FIT® Underwear for Women	XL	122-162	122-162	91-136	920ml	8 x 4	9310088007923	19310088007913
19605	DEPEND® REAL FIT® Underwear for Men	S/M	86-117	71-102	52-86	1320ml	8 x 4	9310088007930	19310088007920
19606	DEPEND® REAL FIT® Underwear for Men	L/XL	112-162	97-162	77-136	1320ml	8 x 4	9310088007947	19310088007937
19607	DEPEND® REAL FIT® Underwear Super for Women	S/M	86-117	71-102	52-86	1320ml	14 x 4	9310088008548	19310088008576
19608	DEPEND® REAL FIT® Underwear Super for Women	L	112-137	97-127	77-117	1320ml	13 x 4	9310088008555	19310088008583
19609	DEPEND® REAL FIT® Underwear Super for Women	XL	122-162	122-162	91-136	1320ml	12 x 4	9310088008562	19310088008590
19615	DEPEND® Underwear Super Unisex	S/M	71-102	71-102	-	1450ml	12 x 4	9310088919615	19310088819615
19616	DEPEND® Underwear Super Unisex	L	97-127	97-127	-	1450ml	12 x 4	9310088219616	19310088196167
19617	DEPEND® Underwear Super Unisex	XL	122-162	122-162	-	1450ml	12 x 4	9310088519617	19310088419617
19610	DEPEND® Underwear Super Plus Unisex	S/M	60-90	60-90	-	2000ml	10 x 4	9310088008524	19310088008552
19611	DEPEND® Underwear Super Plus Unisex	L	80-130	80-130	-	2000ml	9 x 4	9310088008531	19310088008569

* Total Capacity as per ISO 11948-1

For more information or for a free product sample, ask your Kimberly-Clark representative, Freecall 1800 028 334 or visit Health professionals dependprofessional.com.au

Consumers

depend.com.au