

MCKESSON

Forehead Temperature Strips

DISPOSABLE

For use with MFR # 16-TSTRP.

INSTRUCTIONS FOR USE

1. Temperature strip should be kept at room temperature for 5 minutes prior to use.
2. Ensure the forehead is clean and dry.
3. Place temperature strip on the forehead, just above the eyebrows, and wait until the color stops changing.
4. While the temperature strip is still against the forehead, read the temperature of green block. If there is no green block, the temperature is between the brown and blue colored blocks.

WARNING: For external use.

PRECAUTION: If you think your temperature differs from that shown on the forehead temperature strip, you should check by using an oral or rectal thermometer. Note that environmental temperature can influence the performance of this product.

Avoid storage in direct sunlight/fluorescent lighting and store in a cool, dry, and well-ventilated area. Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN A1117

Made in China