

# SURFACE DISINFECTION – CAVICIDE®

## SURFACE DISINFECTION USING CAVICIDE:

# Spray ➡ Wipe ➡ Spray

### STEPS 1, 2, 3 – CLEAN BEFORE DISINFECTING



1. Spray CaviCide directly onto surface to be cleaned.



2. Wipe with clean paper towel to remove debris and bioburden.



3. Discard towel.

### STEPS 4, 5, 6 – DISINFECT AFTER CLEANING



4. Spray CaviCide directly onto pre-cleaned surface, thoroughly wetting area. Follow label instructions for appropriate contact times.



5. Wipe with clean paper towel.



6. Discard towel.

### Use CaviCide to clean and disinfect:

- Potentially contaminated noncritical instruments and equipment.
- Hard, nonporous surfaces or items to prevent cross contamination, such as:
  - Light fixtures
  - Clinical and laboratory surfaces
  - Telephone and office machines

For a complete listing of uses, go to [metrex.com](http://metrex.com).

PART NO. US / Canada	PRODUCT DESCRIPTION	PACKAGING
13-1002	CaviCide – 2 fluid ounce spray	48 bottles/case
13-1008	CaviCide – 8 fluid ounce spray	12 bottles/case
13-1024 / 11-1024	CaviCide – 24 fluid ounce spray	12 bottles/case
13-1000 / 11-1000	CaviCide – 1 gallon	4 bottles/case
13-1025 / 11-1025	CaviCide – 2.5 gallon	2 bottles/case

Note: Follow standard precautions for personal protection as appropriate.



For more information, visit [metrex.com/education](http://metrex.com/education), or call **800.841.1428**.

Lit #77-1004-3