

Hand Control

The hand control is a completely sealed unit containing a rechargeable battery and battery state indicator light; it contains no user serviceable parts. DO NOT attempt to open.

Battery State Indicator


When either the 'Up' or 'Down' buttons are pressed, the indicator light (**A** fig.8) shows the state of the battery as follows:

GREEN - OK to use.

FLASHING GREEN - OK to use but the battery needs to be recharged after this bath.

RED - The battery needs recharging before use. The seat will not lower. **NOTE:** The microprocessor in the hand control will not allow the seat to be lowered if it detects that there will not be enough battery power to raise it again.

The battery is not replaceable. When it reaches the end of its working life, indicated by failure to accept or retain charge (frequently showing red on the battery state indicator), the complete hand control will need to be replaced with a new one. It should be disposed of in accordance with local regulations or returned to your supplier or to Mangar International for safe disposal. DO NOT dispose of the hand control with your normal household waste.

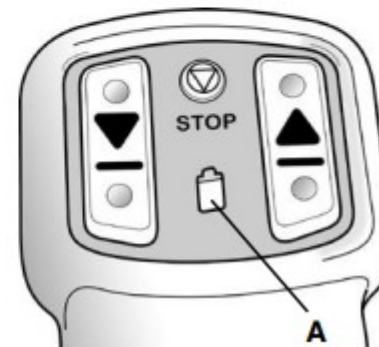

Battery State Indicator