

WHEELCHAIR LEG PAD


Reorder #	Description	Unit
<input type="checkbox"/> 703070	Wheelchair Leg Pad 18"	Each
<input type="checkbox"/> 703071	Wheelchair Leg Pad 20"	Each
<input type="checkbox"/> 703072	Wheelchair Leg Pad 20"- 24"	Each

PURPOSE

Foam-padded and rigid-backed, this vinyl-covered leg pad adds comfort and safety to any wheelchair. Skil-Care's leg pad protects sensitive legs against contact with metal wheelchair parts; serves as a cushioned leg rest when footrests are in the raised position; and, prevents your resident's legs from slipping off footrests.

Easy To Use

Place the leg pad across the footrest support bars, padded side facing out. Position the leg pad just above the footrests as shown in the illustration. Bring the straps around the support bars and behind the leg pad. Secure the buckle. Pull the loose end of strap through the buckle to take up any slack.

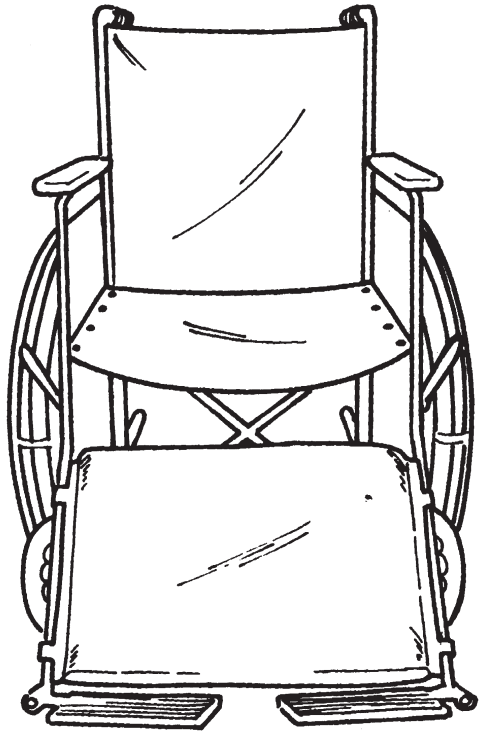

Easy to Clean

The durable vinyl cover cleans easily with a damp cloth or sponge. A household spray cleaner may be used.

[Click here to view detailed product descriptions and prices.](#)

[Call 800-397-5899 to order today!](#)