

P.A.D.S. Bed Sensor


The Personal Accident Deterrent System

Can be used with Secure® Universal Alarm (SUA-1), Secure® PADS 2


Alarm (PADS-2), Secure® Voice Alarm (SVA-1), Secure® Silent Monitor (SSM-1),

Secure® Code Alarm (SCA-1), Secure® 120 Alarm (SUA-120)

Model # Bed PAD-1 (12" x 30") or Bed PAD-3 (24" x 30") • Made in China

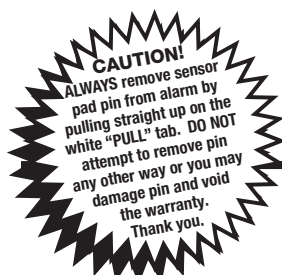

U.S. PATENT
NO. 6,847,301

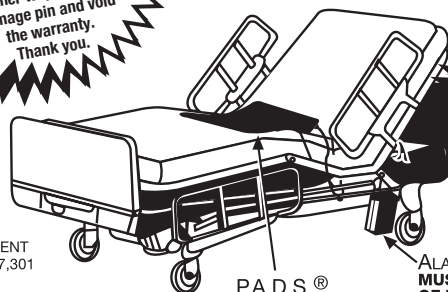

**Anti-Microbial
and Non-Skin
Sensitive
Covers!**

1. The alarm and sensor pad should be checked prior to each use for proper functioning. The "Test/Reset" button also functions as a battery tester.
2. Place P.A.D.S.® Bed Sensor widthwise across bed on top of mattress. The sensor pad must be positioned on the mattress to be directly under buttocks or shoulder of user. Place side of pad with movement resistant cushions down against mattress top. Mattress cover sheets will be placed over the sensor.
3. Install batteries (included) in Alarm by opening cover with screwdriver. Put sensor pad connecting pin into opening at top, front or pad jack opening of alarm. **CAUTION! ALWAYS remove sensor pad pin from alarm by pulling straight up on the white "PULL" tab. DO NOT attempt to remove pin any other way or you may damage pin and void warranty.**
4. Affix Alarm Holder securely to side of bed **out of sight and reach of the resident**. Simply wrap Velcro® strap with rubberized non-slip backing around railing or frame of bed. Holder placement should be easily accessible to staff. Put alarm in holder and fasten Velcro® closure of holder to Secure® alarm in holder. Secure excess cord to avoid damaging.
5. Carefully assist user into bed and on top of sensor pad.

6. When using the Secure® Universal Alarm (SUA-1) or the Secure® PADS-2 Alarm (PADS-2) with this pad push the "Test/Reset" button on alarm to test. The alarm will sound 4 times and shut off to indicate proper functioning. Alarm will allow for normal movement. However, if user gets off sensor pad the alarm will sound to alert staff. When user gets back on pad the alarm will automatically reset. PLEASE NOTE: If using a Secure® brand alarm other than the Secure® Universal Alarm (SUA-1) or the Secure® PADS-2 Alarm (PADS-2) with this pad please follow instructions for the particular alarm. Instructions for all Secure® brand alarms can be printed from our web site at www.padalarm.com.
7. When staff removes user from pad for whatever reason the alarm will sound. To silence alarm press "Test/Reset" button per alarm's instructions.

Cleaning Instructions

Use the same cleaning method as you use to clean vinyl mattress covers.

Water and Incontinent Proof

The P.A.D.S.® Sensor has been specially designed to be water and incontinent proof. If sensor is exposed to extreme heat, punctured or torn it will no longer be water and incontinent proof and should no longer be used. Warranty will be voided. Never submerge sensor and pad completely in water.

12 Month Limited Warranty

Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of 12 months from the date on the product. Void if this product is misused, abused or punctured. No warranty or responsibility of use expressed or implied.

CAUTION!

This product is designed as a monitor, alerting the staff when a patient has risen from their bed or chair. This product is not a restraint and therefore will not prevent a patient from getting out of their bed or chair. This product is to be used in conjunction with a total fall prevention program.

DO NOT ASSUME!

This product must be tested each time it is used with a patient to make sure it functions properly.

Please see our complete line of Secure® fall & wandering prevention products at www.PadAlarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873