


Pressure Reducing Elbow Pads

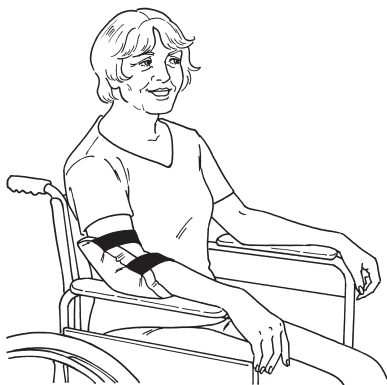

Reorder #	Description	Unit
<input type="checkbox"/> 503230	Visco-Foam Elbow Pad	Pair
<input type="checkbox"/> 503240	Visco-Foam Elbow Pad	36 Pr/Cs
<input type="checkbox"/> 503310	Synthetic Sheepskin Elbow Pad	Pair
<input type="checkbox"/> 503320	Synthetic Sheepskin Elbow Pad	36 Pr/CS

Purpose

Elbow Pads protect sensitive skin from breakdown caused by pressure. They also guard against skin tears resulting from contact with the wheelchair. These pads are as comfortable as they are protective.

Easy to Use

Place the pads over the patient's elbows. The stockinet bands keep the pads in place. (See illustration above.)

Easy Care

Synthetic Sheepskin Elbow Pads are completely launderable. Where as the Visco-Foam Elbow Pads aren't and could be wiped clean with a standard disinfectant. They may be washed and dried at temperatures that do not exceed 180°F (82°C). Do not use chlorine bleach.

[Click here to view detailed product descriptions and prices.](#)

Call 800-397-5899 to order today!