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REF 002620

AirLife™

en Prefilled Humidifier. Sterile Water for Inhalation, USP, 500 ml with Humidifier Adapter
Applicable for Canada only: Medicinal Ingredient: Water 1 g/ml

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 Product and packaging are not made with natural rubber latex

USA Rx Only

 For Single Use Only

STERILE

AirLife, CareFusion and the CareFusion logo are trademarks or registered trademarks of CareFusion Corporation or one of its subsidiaries.

© 2011-2013 CareFusion Corporation or one of its subsidiaries. All rights reserved.

NPN 80023548 1027441

Manufactured for
CareFusion
22745 Savi Ranch Pkwy
Yorba Linda, CA 92887 USA
CareFusion France 309 S.A.S.
8 bis rue de la Renaissance
44110 Châteaubriant, France 0086
Imported in Canada by
Cardinal Health Canada, Inc.
Mississauga, Ontario
L5T 1C2 Canada
Made in Mexico
36-10508 2013-10

Proofed by: _____ Date: _____
Dimensions checked: _____ Copy checked: _____

EF102913508-R1
Yorba Linda, CA
Grace Contreras
Nov 5, 2013
36-10508

en Sterile Water for Inhalation, USP, 500 ml with Humidifier Adapter

Approximate fluid contents 500 ml. No bacteriostatic or other preservatives added. • Sterile, nonpyrogenic. • Latex free • External surfaces of bottle not sterile. • Recommended storage: Room temperature (25°C or 77°F). Avoid excessive heat. Protect from freezing.

Directions for use:

1. Remove humidifier adapter from package. Remove break-away "trigger" from bottle top by sharply pulling up on "trigger". Secure humidifier adapter to bottle by turning in a clockwise direction until adapter has been completely seated to the bottle top.
2. Attach the humidifier adapter nut to the outlet of the flow meter. Remove break-away "trigger" from outlet of humidifier with an upward snapping motion. Do not twist off.

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3. Attach delivery tube to the outlet port of the humidifier.
4. To check audible relief alarm, set flow meter at 4 LPM, fold back and pinch tubing. When audible relief alarm is heard, release tubing and proceed with therapy.
5. Adjust liter flow in accordance with doctor's prescription.

Warning: Read humidifier bottle label prior to use. **Caution:** Do not use sterile water for inhalation for parenteral administration. **Contraindications:** Do not use unless solution is clear and container intact. **Caution:** For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.

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