

UNISEX HIP PROTECTOR

With Three Removable Pads

Model # SHP-RP-XSW, SHP-RP-SW, SHP-RP-MW,
SHP-RP-LW, SHP-RP-XLW, SHP-RP-XXLW • Made in China

Instructions

Secure® Hip & Tailbone Protectors are to be used as part of a total fall management program. Please note that no hip protector can prevent all injuries. The Secure® Hip Protector's three strategically positioned high impact absorbing foam pads will greatly assist in optimizing the potential to prevent serious injuries including fractured hips and tailbones as a result of a fall.

The Secure® Hip Protector easily fits over underwear. Made of 94% Cotton and 6% Spandex they can also be comfortably worn as underwear. The garment's unique memory foam pads will conform to the user's body contours. The memory foam pads will be undetectable when worn with regular, loose fitting clothing. Please note it may take some users one or two days to acclimate to wearing the protective garment.

Using a permanent ink pen write the user's name, room number and date of first use on the garment label and the three removable pad labels. Insert pads securely inside pockets of garment. Assist the user or have the user place one foot at a time into the leg openings and carefully pull the hip and tailbone protector up to align the waist band of the garment with the user's waist at the navel. Carefully adjust the hip protector's pads to make certain the pads adequately cover both hips and the tailbone of the user.

Washing Instructions

According to the CDC's Guideline for Laundry in Health Care Facilities found on their Web site (www.cdc.gov/od/ohs/biosfty/laundry.htm), "Hot water provides an effective means of destroying microorganisms." The guidelines further state: "Rather than rigid rules and regulations, hygienic and common-sense storage and processing of clean and soiled linen is recommended."

Please note that the removable high impact foam pads of the Secure® Hip Protectors are waterproof sealed. **FOR BEST RESULTS PLEASE REMOVE PADS WHEN WASHING OR DRYING THIS GARMENT.** If the pads

are removed prior to washing, please clean with a mild, liquid disinfectant prior to inserting pads in pockets. Never puncture or cut the waterproofed seal of the foam pads or the foam pads will be damaged. The garment can no longer be used and the 6-month limited warranty will be void. See 6-month warranty details below.

We recommend washing the garment in water with a temperature of at least 160°F (71°C). Dry at higher temperatures. Please note that drying at lower temperatures will prolong the life of the garment. Chlorine bleach provides an extra margin of safety according to the CDC. Bleach as directed on your bleach container.

Sizes Available:


Model #	Size
SHP-RP-XSW	Extra Small: Waist (24"-27") Hips (31"-34")
SHP-RP-SW	Small: Waist (28"-31") Hips (35"-38")
SHP-RP-MW	Medium: Waist (32"-35") Hips (39"-42")
SHP-RP-LW	Large: Waist (36"-38") Hips (43"-45")
SHP-RP-XLW	Extra-Large: Waist (39"-42") Hips (46"-49")
SHP-RP-XXLW	Extra-Extra-Large: Waist (42"-46") Hips (49"-53")

6 Month Limited Warranty

Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of 6 months from date of first use. Void if this product's waterproofed sealed foam pads have been misused, abused or punctured. No warranty or responsibility of use expressed or implied.

Replacement Pads Available (SHP-PAD)

\$1.99 each

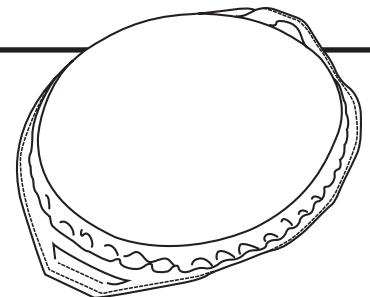

CAUTION!

There is no guarantee that the user will be protected against all injuries as a result of a fall. Please note in your procedures for using this product that the user's skin should be checked regularly. The garment must be washed after each incontinent event to protect against skin rash.

Please see our complete line of Secure® fall & wandering prevention products at www.PadAlarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873