

BEDSIDE SAFETY MAT TRI-FOLD DESIGN

30" x 66" x 2" — Model # - SBSM-1

The Secure[®] Tri-Fold Bedside Safety Mat has been biomechanically tested to reduce serious head injuries to 0% from an 18" sideways fall out of bed. See www.padalarm.com/bedsidefloormat.htm for complete test results. The Secure[®] Bedside Safety Mat has been designed to be used as part of a total fall management program. Please note that no bedside safety mat can prevent all injuries as a result of an accidental rolling out of bed. However, the mat has been designed with a two-inch high-density foam for maximum fall impact absorption. The mat will greatly assist in optimizing the potential to prevent serious injuries as a result of an accidental rolling out of bed.

UNIQUE FEATURES AND BENEFITS

- Slip-Resistant On Both Sides
- Latex-Free
- Mildew-Resistant
- Flame Retardant
- Two Inch High-Density, High-Impact Foam
- Tri-Fold For Easy Storage

HOW TO USE

The Secure[®] Bedside Safety Mat is a 30" wide x 66" long, which is an ideal size for maximum protection against injuries as a result of accidentally rolling out of bed. Position the safety mat lengthwise along the side of the bed with the front end of the mat positioned at the head of the bed. Make certain the lengthwise edge of the safety mat closest to the side of the bed is about 1" under the bed. This will insure that the user will roll directly onto the safety mat in the event of an accidental fall.

CLEANING INSTRUCTIONS

The Secure[®] Bedside Safety Mat was designed with a black outer cover to minimize the need for frequent cleaning. When necessary clean the mat with the same cleaning agents used to clean other vinyl materials used in your facility.

ONE YEAR LIMITED WARRANTY

Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of one year from date of purchase. Void if this product's outer covering is misused, abused, cut or punctured. No warranty or responsibility of use expressed or implied.

WARNING!

There is no guarantee the user will be protected against all injuries as a result of a fall. Please properly position the safety mat according to the instructions above with each use to provide maximum protection against serious injuries as a result of a fall.

Please see our complete line of Secure[®] fall & wandering prevention products at www.PadAlarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Revised 07/19/12

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873

[Click here to view detailed product descriptions and prices.](#) **Call 800-397-5899 to order today!**