

SEAT BELT ALARM

For Restraint-Free Fall and Wandering Prevention

To be used with the Secure® Quick-Release Seat Belt, SB-1 — Holder Options: Alert Mate® Holder, AMH-1 & Alert Mate® Holder (Mesh Front), AMH-3
Model # SBA-1 • Made in Taiwan

The Secure® Seat Belt Alarm was created to be used with Secure® Quick-Release Seat Belt. This alarm is currently used in thousands of concerned care facilities across the U.S. The alarm features:

- Always “ON” for resident safety.
- Flashing alert light will instantly identify roommate in need... day or night.
- OSHA approved 80-decibel sound level.
- Alerts staff without frightening resident.
- Two AAA batteries included.
- Low battery indicator light.
- Test & reset button.
- Velcro® attachment.



INSTRUCTIONS

1. **PLEASE NOTE: Carefully read and understand the Secure® Quick-Release Seat Belt instructions prior to use.**
2. Install two AAA batteries (included) in alarm by opening back cover with Phillips screwdriver. **PLEASE NOTE: Alarm will immediately sound once batteries are installed. To silence the alarm press “Test/Reset” button on side of alarm 3 times within 2 seconds.** Replace battery cover and tighten screws.
3. Insert metal pin of seat belt being used into opening at top of alarm. **CAUTION! ALWAYS remove seat belt pin from alarm by pulling straight up on the white “PULL” tab. DO NOT attempt to remove pin any other way or you may damage pin and void warranty.**
4. Put alarm in holder and fasten Velcro® closure of holder to secure alarm in holder.
5. Affix Secure® Alarm Holder (AMH-1) to bottom or back of wheelchair out of sight and reach of resident. Simply wrap Velcro® strap with rubberized non-slip backing around frame of wheelchair. **CAUTION: Do not affix alarm to a PVC rail. The strap may permanently stain the rail.** Holder placement should be easily accessible to staff but out of reach of resident. Secure excess cord of seat belt to avoid damage.
Note: If not using an alarm holder remove the paper backing of Velcro® strip attached to back of alarm.

Affix the alarm to wheelchair by using the adhesive backing of the Velcro® strip.

6. Carefully assist resident into wheelchair and seat belt as explained on instruction sheet included with seat belt.
7. Push “Test/Reset” button on side of alarm. The alarm will sound four times, stop sounding and automatically reset to sound in the event of an exit attempt. However, if resident pulls seat belt apart the alarm will sound to alert staff. If resident puts the seat belt back on or puts the two pieces back together the alarm will stop sounding and will automatically reset.
8. When staff removes resident from seat belt for whatever reason the alarm will sound. **To silence the alarm press “Test/Reset” button on side of alarm 3 times within 2 seconds.** When the resident is positioned back in the seat belt the alarm will automatically reset to the alarm mode. **Please note: For resident safety, the alarm does not have an “On/Off” switch. Staff members do not need to remember to turn the alarm back “on” as the alarm has an automatic reset feature.**
9. The alarm must be checked prior to each use for proper functioning. The “Test/Reset” button also functions as a battery tester. In addition the Low Battery Light on the front of the alarm will light up when it is time to change the batteries.

Please note: If using as a personal alarm with Pin/Cord & Clip, adjust cord to desired length. Insert pin at end of cord into opening at top of alarm. Attach cord to resident’s clothing with locking clip. When resident moves beyond desired range, pin will come out and alarm will alert staff.

WARNING: The Secure® Quick Release Seat Belt Alarm Set should not be used on residents or patients whose mental and/or physical abilities do not enable them to easily open the seat belt on their own or when directed to do so by a staff member.

WARRANTY: Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of one year from the date of purchase. Warranty is void if this product is misused, abused or damaged. No warranty or responsibility of use expressed or implied.

CAUTION!

This product is not a restraint. It will not prevent a resident from getting out of their bed or chair.
This product is designed as a monitor, alerting the staff when the resident has risen from their bed or chair.
This product is to be used in conjunction with a total fall prevention program

Please see our complete line of Secure® fall & wandering prevention products at www.PadAlarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Revised: 07/11/12

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873

[Click here to view detailed product descriptions and prices.](#) Call 800-397-5899 to order today!