

FLATPAD WATERPROOF BEVELED EDGE FALL SAFETY MAT – GRAY

36"X70"X3/4" • Model # - SBFP-2G • Made in PRC

DESCRIPTION

The Secure[®] FlatPad Waterproof Beveled Edge Fall Safety Mat has been designed to be used as part of a total fall management program. Please note that no safety mat can prevent all injuries as a result of an accidental rolling out of bed. However, this mat has been manufactured using a one-inch thick proprietary elastomeric foam core for maximum fall impact absorption. In addition, the low one-inch profile makes it easy for residents to step on and off the mat. The beveled edges will minimize a shuffling resident's potential to trip when walking up to and onto the mat. The beveled edges of the mat were also designed to easily roll a wheelchair onto the mat to safely transfer a patient or resident from the chair to the bed or vice versa. Staff can also roll feeding or medical supply tables onto the mat for easy access to the patient or resident. The mat will greatly assist in optimizing the potential to prevent serious injuries as a result of an accidental rolling out of bed.

INSTRUCTIONS FOR USE

1. Make note in the resident's care plan that a safety mat is being used. Staff members, the resident, a roommate or any visitors must exercise caution when stepping on or off the mat to avoid tripping.
2. The Secure[®] Beveled Edge Safety Mat is 36" wide x 70" long, which is an ideal size for maximum protection against injuries as a result of accidentally rolling out of bed. Position the safety mat lengthwise along the side of the bed and approximately 1" under the bed. This will insure that the user will roll directly onto the safety mat in the event of an accidental fall.
Note: If the mat is used with a low bed do not place the mat under the bed. Leave enough space so that the handrail will not damage the mat.
3. To store the mat simply roll up the mat and stow in a convenient location

CLEANING INSTRUCTIONS

The Secure[®] FlatPad Waterproof Beveled Edge Fall Safety Mat is designed with a gray waterproof cover to minimize the need for frequent cleaning. When necessary clean the mat with the same cleaning agents used to clean other vinyl materials used in your facility.

**MAT COVER IS
WATERPROOF &
ANTI-MICROBIAL**

ONE YEAR LIMITED WARRANTY

Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of one year from date of purchase. Void if this product's outer covering is misused, abused, cut or punctured. No warranty or responsibility of use expressed or implied.

WARNING!

There is no guarantee the user will be protected against all injuries as a result of a fall. Please properly position the safety mat according to the instructions above with each use to provide maximum protection against serious injuries as a result of a fall.

Please see our complete line of Secure[®] fall/wandering and personal safety products at www.PadAlarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Revised 11/30/15

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873

[Click here to view detailed product descriptions and prices.](#) **Call 800-397-5899 to order today!**