



# TRANSFER & WALKING BELT WITH METAL BUCKLE



Model No. STWBM-60G | Fits Waist Sizes 28"-52" | Made in PRC

## Instructions

The Secure® Transfer & Walking Belt helps prevent caregiver back injuries and assist in safe patient transfers and ambulation. The Transfer & Walking Belt is wider than traditional gait belts for greater contact area with the patient. Each belt comes with multiple vertical and horizontal hand grips that allow the caregiver to choose the best leverage point.

The Secure® Transfer and Walking Belt is to be used as part of a total fall management program. Please note that the belt should only be used as authorized by the results of the resident's risk assessment. The Secure® Transfer & Walking Belt is to be used as an effective aid when lifting, walking with or transferring a resident or patient. **AT NO TIME IS THE BELT TO BE USED AS A RESTRAINT TO SECURE A RESIDENT TO A BED, CHAIR OR A STATIONARY OBJECT OF ANY KIND.**

**VERY IMPORTANT! ASSESS WHETHER THE RESIDENT'S SIZE AND/OR WEIGHT WILL REQUIRE MORE THAN ONE PERSON TO SAFELY LIFT, WALK WITH OR TRANSFER RESIDENT. WHEN IN DOUBT, TWO CAREGIVERS SHOULD ALWAYS BE USED FOR RESIDENT SAFETY.**

1. Position the belt around the resident's waist. **DO NOT POSITION BELT AROUND RESIDENT'S CHEST.**
2. Adjust belt length to fit securely around the waist. Make sure the belt is snug, but not uncomfortable. You should be able to slide your open hand (flat) between the belt and the patient.

3. Secure the belt length through the metal buckle. The belt is now positioned for use.
4. Use both hands to firmly grip the belt using the attached hand grips located along the back of the belt. If two caregivers are required, position one caregiver on the right side and one caregiver on the left side of the resident. You are now ready to safely lift, walk with or transfer the resident or patient.

## Cleaning Instructions

The Secure® Transfer & Walking Belt can be laundered at temperatures up to 160°F (71°C). Do not bleach. The belt may be dried on the low setting. The specially designed buckle will withstand temperature extremes. However, do not use a water extractor as it may damage the buckle.

**Warning: NEVER** use a Gait Belt if there is any compromise of safety or indication of inappropriateness to the patient or caregiver. **ALWAYS** test to make sure the belt is applied securely and will not loosen when transferring patient or resident. **INSPECT** before each use by checking for broken stitches, torn material, or cracked buckles.

## One Year Limited Warranty

Personal Safety Corporation warrants this product to be free from factory defects and workmanship for a period of one year from date of first use. Void if this product has been misused, abused or cut. No warranty or responsibility of use expressed or implied.

## WARNING:

**NEVER** use a Gait Belt if there is any compromise of safety or indication of inappropriateness to the patient or caregiver. **ALWAYS** test to make sure the belt is applied securely and will not loosen when transferring patient or resident. **INSPECT** before each use by checking for broken stitches, torn material, or cracked buckles.

Please see our complete line of Secure® fall & wandering prevention products at [www.PadAlarm.com](http://www.PadAlarm.com).

**Highest Quality...Lowest Prices...Guaranteed!**

Revised: 10/27/16

**Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873**