

# SecureSleeves™ for Arms

**Small • Medium • Large • Extra-Large**

**Model #s - ARMS-S Brown, ARMS-M Brown, ARMS-L Brown & ARMS-XL Brown, ARMS-S Ivory, ARMS-M Ivory, ARMS-L Ivory & ARMS-XL Ivory**



The super soft SecureSleeves™ are designed to protect sensitive skin from tears and abrasions while providing ultimate wearer comfort. Made from a proprietary blend of Chitosan rayon, nylon, cotton and Spandex, SecureSleeves™ anti-microbial properties protect against odor causing bacteria often-accompanying deep wound and skin problems. In addition the anti-microbial properties of Chitosan rayon protects against the growth of bacteria, molds and fungi. SecureSleeves™ can also be used to cover I.V. and wound sites. Machine washable and Latex-free.




**Ultra-Soft!**



Step 1



Step 2

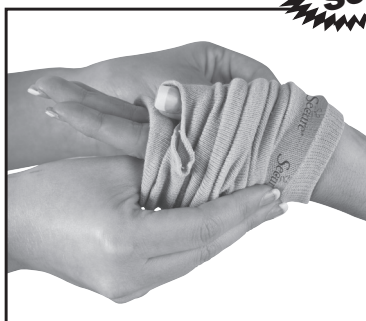

Step 3



Step 4

## INSTRUCTIONS FOR USE

1. Hold the SecureSleeve™ between your two hands with the thumb hole opening at the top and facing away from you and the Secure® logo band opening hanging down. See Step 1 photo above.
2. Insert your thumbs between material and in both sides of the top opening. Using the fingers of both hands begin to pull the material upward in an accordion-like fashion until you can no longer gather the material. See Step 2 photo above.
3. Carefully slip the SecureSleeve™ over the patient's/resident's hand placing the patient's/resident's thumb in the sleeve's thumb hole and continue pushing the sleeve upward until the patient's/resident's fingers are fully exposed at the bottom of the sleeve. See Step 3 photo above.
4. Slowly and carefully begin pushing the sleeve upwards releasing the accordion pleats until the patient's/resident's arm is covered. Adjust the sleeve to achieve maximum patient/resident comfort. See Step 4 photo above.

Some patients/residents may prefer to have their hands completely free from the sleeve. If so, carefully lift up on the material surrounding the patient's/resident's thumb and remove the thumb from the sleeve's thumb opening. The lower part of the sleeve can then be pushed up to the patient's/resident's wrist completely freeing the patient's/resident's hand from the sleeve. Caution: Make certain the bottom of the sleeve is not binding around the patient's/resident's wrist. Check for a comfortable, non-binding fit.

## CAUTION

**Conduct frequent and careful visual checks of the patient's/resident's skin to observe changing conditions of existing skin conditions being treated and take appropriate treatment action for any new skin problems. Follow facility policy with respect to replacing or laundering patient's/resident's SecureSleeve™.**

## CLEANING INSTRUCTIONS

Machine wash at lukewarm (110°F, 38°C) and dry at low setting. Hang dry for longer sleeve life.



**Please see our complete line of Secure® fall/wandering prevention & patient safety products at [www.SecureSafetySolutions.com](http://www.SecureSafetySolutions.com)**

**Highest Quality...Lowest Prices...Guaranteed!**

Revised 02/15/17

**Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873**

[Click here to view detailed product descriptions and prices.](#)

**Call 800-397-5899 to order today!**