


Foot-Drop Boot

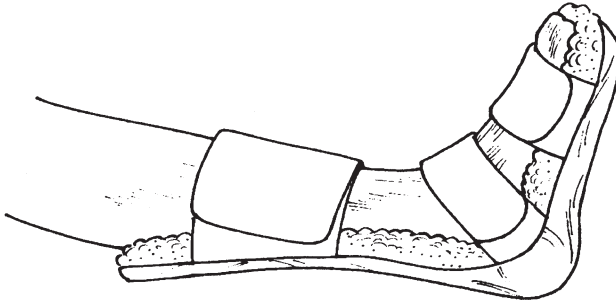

Reorder #	Description	Size	Shoe Size	Unit
☐503170	Foot-Drop Boot	Small	Men's 5-7, Women's 3-6	Each
☐503171	Foot-Drop Boot	Medium	Men's 7-9, Women's 6-9	Each
☐503172	Foot-Drop Boot	Large	Men's 9-12, Women's 9-11	Each
☐503174	Adjustable Foot-Drop	Medium	Men's 7-9, Women's 6-9	Each
☐503180	Replacement Cover	Small	Men's 5-7, Women's 3-6	Each
☐503181	Replacement Cover	Medium	Men's 7-9, Women's 6-9	Each
☐503182	Replacement Cover	Large	Men's 9-12, Women's 9-11	Each

Purpose

Designed with a supportive plastic inner splint that controls plantar flexion to reduce foot drop. Patient's heel is suspended above a molded cavity for maximum pressure reduction. Synthetic sheepskin liner adds comfort while providing protection against skin shear.

Application

1. Select proper size for patient. See size chart above.
2. Place patient's foot in boot so that heel is suspended over cavity.
3. Secure Velcro[®] straps.

Laundering

Remove cover. Do not launder inner splint. The Foot-Drop Boot cover may be laundered and dried using standard white goods procedures. Do not wash or dry at temperatures above 180°F (82°C). The use of bleach is not recommended.

[Click here to view detailed product descriptions and prices.](#)

Call 800-397-5899 to order today!