

Re-Order #	Description	Size	U/M
703280	Footrest Extender	16"-18" x 1"	Set
703282	Footrest Extender	16"-18" x 3"	Set
703283	Footrest Extender	16"-18" x 4"	Set



Note: All models above fit 16"-18", 18"-20" and 20"-22" Wheelchairs and Geri-Chairs

original
SkiL-Care
CORPORATION
design

Re-Order #	Description	Size	U/M
703210	Footrest Extender	18"-20" x 1"	Set
703220	Footrest Extender	18"-20" x 2"	Set
703230	Footrest Extender	18"-20" x 3"	Set

[Click here to view detailed product descriptions and prices.](#)

Call 800-397-5899 to order today!

Purpose

SkiL-Care's Drop-Stop is designed to solve two important problems: It provides support to control foot drop and it prevents residents' feet from slipping off footrests. The Drop-Stop is designed to fit 16-inch, 18-inch and 20-inch wide wheelchairs and geri-chairs.

Application

1. Before seating resident, lock wheel brakes and rotate footrests to the out-of-the-way position. Assist patient into chair and position him/her as far back on the seat as possible. Reposition footrests.
 2. Place Drop-Stop on footrests. Removeable cushion section on footrest, padded backboard against footrest support bars (*Fig. 1*).
 3. Place straps behind footrest support bars, secure buckles and tighten straps (*Fig. 2*).
 4. The Drop-Stop may be positioned to accommodate various degrees of ankle plantar flexion. This may be accomplished by securing the top straps to the upper section of the footrest support bars as shown in the illustration (*Fig. 3*).
 5. Important Safety Note. Remove the Drop-Stop and rotate the footrests to the out-of-the-way position when assisting your resident from the chair. Do not permit your resident to stand with footrests and Drop-Stop in place.
- Replacement footrest cushions are available in one-, two-, three- and four-inch heights.

Easy Care

SkiL-Care's Drop-Stop is made from durable vinyl for lasting use. To clean, just spray and wipe with a damp cloth. Accumulated lint and other particles may be removed from the hook-and-loop material with a small stiff brush.

Fig. 1

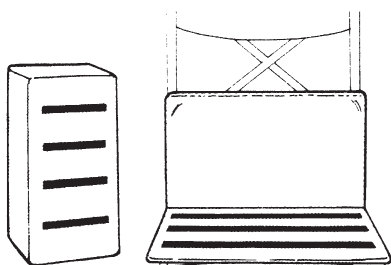


Fig. 2

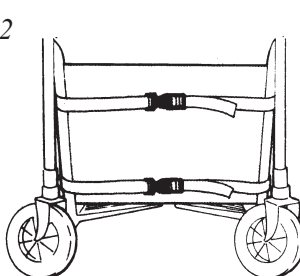


Fig. 3

