


Geri-Chair Foot Cradle

Reorder #	Unit
703430	Each

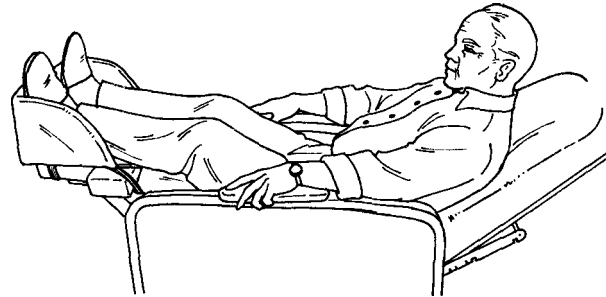

Purpose


The Geri-Chair Foot Cradle is designed to control foot drop and prevent a resident's feet from slipping off the sides of an elevated geri-chair footrest.

Important notice


This product is designed and intended to be used only when the geri-chair is in the reclined position and the footrest is elevated. It must be removed before the geri-chair is placed in the upright position.

Application

1. Place the Foot Cradle on the elevated footrest and position the forward strap as shown in illustration A. Secure the buckle and take up as much slack as possible.
2. Place the other strap under the bottom part of the footrest as shown in illustration B. Secure the buckle and take up the slack.

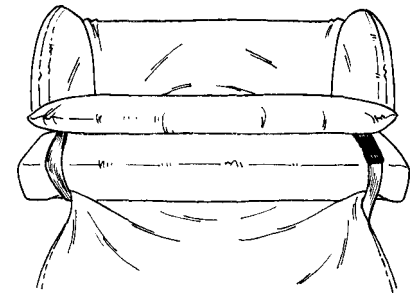

A

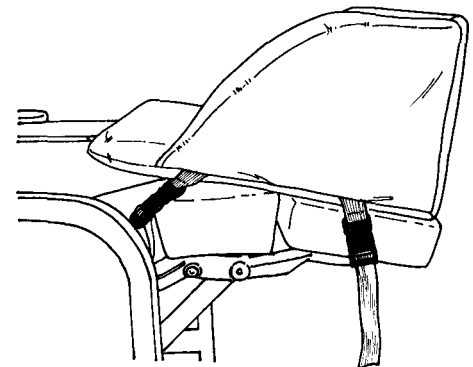

B

Easy Care

The Foot Cradle wipes clean using a damp cloth sponge or sponge and a spray-type cleanser. A mild disinfectant may be used.

[Click here to view detailed product descriptions and prices.](#)

Call 800-397-5899 to order today!