


Modesty Apron

| <u>Reorder#</u> | <u>Unit</u> |
|-----------------|-------------|
| 707030 | Each |

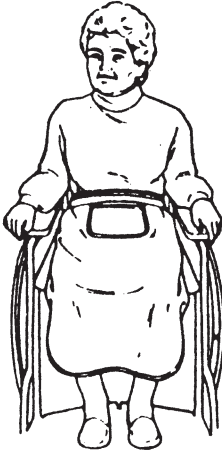

Purpose

Skil-Care's Modesty Apron provides dignity and convenience for female residents in wheelchairs or geri-chairs. This full-sized apron is an appreciated modesty cover that extends well past the knees. It features a large pocket that provides easy access to personal items

Easy to Use

Place the apron at the resident's waist and secure the slide-look buckle. Notice: This apron is neither designed nor intended to be used as a physical restraint. Do not secure strap around the back of the chair. Doing so will impede the resident's autonomy.

Easy Care

The Modesty Apron may be laundered and dried at temperatures up to 180°F (80°C). Do not use bleach.

[Click here to view detailed product descriptions and prices.](#)

Call 800-397-5899 to order today!