

TOILET SEAT SENSOR PAD ALARM SYSTEM

Anti-Microbial
and Non-Skin
sensitive
covers!


Can be used with Secure® Alarm Universal (SUA-1), Secure® 120 Alarm (SUA-120), Secure® PADS®-2 Alarm (PADS-2), Secure® Voice Alarm (SVA-1), Secure® Code Alarm (SCA-1)
Model # STP-1 • Made in China

The Secure® Toilet Seat Sensor Pad Alarm System was designed to be part of a total fall management program. As many as 3 out of 4 people in nursing homes fall each year. Potential falls in bathrooms pose a unique challenge in balancing privacy and dignity with effective fall interventions. Certainly installing grab bars next to the toilet and adding raised toilet seats can assist in preventing bathroom falls.

The Secure® Toilet Seat Sensor Pad Alarm System was designed to supplement toilet grab bars and raised toilet seats. The system was particularly designed as a bathroom fall intervention device that has marked potential to also provide privacy and dignity to the user. Depending on an individual's risk factor assessment the system may be used with the bathroom door closed to provide user privacy and dignity. **Please note that high-risk individuals must be visually monitored at all times.**

Instructions:

1. The toilet seat sensor pad has adhesive squares on the bottom of the pad. Remove the paper backing on the adhesive squares. Carefully position the pad lengthwise over the raised "bumper" on the underside of the toilet seat as shown in Illustration A. Press down firmly on the pad over each adhesive square to insure that the pad is securely affixed to the underside of the toilet set.
2. Remove the paper backing from the two plastic pieces on the pad's connecting wire. Secure the wire to the underside of the toilet seat using these wire hold pieces as shown in the picture above.
3. **Please Note:** If using the Secure® Universal Alarm (SUA-1), Secure® 120 Alarm (SUA-120), Secure® PADS®-2 Alarm (PADS-2), Secure® Voice Alarm (SVA-1) or Secure® Code Alarm (SCA-1) Secure® Silent Monitor (SSM-1), see instruction sheet included with alarm.
4. Affix the alarm to the toilet tank using the Velcro backing of the alarm. **PLEASE NOTE: The alarm may also be affixed to the toilet with our Secure® Universal Bracket (SUB-1) and Alarm Holder (AMH-1). Contact your representative for details.**
5. Connect the metal pin at the end of the pad's wire to the alarm. The system is ready to use. **CAUTION! Always**

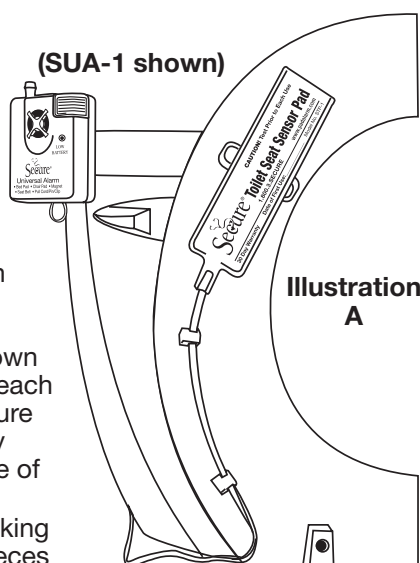

remove sensor pad pin from the alarm by pulling straight up on the white "PULL" tab. DO NOT attempt to remove pin any other way or you may damage pin and void warranty.

6. Push "Test/Reset" button on alarm to test. The alarm will sound to indicate proper functioning. Lower the toilet seat into normal position. Sit on the toilet seat or press down on the seat with your hands. Stand up or release your hands. The alarm will sound until you sit back down on the seat, press on the seat with your hands or press the reset button. This completes the test. The system is now ready to use.

Please note: For resident safety, the alarm does not have an "On/Off" switch. Staff members do not need to remember to turn the alarm back on as the alarm has an auto reset feature.

7. **DO NOT ASSUME!** The alarm and toilet seat pad must be checked for proper functioning prior to each use.

30 Day Limited Warranty

Personal Safety Corporation warrants the Toilet Seat Sensor Pad to be free from factory defects in materials and workmanship for a period of 30 days from date of first use. Please use a permanent marker to write in the date of first use on the line provided on the front of the sensor pad. Void if sensor pad is misused, abused or punctured. No warranty or responsibility of use expressed or implied.

NOTE: All Secure® alarm boxes carry a one year warranty.

Washing Instructions

Use the same cleaning method as you use to clean vinyl mattress covers.

WARNING

This product is designed as a monitoring device, alerting staff when user has risen from the toilet. Depending on the user's risk assessment the bathroom door may be closed to give the user privacy and preserve his or her dignity. **IN ALL CASES HIGH RISK USERS MUST BE VISUALLY MONITORED AT ALL TIMES.** This product is not a restraint and therefore will not prevent a user from getting off the toilet. This product is to be used in conjunction with a total fall prevention program.

Please see our complete line of Secure® fall & wandering prevention products at www.PadAlarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873

[Click here to view detailed product descriptions and prices.](#)

Call 800-397-5899 to order today!

Revised 07/16/12