


UNDER MATTRESS BED SENSOR PAD

Can be used with Secure® Alarm Universal (SUA-1), Secure® 120 Alarm (SUA-120), Secure® PADS®-2 Alarm (PADS-2), Secure® Voice Alarm (SVA-1), Secure® Silent Monitor (SSM-1) or Secure® Code Alarm (SCA-1)
Model # SUMP-1 • Made in China

Anti-Microbial
and Non-Skin
Sensitive
Covers!

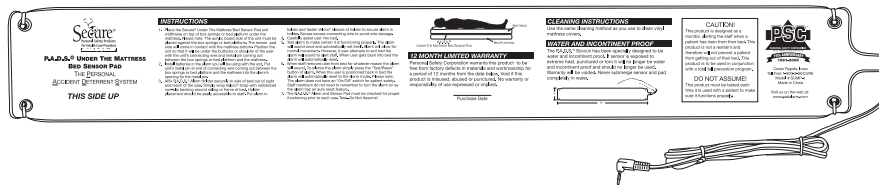

CAUTION!
ALWAYS remove sensor pad pin from alarm by pulling straight up on the white "PULL" tab. DO NOT attempt to remove pin any other way or you may damage pin and void the warranty. Thank you.

1. Place the Secure® Under Mattress Sensor Pad widthwise on top of box springs or platform under the mattress. Please note: The acrylic board side of the unit must be placed against the box springs or bed platform. The sensor pad side will come in contact with the mattress bottom. Position the unit so that it will be under the buttocks or shoulders of the user, as shown in Illustration A, with the unit's connecting wire and metal pin coming out between the box springs or bed platform and the mattress.

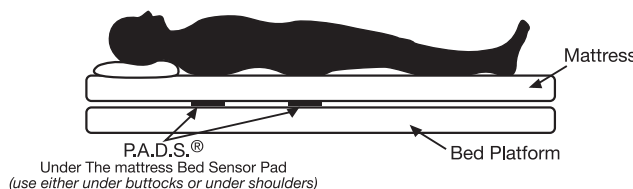


Illustration A

2. **Please Note:** If using the Secure® Universal Alarm (SUA-1), Secure® 120 Alarm (SUA-120), Secure® PADS®-2 Alarm (PADS-2), Secure® Voice Alarm (SVA-1) or Secure® Code Alarm (SCA-1) Secure® Silent Monitor (SSM-1), see instruction sheet included with alarm.
3. Put unit's metal pin at end of connecting wire coming out between the box springs or bed platform and the mattress into the alarm's opening for the metal pin. **CAUTION! Always remove sensor pad pin from the alarm by pulling straight up on the white "PULL" tab. DO NOT attempt to remove pin any other way or you may damage pin and void warranty.**
4. Affix P.A.D.S.® Alarm Holder securely to side of bed out of sight and reach of the user. Simply wrap Velcro® strap with rubberized non-slip backing around railing or frame of bed. Holder placement should be easily accessible to staff. Put alarm in holder and fasten Velcro® closure of holder to secure alarm in holder. Secure excess connecting wire to avoid wire damage.
5. Carefully assist user into bed.
6. Test alarm to make certain it is functioning properly. The alarm will sound and automatically set itself. Alarm will allow for normal movement. However, if user attempts to exit bed the alarm will sound to alert staff. When user gets back into bed the alarm will automatically reset.
7. When staff removes user from bed for whatever reason the alarm will sound. To silence the alarm simply press the "Test/Reset" button per alarm's instructions. When the user is positioned back in bed the alarm will automatically reset to the alarm mode. **Please note: For resident safety, the alarm does not have an "On/Off" switch. Staff members do not need to remember to turn the alarm "on" as the alarm has an automatic reset feature.**
8. **DO NOT ASSUME! The P.A.D.S.® Alarm and Sensor Pad must be checked for proper functioning prior to each use.**

ONE YEAR LIMITED WARRANTY

Personal Safety Corporation warrants this product to be free from factory defects in materials and workmanship for a period of 12 months from the date below. Void if this product is misused, abused or punctured. No warranty or responsibility of use expressed or implied.

WARNING

There is no guarantee the user will be protected against all injuries as a result of a fall. Please properly position the safety mat according to the instructions above with each use to provide maximum protection against serious injuries as a result of a fall.

Please see our complete line of Secure® fall & wandering prevention products at www.PadAlarm.com.

Highest Quality...Lowest Prices...Guaranteed!

Personal Safety Corporation • Hiawatha, Iowa 52233 • 800.373.2873