CapTel® 2400iBT





CapTel Acceptable Use Policy

The CapTel phone is intended for use by people with hearing loss. People who do not need captioning support should only use CapTel with the CAPTIONS feature turned off.

CapTel 2400iBT Specifications

Physical Dimensions

Size: 12.56" wide \times 9.54" tall \times 3.64" deep (Dimensions with the stand folded against the phone.) Weight: 3 lbs. 2.3 oz.

Power

AC Adapter: 5.9-6.0V 2Amps (barrel is negative) Adapter is UL listed.

Dial Pad

Large keys: 0.67" × 0.56"

Display

10.1" Backlit, graphical LCD color display 1024×600 pixels Resistive touch panel

Function Buttons

Volume Speaker Redial Flash Mute Customer Service speed dial Home Captions

Controls

Conversation: Amplification when Captions on: 0-40dB gain when Captions off: 0-18dB gain Tone: low, medium, high Customizable to match personal audiogram

Ringer

Ringer Volume: off, low, medium, high (0-90dB at 1 meter) Ringer Tone: low, medium, high

Indicator Lights

Mute Volume Levels (1 through 12) Captions on/off Ring Flasher

Handset

Type: HAC (Hearing Aid Compatible)

Dialing

Phone Book (100 entries) Favorites (10 entries) Speed Dial (4 entries) Redial (Last number dialed)

Captions

Nine (9) Caption font sizes: Small

1 – 0.20" (5 mm) 2 – 0.23" (6 mm)

3 – 0.25" (6.5 mm)

Medium

4 - 0.30" (7 mm) 5 – 0.33" (9 mm)

6 – 0.48" (10.5 mm)

Large

7 – 0.51" (12 mm) 8 – 0.54" (13.5 mm)

9 – 0.90" (22 mm)

Adjustable colors for captions font and background Five (5) font style options

Connections

Telephone line: requires RJ11 connection (can be standard analog line, VOIP, DSL with filter, FIOS. Does not support digital PBX system unless analog port available). 3.5 mm Headset with microphone Bluetooth® connectivity Ethernet or WiFi wireless network AC Power

Approvals

ACTA Product-Labeling Number: US:D8KTE00BCAP2400 FCC ID: D8K-CAPTFI 2400 FCC approved: Part 15B RFN = 0.0B

Conversation Memory

360,000 characters (minimum of 10 hours of conversation) per call

Caller ID History

Info saved for last 100 calls

Answering Machine

100 recordable messages Default or personalized greeting Selectable number of rings before answer Max message length: 90 seconds

NOTE: Your CapTel phone will periodically connect to the Captioning Service through the Internet and/or phone line connection to report its status and check for software updates.

Congratulations on Choosing CapTel!

Welcome to the Captioned Telephone – CapTel! With CapTel, you can see captions of everything people say over the phone, letting you enjoy calls with confidence.

Your CapTel phone is designed to be easy to use. It works just like any other phone. You can dial on the number keypad and listen on the handset. Or, you can use the easy touch screen display to make and answer calls. Whatever works best for you.

If you ever have any questions, please refer to this guide to learn:

- How to start using your CapTel 2400iBT
- How your CapTel 2400iBT works
- How to get help when you have questions

We want the CapTel phone to be enjoyable and convenient for you. As you use your new phone, we welcome your questions and comments. If we can help, please call.

Thank you for choosing the CapTel captioned telephone.

Sincerely,


CapTel Customer Service 1-888-269-7477 www.CapTel.com

TABLE OF CONTENTS

Overview - How CapTel Works	4
Section 1: Getting Started	5
What is in the box	
Overview of CapTel 2400iBT phone	6
Setting up your CapTel 2400iBT	
Using a headset or neckloop (optional)	
Connecting a Bluetooth headset/neckloop (optional)	
Section 2: Making and Answering Calls	25
Dialing a phone number directly	
Answering an incoming call	
Answering a call on a Bluetooth headset/neckloop	
Ending a call on a Bluetooth headset/neckloop	
Turning captions on/off during a call	
Adjusting the volume	31
Adjusting the Tone Control	32
Customizing the Tone setting	33
Using the Speakerphone	
Knowing what sounds are on the line	
Section 3: Viewing Captions	37
Turning captions on/off during a call	38
Viewing corrections	39
Reviewing captions during a call	40
Reviewing captions after hanging up	41
Saving conversation captions	42
Erasing captions	43
Changing the color of the captions	44
Changing the captions font size and style	46
Setting how captions scroll across the Display Screen	47
Adjusting the brightness of the Display Screen	48
Getting help	49
Section 4: Call History	50
Reviewing previous calls	51
Reviewing captions from a previous call	52
Dialing from Call History	53
Erasing previous calls	54

00	Section 5: Answering Machine	55
	About the Answering Machine	
	Turning Answering Machine on/off	56
	Playing your Answering Machine messages	57
	Erasing Answering Machine messages	58
	Deleting all messages	59
	Recording a personal greeting	60
	Seeing captions of voice mail messages	61
	Captioning messages from an external answering machine	62
	Setting the ring count	
•	Playing incoming messages aloud/silent	64
	Section 6: Contacts	65
	Adding a new contact	
	Dialing a phone number in your Contact List	
	Editing an existing contact	
	Removing a contact	
	Adding a photo to a contact	
	Adding a contact to Speed Dial	
	Section 7: Favorites	83
	About Favorites	
	Adding a contact to Favorites	85
	Dialing Favorites	
	Removing a contact from Favorites	86
	Section 8: Settings	87
	Setting Display Brightness88 Keypad lights	
	Screen timeout88 Language	
	Night mode88 Setting the ringer volume	
	Connections88 Adjusting the ringer pitch	92
	WiFi settings89 Tone control	92
	Bluetooth89 Time & date	92
	Saving captions after hang up89 About your CapTel phone	
	Captions button90 Updating your software	
	Ring flasher90 Advanced settings	93
	Saving conversation volume90	0.4
	Troubleshooting	
	Index	
	FCC	70

How CapTel Works

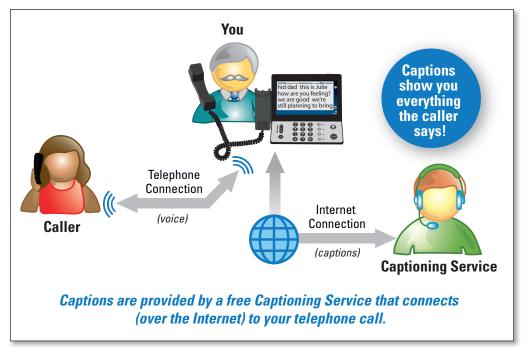

Your new CapTel 2400iBT works like any other phone – just pick up the handset and dial the person you want to call on the dialing keypad. Every time you make a call, the phone also connects over the Internet to a free Captioning Service. An operator (Captioning Assistant) at the Captioning Service uses speech-recognition technology to create captions of everything your caller says. The captions appear on your CapTel 2400iBT display screen for you to read.

You can enjoy conversations over the phone, with the added help of written captions.

E TE

The CapTel 2400iBT phone does not generate captions of the phone call on its own. The phone works with the Captioning Service (over the Internet connection) to display captions during your calls.

Getting Started

This section explains how to start using your new CapTel 2400iBT phone.

TOPICS

- What is in the Box
- Overview of CapTel 2400iBT
- Setting up Your CapTel 2400iBT
- Using a Headset or Neckloop (Optional)
- Connecting a Bluetooth Headset/Neckloop (Optional)

What is in the Box?


Your CapTel 2400iBT includes the following parts:



- CapTel 2400iBT Phone
- Telephone Cord



 Power Adapter to plug into a

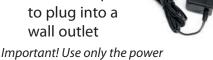

adapter that came in the box with your CapTel 2400iBT

- Ethernet Cable to connect to Internet service (not necessary for WiFi installations)
- CapTel Setup Guide and CapTel User Manual





Overview of CapTel 2400iBT Phone



Please see the descriptions that correspond with the matching numbers on the next few pages to help explain what each feature is.



Touch Screen Display

Shows captions, Caller ID, Answering Machine messages, Contacts, and more. You can touch an item on the Display Screen to select it.



2 Time & Date



Ring Flasher

Bright lights blink to alert you to any incoming calls and new answering machine messages.



4 Settings

Lets you control and adjust the features of your phone.



Contacts

Allows you to store and quickly dial people you call often. Saves up to 100 contacts.



6 Favorites

Saves the phone number of people you call most often for quick and easy dialing.



Bluetooth® Connectivity

You can listen to conversations using a Bluetooth headset or neckloop (sold separately).



8 Call History

Shows your most recent incoming and outgoing calls, plus lets you read captions from previous phone calls.



Answering Machine

Turns the Answering Machine on/off and lets you check messages – with captions.



Wolume Button

Lets you adjust the volume to a comfortable level, up to a 40dB gain from min/max. To increase the volume, press on the VOL button. Press on the VOL button to decrease the volume. The indicator lights next to the volume bar indicate the volume setting.

The maximum volume setting on captioned calls is extremely loud. People who do not need amplification should not use the phone at the highest volume level.

> The volume returns to a mid-range setting every time you hang up the handset. To save your volume setting so that it will be used during every captioned call, see "Saving Conversation Volume" on page 90.



SPEAKER Button

Lets you talk/listen over the built in speaker, to enjoy hands-free conversations.

🖁 🔒 🕲 Dialing Keypad

Use to dial telephone numbers – just like with any other phone.



REDIAL Button

Lets you dial the most recently called number.



1 FLASH Button

Some telephone services like Call Waiting require a "hook flash". The hook flash is a very brief interruption of the connection, like hanging up the phone for one second. There is a brief click or silence on the line.



15 MUTE Button

Silences the sound from your end of the conversation. When the MUTE button is pressed (blue light around the button is ON), the person on the other end of your call will not be able to hear you. You will still be able to hear the caller and get captions of everything they say. To turn the Mute feature off, press the MUTE button again (light around the button is OFF).



10 CUSTOMER SERVICE Button

Pressing the CUSTOMER SERVICE button will automatically dial the CapTel help line. Our helpful Customer Service team is happy to help you set up and use your CapTel 2400iBT phone – we're only a button push away.

Help is available 24 hours a day, 7 days a week (closed on major holidays). Help materials are also available online at www.CapTel.com



TO HOME Button

Press the HOME button at any time to return to the main menu screen. The HOME button is your easy way to go back to the beginning and start again.

If you are ever "lost" in the menu system, press the HOME button to return to the main screen.



(B) CAPTIONS Button

Customer Service: 888-269-7477

When the CAPTIONS button is on (blue light around button is lit) you will see word-for-word captions of everything the other party says. Press the CAPTIONS button at any time during a call to turn captions on or off.

Back view of CapTel 2400iBT



1 Telephone Jack

Plug the telephone cord (provided) in here (RJ11 jack). Telephone cord can be for most types of telephone service, including: analog, VOIP, fiber optic, digital cable service, or analog with DSL service. CapTel does NOT work with PBX systems unless an analog port is available.

2 Ethernet Jack

Plug the Ethernet cable (provided) here to connect to your Internet service. If you connect to the Internet using a WiFi network, you do not need to plug in the Ethernet cable.

3 LNK/ACT - Status Light

Lets you know if your Ethernet cable is connected to the Internet and whether activity is on the line. NOTE: the LNK/ACT indicator does not light up when connected via WiFi.

4 Power Jack

Plug the AC power adapter here. Use only the AC power adapter that came with your CapTel 2400iBT phone. Any other adapter may damage the phone.

Setting up Your CapTel 2400iBT

For complete instructions, please see the Setup Guide.

Requirements

- 1. High-speed Internet access. Your CapTel 2400iBT can connect to your Internet service either with an Ethernet cable or through a wireless (WiFi) network. Depending on your Internet setup, a router may also be required to allow you to connect more than one device to your Internet service.
- **2. Telephone Service.** Can be analog line, DSL with filter, VOIP, fiber optic, or Digital Cable phone service. CapTel does NOT work with digital Private Branch Exchange (PBX) systems found in some office environments, residential facilities, and hotels – unless an analog port is available.
- **3. Standard electrical power** (AC adapter plugs into standard wall outlet).

Step 1: Select a Location

Set up the CapTel 2400iBT in a place where:


- A high-speed Internet (Ethernet) jack or your wireless router is located nearby.
- A telephone jack is located nearby (can be analog line, DSL with filter, VOIP, fiber optic, or Digital Cable phone service).
- There is an electrical outlet nearby.
- The surface it will be set on is stable and secure.
- The area is protected from excessive heat or humidity.

Guide or contact CapTel Customer Service for help setting everything up.

The outlets for power, the phone line, and the Ethernet connection should all be located nearby, as each cord needs to plug into your CapTel 2400iBT. If your Internet access is located in a different room than where you plan to use the CapTel 2400iBT, a WiFi setup is recommended. Please see the Setup

Step 2: Plug in the Power Adapter

- **1.** Plug the power cord into the jack labeled "Power" on the back of the CapTel 2400iBT phone.
- 2. Plug the power adapter into a wall outlet or power strip. The display screen and Volume indicator lights will light up to let you know the power is connected.

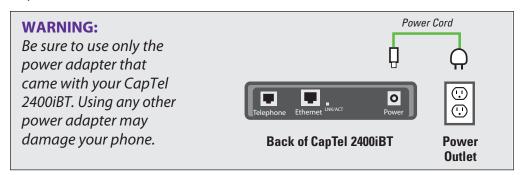

After a moment, the Initial Setup screen will appear. Follow the instructions on the display screen.

ПP

To choose an option, touch the icon on the screen and then touch () (next).


Step 3: Select Language




- 1. Touch **(English)** or **(Espanol)** for the language you want.
- **2.** Touch (>) (next) to continue.

12 GETTING STARTED

Step 4: Telephone Line

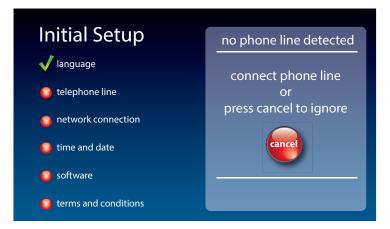

- 1. Plug your telephone line into the back of the CapTel phone.
- 2. Plug the other end of the line into your telephone wall jack.

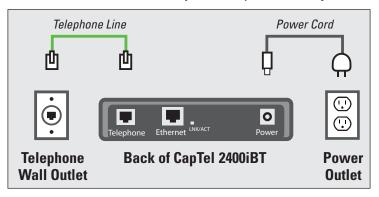

Telephone service can be analog, VOIP, fiber optic, digital cable phone service, or analog with DSL filter. CapTel does not work with digital PBX office phone systems unless an analog port is available.



3. Use dialing prefix?

Some office environments require you to dial a number in order to get an outside line. For example, you may need to dial a 9 or another number sequence to get an outside line. CapTel 2400iBT lets you add the dialing prefix here, and will dial it for you automatically each time you place a call from your Contact list, or when using the Speed Dial, Call History, or Answering Machine history. If you dial the phone number directly, you will need to enter the Dialing Prefix at the time you dial.

If you do not need a dialing prefix, touch (no).

If you want to enter a dialing prefix, touch (yes).

Use the on-screen dial pad to enter the dialing prefix. To include a two second pause, touch Pause (Pause).


When you are satisfied, touch (Save).

If you would like to exit without changing the dialing prefix, touch (Cancel).

OTE

If you are unsure whether your office requires a dialing prefix to get an outside line, you probably do NOT need to enter one.

Step 5: Network Connection

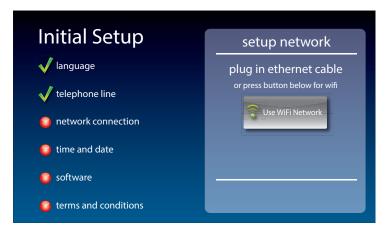


For Wired Connections:

- 1. Plug the Ethernet cable that came with your phone into the jack on the back labeled "Ethernet".
- 2. Plug the other end of the Ethernet cable into the jack you use to access your Internet service. Depending on how your Internet service is set up, you could be plugging into an available Ethernet jack on any of the following:
 - On a digital cable modem
 - On a DSL modem
 - On a router in your home
 - In an Ethernet wall outlet


Use the examples on the following page to match your setting with the correct installation method.

All outlets located together (Wired Setup)

Example 1: All of the connection outlets (telephone, Internet, and power) are available in the same room that you are planning to use the CapTel 2400iBT.

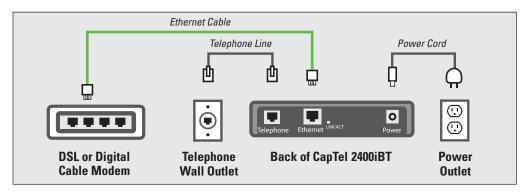

Example 2: If there is not an available Ethernet jack in your DSL or cable modem, you will need a router or switch that lets you connect more than one device to the Internet. Contact your Internet Service Provider to learn what router or switch will work with your modem. The router should support DHCP. Connect the router to your DSL/cable modem. Then connect the CapTel 2400iBT to the router or switch using the Ethernet cable.

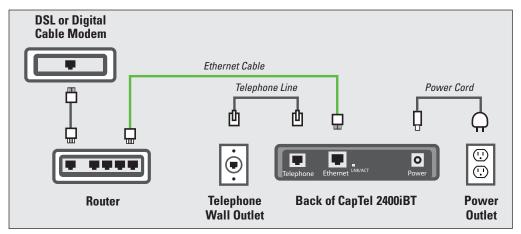

If your Internet outlet is in a different room from where you plan to use CapTel, see the Setup Guide for instructions.


For Wireless (WiFi) Connections:

If you have a wireless ("WiFi") network in your home or office, your CapTel phone can connect to the Internet using the WiFi network. You do not need to plug in an Ethernet cable.

To set up your CapTel for WiFi use, you will need to know the name of your wireless network as well as any passwords that are needed to access the network. The way you access your WiFi network with CapTel will be very similar to how you access the network from a computer or mobile device.

VOTE

If someone helped install the wireless network for you, it may be helpful to consult with that person as you set up your CapTel phone.

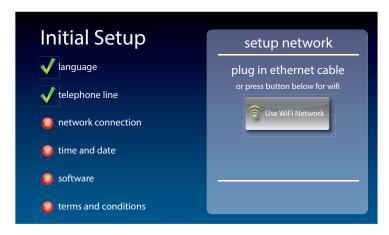

- **1.** Touch **(Use WiFi Network)** on the display screen. CapTel scans for wireless networks within range of your phone.
- **2.** The display lists any WiFi networks detected, in order of strongest signal strength to the weakest signal strength.

OTE

CapTel may detect wireless networks in your location that you do not recognize – such as a wireless network from a neighbor or nearby office. Be sure to select the name that matches your own network.



3. Touch the name of the network that you want to use. Then touch **(Connect)**.

If you do not see the network you want to use:

- Try moving your CapTel phone to an area closer to your wireless router. Then, touch available networks. (**Refresh**). CapTel will re-scan for available networks.
- Touch Enter Manually). CapTel will ask you to enter information about your wireless network, then it will try to connect.
- **4. Enter WiFi Password:** On the display, use the onscreen keyboard to enter the password for your WiFi network.



Remember to use the (shift) key for capital letters when needed. Passwords are case-sensitive. To enter special characters, touch Numbers (Numbers).

When you are done, review your password carefully. If it does not exactly match the password for your network, CapTel will not be able to connect. Use the key to correct any errors.

When you are satisfied the password is correct, touch save (save).

Step 6: Time and Date

Touch the Time Zone on the display screen that you live in. Touch (>) (next) to continue.




Your CapTel phone will fill in the date and time based on the time zone you indicated. If you need to adjust the time and date, touch the field on the display screen that you want to change. Use (arrows) on the display screen to scroll to the correct number. Then touch the next field that you want to change.

When the date and time are set, touch (>) (next) to continue.

Step 7: Software

Your new phone checks to make sure that you have the most current software available. If needed, CapTel will automatically download the most recent software version. Please be patient, the download may take a few moments. Watch the progress on the CapTel display screen to know the status.

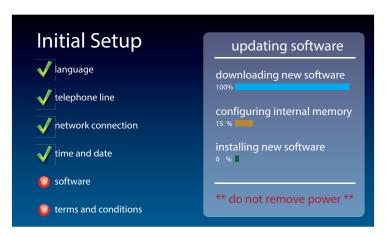

VOTE

Do not interrupt the power supply during a software download. It may damage your CapTel phone.

Once the software download is complete, your CapTel phone will re-boot as part of the update process. It will automatically run through the setup steps again, using the settings that you've already chosen.

Once the update process is completed, you will be able to continue with setup.

20 GETTING STARTED Customer Service: 888-269-7477


Step 8: Registration Terms and Conditions

The very first time you set up CapTel, you may be asked to review the Registration Terms and Conditions on the CapTel display. Follow the directions on the screen.

If your CapTel 2400iBT is already registered, your phone will not show the following registration screen.



Per requirements from the Federal Communications Commission (FCC), the Captions feature on your phone will not be activated until you accept the Registration Terms & Conditions. If you need assistance, press the blue **CUSTOMER SERVICE** button for help. Details about the FCC requirements can be found at www.CapTel.com.

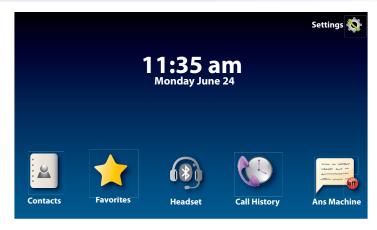


Congratulations you are ready to go!!

Using a Headset or Neckloop (Optional)

You can use earphones, a headset, a neckloop or another type of listening device with the CapTel 2400iBT to maximize the audio quality. CapTel works with Bluetooth devices, giving you cordless connectivity to Bluetooth headsets and neckloops. Or, you can simply plug a headset/ assistive device into the audio jack located on the right side of the CapTel phone. Headsets/devices sold separately.

Connecting Headset/Neckloop with Audio Jack (3.5 mm)

Simply plug the headset/assistive device into the 3.5 mm audio jack located on the right side of the CapTel 2400iBT (open the protective cover to reach the jack).

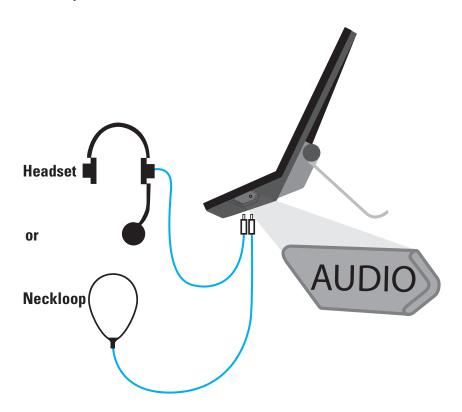

22 GETTING STARTED Customer Service: 888-269-7477

Connecting a Bluetooth® Headset/Neckloop (Optional)



The instructions may vary depending on what type of Bluetooth device you have. It's important to be familiar with how your Bluetooth device works before you get started.

Things to know about your Bluetooth device before you start:

- How to turn your Bluetooth device on and off
- How to set your Bluetooth device in "pairing" mode (make it able to be discovered)
- What "name" is assigned to your Bluetooth device

To connect a Bluetooth device:

- 1. Make sure your Bluetooth device is turned on and set in "pairing" mode.
- 2. On the CapTel display screen, touch 🐔 (Settings).
- **3.** Touch (Connections).
- **4.** Touch (Bluetooth) to go to Bluetooth settings.
- (on) to turn Bluetooth on. **5.** Touch

The display shows a Status list of any devices that are already paired with your CapTel phone.

6. Touch Scan (Scan).

Your CapTel phone will scan for any available Bluetooth devices within range. A list of available Bluetooth devices will appear in the Status screen on the CapTel display.

If you do not see the name of your device listed, make sure that your Bluetooth device is turned on and set to "pairing" mode. Then touch scan (Scan) again.

- **7.** On the CapTel display screen, touch the name of your Bluetooth device in the list so that the device name is highlighted.
- **8.** Touch (Pair). Your Bluetooth device is now connected and can be used when making/answering calls.
- **9.** Press (HOME button) to leave the Bluetooth menu.



While you may have multiple Bluetooth devices paired to your CapTel 2400iBT, only one Bluetooth device can be used at a time. The name of the "active" Bluetooth device is listed in the top left corner of the CapTel display screen.





Bluetooth icon turns blue when Bluetooth option is ON.

Making and Answering Calls



This section explains how to place outgoing calls and how to answer incoming calls.

TOPICS

- Dialing a Phone Number Directly
- Answering an Incoming Call
- Turning Captions On/Off during a Call
- Adjusting the Volume
- Adjusting the Tone Control
- Customizing the Tone Setting
- Using the Speakerphone
- Knowing What Sounds are on the Line

Dialing a Phone Number Directly




Like with any other phone, you can dial the phone number of the person you want to call.

- **1.** Before you dial, check that the blue light around the **CAPTIONS button**) is on. If not, press the CAPTIONS button to turn captions on.
- **2.** Lift the handset. Or, to make a call using the speakerphone, press (SPEAKER button).

If you are using a Bluetooth headset, touch (1) (Headset) to turn on your Bluetooth device.

- 3. Use the CapTel dialing keypad 🐉 to dial the phone number of the person you wish to call.
- **4.** As you dial, the display screen shows the status as your CapTel phone connects to the captioning service.



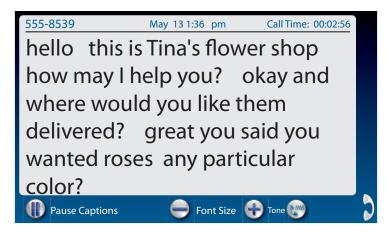

5. Listen or watch the display screen to know when someone answers. Begin your conversation as you normally would. Captions will appear on the display screen as the other party speaks.

You can adjust the volume during the call to a level you hear best by pressing the (up) or (down) arrows on the **VOL button**.

When you are finished with your call, hang up the handset.



If you are using a Bluetooth headset, you may need to adjust the volume directly on your Bluetooth device.



You can turn captions on/off at any point in the call. Just press the CAPTIONS button. When the blue light is on, you will get captions.


Talk back and forth as you normally would, reading the captions as you go. You don't have to wait for the captions to finish appearing on screen before you answer.

You can also place a call to someone listed in your Contacts, Favorites, Call History, or Speed Dial settings. You need to have entries saved in Contacts before you can use these features. See Contacts on page 65 for instructions.

Answering an Incoming Call



- 1. When the phone rings, the display lights up to let you know there is an incoming call. (If you have Caller ID service, the caller's name and phone number will appear.)
- 2. Lift up the handset to answer the call. Or, to answer on speakerphone, press (SPEAKER button).
- 3. Check that the blue light around the (CAPTIONS button) is on. If not, press the CAPTIONS button to turn captions on.

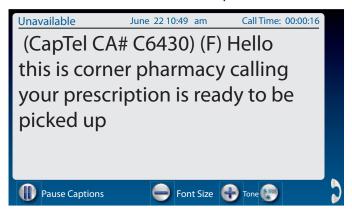

When you are finished with your conversation, hang up the phone as you normally would.

If you answer on an extension telephone and want to get captions, just pick up the CapTel handset and press any of the number keys. As long as the CAPTIONS button is on, your CapTel phone will connect to the Captioning Service and you will see captions.

Answering a Call on a Bluetooth® Headset/Neckloop



Your Bluetooth device must be paired and active with the CapTel 2400iBT before you can make/answer calls with it. Please see "Connecting a Bluetooth Headset/Neckloop" on page 23 for instructions.

To Answer a Call:

Customer Service: 888-269-7477


- 1. Check that your Bluetooth device is connected by confirming that the name of your device is listed in the upper left corner of CapTel display screen.
- 2. When the phone rings, the CapTel display lights up to let you know there is an incoming call.
- **3.** On the CapTel display, touch (answer by Bluetooth).

You will be able to hear the caller's voice over your Bluetooth device. Captions of the conversation will appear in the CapTel display screen.



You may be able to answer the call directly on your headset/neckloop. The answering process will vary depending on the type and style of your Bluetooth device. Check the instructions that came with your Bluetooth device for details.

Ending a Call using a Bluetooth® Headset/Neckloop

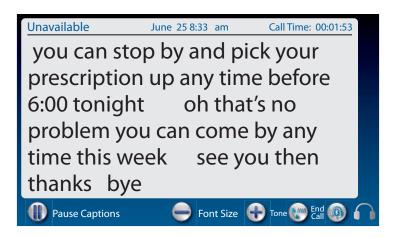

When you are ready to end your call:

1. On the CapTel display, touch (a) (END CALL). The call will "hang up". Or, you can pick up the handset briefly and hang it up again to end the call.



You may be able to end the call directly on your headset/neckloop. The end call process will vary depending on the type and style of your Bluetooth device. Check the instructions that came with your Bluetooth device for details.

Turning Captions On/Off During a Call

You can turn captions on/off at any time during your call by pressing (arms) (CAPTIONS button). When the blue light around the CAPTIONS button is lit, you will see captions of everything the other party says.

To turn captions off, press (CAPTIONS button) again. The blue light around the CAPTIONS button goes off.

Adjusting the Volume

You can increase the volume of the person's voice coming through the handset or speakerphone, up to a 40dB gain from min/max when the CAPTIONS button is on. (If captions are turned off, the maximum volume is 18dB gain). Watch the lights next to the VOL bar to see what level the volume is set to.

- To increase the volume, press on the **VOL button**.
- To decrease the volume, press on the **VOL button**.


The maximum volume setting on captioned calls is very loud. People who do not need amplification should not use the phone at its highest volume setting. The volume level automatically returns to a mid-range setting whenever the phone is hung up.

To save your volume setting so that it will be used during every captioned call, see Saving Conversation Volume on page 90.



If you are using a Bluetooth headset, you may be able to control the volume directly on your Bluetooth device. See instructions that came with your Bluetooth device for directions.

Adjusting the Tone Control

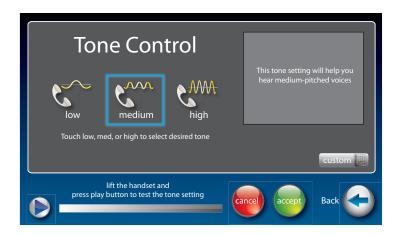

You can adjust the Tone of the handset sound to a frequency range that is better to hear. Use the Tone setting to emphasize HIGH, MEDIUM, or LOW frequency sound ranges. You can even customize the sound frequency to match your own audiogram. The setting will remain in effect until you change it again.

You can adjust the Tone setting before, during, or after a call.


To adjust the Tone setting:

- 1. On the display screen, touch **(Settings)**.
- **2.** Touch (Sounds).
- 3. Touch (Tone).
- 4. The display shows what the current setting is: LOW, MED, HIGH, or CUSTOM.
- **5.** Touch the Tone setting you wish to use.

To test how the Tone setting will sound, lift the handset and press on the display screen. You will hear a recording of first a man's and then a woman's voice, letting you check if the setting makes it easier to hear. Experiment with the different Tone settings to find the range that works best.

During a call: touch (**(Tone)** repeatedly to toggle between settings to the level that you hear best.

Customizing the Tone Setting

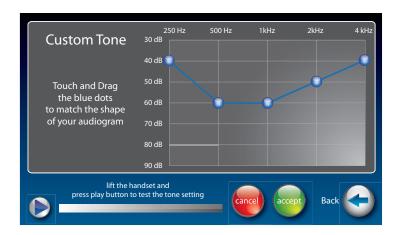

CapTel 2400iBT lets you personalize the Tone setting to enhance the frequency ranges that you hear best. For example, you can adjust the tone settings to match your own personal audiogram. CapTel lets you listen to test each setting as you make changes, to find the optimal customized hearing level.

To customize the Tone setting:

- **1.** On the display screen, touch **(Settings)**.
- **2.** Touch (Sounds).
- 3. Touch (Tone).
- **4.** On the display screen, touch custom (Custom).
- 5. Touch and drag the blue circles on the frequency chart to adjust the various sound levels.

As you make changes, test the sound quality by touching (play) while listening over the handset.

If you have a personal audiogram, adjust the blue circles to match the shape of your audiogram to give you optimal sound levels for your personal hearing level.

6. When you are finished, hang up the handset and touch (Accept) to save your changes or (Cancel) to exit.

Using the Speakerphone

Turning the Speakerphone On

At any point before or during a call, press the (SPEAKER button) on the CapTel dialing pad to turn the Speakerphone on (light on Speaker button goes ON). The CapTel handset will turn off, and any sounds on the line will be amplified out loud over the CapTel speaker.

<u>a</u>

When the Speakerphone is on, you can talk as you normally would directly facing the front your CapTel 2400iBT phone. You do not need to speak into the CapTel handset. The CapTel 2400iBT microphone will pick up your speech for your caller to hear.

When the Speakerphone is on (light on Speaker button is ON) you can set down or hang up the CapTel handset without ending the call. The conversation will be carried on over the Speakerphone and captions of your call will appear on the display screen.

Turning the Speakerphone Off

You can alternate between using the Speakerphone or using the CapTel handset during your calls, by turning the Speakerphone on and off. The way you turn the Speakerphone off depends on whether the CapTel handset is hung up or not.

If the CapTel handset is not hung up:

Press the (SPEAKER button) on the CapTel dialing pad to turn the Speakerphone off. The light on the Speakerphone will go off and the sound will be directed to the CapTel handset for you to continue the call.

If the CapTel handset is hung up and you'd like to continue the call:

Lift the CapTel handset. The Speakerphone will go off (light on Speaker button goes OFF) and the sound will be directed to the CapTel handset for you to continue the call.

Customer Service: 888-269-7477

If the CapTel handset is hung up and you'd like to end the call:

Press the (SPEAKER button) on the CapTel dialing pad. The Speakerphone will go off and the call will end.

Ending a Call when the Speakerphone is On

The way to end a call when the Speakerphone is on depends on whether the CapTel handset is hung up or not.

If the CapTel handset is hung up:

Press the (SPEAKER button) on the CapTel dialing pad. The Speakerphone will turn off (light on the Speaker button goes OFF) and the call will end.

If the CapTel handset is not hung up:

Hang up the handset, as you would end with any other call. If needed, press the (SPEAKER button) to turn the Speakerphone off (light on the Speaker button goes OFF). The Speakerphone will turn off and the call will end.

Adjusting the Sound on the Speakerphone

Use the volume key to adjust the volume of the sound coming out of the Speakerphone.

Knowing What Sounds are on the Line



The handset graphic on the display screen tells you what incoming and outgoing sounds are occurring during your call with two visual indicators: the Signal meter (yellow waves) and the Outgoing Volume meter (green waves).



Customer Service: 888-269-7477



Signal Meter – Sounds on the Line

Shows when there is sound coming over the line, even if the sound is not loud enough for you to hear. The more sound waves coming out of the handset, the louder the sound.



Outgoing Volume Meter

Helps you to see how loudly you are speaking. The more lines going into the handset/MIC graphic, the louder your voice is. If the lines become red, you may want to lower the volume of your own voice during a call.

The display shows whether sound is coming over the CapTel headset 1, speakerphone 🗽 or over a Bluetooth® headset 🕻

These meters only appear during calls that have captions.

Viewing Captions



This section explains how to adjust and review conversation captions.

TOPICS

- Turning Captions On/Off During a Call
- **Viewing Corrections**
- Reviewing Captions During a Call
- Reviewing Captions After Hanging Up
- Saving Conversation Captions
- Erasing Captions
- Changing the Color of the Captions
- Changing the Captions Font Size and Style
- Setting how Captions Scroll Across the **Display Screen**
- Adjusting the Brightness of the Display Screen

Turning Captions On/Off During a Call



You can turn captions on/off at any time during your call by pressing (APROSE) (CAPTIONS button). When the blue light around the CAPTIONS button is lit, you will see captions of everything the other party says.

To turn captions off, press (CAPTIONS button). The blue light around the CAPTIONS button goes off.

You can increase/decrease the size of the captions font by touching 🛖 or 🥌 at the bottom of the screen.






You can press the CAPTIONS button at any time during the call. If captions are not on at the start of your call, don't worry. Captions will appear approximately 3–5 seconds after the CAPTIONS button is turned on.

38 VIEWING CAPTIONS

Viewing Corrections

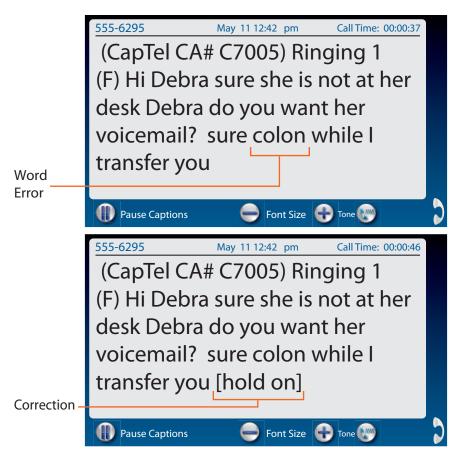

On occasion, word errors may appear in the captions due to the speech recognition system the Captioning Service uses. The Captioning Service will try to correct word errors as they occur. The corrected word appears in [brackets] to let you know a correction has been made.

There may be a slight delay before the word error is corrected. If you are unclear about what was said, ask your caller to clarify.

Reviewing Captions During a Call



There are a couple of ways to review captions during a call:

- 1. Touch (Pause Captions).
- **2.** Touch and move the slide bar at the right up to scroll back through previous captions.
 - OR -


Place your finger on the captions screen and swipe downward to review previous captions.

When you are finished reviewing captions, touch (Back to Live Captions) to return to the current conversation. Anything the other party said during the time you were reviewing captions will appear, and the live captions will continue as usual.

OTF

Your CapTel phone holds captions for up to 100 different conversations in memory (approximately 10 hours of conversations). If the conversation memory fills up entirely, older captions will be erased as new captions come in.

Reviewing Captions After Hanging Up

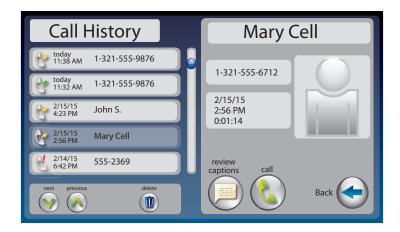

To review captions after your conversation is ended and the phone is hung up:

- **1.** On the display screen, touch (Call History).
- 2. From the Call History listing on the left, touch the call that you wish to review. If needed, use the scroll bar or touch (w) and (A) to scroll through the list of calls.
- 3. Once the call you want is highlighted, information about the call appears on the right. Touch ((review captions) to review captions of the call. When reviewing captions: Touch and drag the scroll bar on the right to move backwards and forwards through captions. Touch (w) and (\lambda) to see captions from a previous/later call.
- **4.** When you are done reviewing captions, press (HOME button) to exit.

The Call History listing lets you know whether a call was ***incoming** or **e outgoing**.

Your CapTel phone comes with the Save Captions feature enabled, letting you view conversation captions after you've hung up the phone. You can turn this feature off so that CapTel will not save conversation captions after every call. Please see Saving Captions after Hang up on page 89 for instructions.

Saving Conversation Captions



You can set your CapTel phone to save captions after the call is ended. This allows you to go back and review what was said after you've hung up. You can review captions from up to 100 different conversations in memory. The default setting is Save Captions – ON.

To change the Save Captions setting:

- 1. On the display screen, touch 🗞 (Settings).
- 2. Touch (General Settings).
- 3. Touch (Save Captions On).
- **4.** Touch **(On)** or **(Off)** to select.

Save Captions On saves conversation captions after the call is finished and the phone is hung up.

Save Captions Off erases the conversation captions after every call.

Customer Service: 888-269-7477

5. Once you have made your selection, press (HOME button) to exit.

Erasing Captions



To erase captions of a single conversation after the call has ended:

- **1.** On the display screen, touch (Call History).
- **2.** From the Call History listing, touch the call that you wish to erase. If needed, touch (and (and or touch and drag the scrollbar to scroll through the list of calls.
- 3. Once you have the call you want highlighted, information about the call appears on the right side of the display. Touch (III) (Delete) to erase captions of the call.
- **4.** The display asks you to confirm that you want to delete the call. Touch (yes) to erase or (no) to exit.
- **5.** When you are done erasing captions, press (HOME button) to exit.

Erasing captions of your call will also delete the call from your Call History.

Changing the Color of the Captions



You can change the color of the captions font and background color on the display screen to a color combination that you prefer. You need to set the conversation colors before or after a call. You cannot change the captions colors during a live captioned call.

OTE

Changing the font color applies only to conversation captions. The font size and style for your CapTel 2400iBT menu options always remains the same.

Customer Service: 888-269-7477

To change the font and background colors:

- 1. On the display screen, touch 🖏 (Settings).
- 2. Touch (Display).
- 3. Touch 🍣 (Display Colors).

44 VIEWING CAPTIONS

4. The display screen shows you an example of how the colors are currently set.

To change the Captions color:

- 1. Make sure **Captions** (Captions) is highlighted (blue).
- 2. In the Color Options table, touch the color that you would like the captions to appear. As you select your color, CapTel shows you an example of how the captions will look.

To change the Background color:

- **1.** Touch Background (Background) to highlight (blue).
- 2. In the Color Options table, touch the color you that you would like the background to appear. As you select your color, CapTel shows you an example of how the background will look.
- **5.** When you have selected the color combination that you prefer, touch (Accept) to save or 🛶 (Cancel) to exit.
- **6.** When you are done, press (HOME button) to exit.

If vou have difficulty reading the captions, you may try different color combinations to create higher contrast between the captions color and the background color.

Changing the Captions Font Size and Style



You can adjust the font size and style of the conversation captions to make them easier to read.

VOTE

Changing the font size or style only applies to conversation captions. The font size and style for your CapTel 2400iBT menu options always remains the same.

To change the font size of the captions:

- 1. With the handset hung up, touch 🗞 (Settings).
- 2. Touch (Display).
- 3. Touch [F (Font Settings).
- **4.** The display shows you what the current font size and style setting is. To change the font style, touch one of the five (5) style options available. To change the font size, touch or .

- **5.** When you have selected the font size and style that you prefer, touch (Accept) to save your change or (Cancel) to exit.
- **6.** When you are done, press (HOME button) to exit.

Setting how Captions Scroll Across the **Display Screen**



You can control how the captions appear on the display screen: either appearing word-by-word across the screen as each word is captioned or by smooth scrolling one line at a time in a continuous movement. The default setting is smooth scrolling.

To change the way captions scroll across the screen:

- 1. On the display screen, touch 🐔 (Settings).
- 2. Touch (General Settings).
- 3. Touch (Smooth Scrolling).
- **4.** Touch the scrolling option that you prefer.

Line-by-Line Scrolling: captions appear word-by-word across the screen as they are captioned

Smooth Scrolling: captions appear one line at a time in a continuous motion

5. Once you have made your selection, press (HOME button) to exit.

Adjusting the Brightness of the Display Screen



You can adjust the brightness of your CapTel 2400iBT display screen to make it easier to read in well-lit or dimly-lit environments.

NOTE

You need to adjust the brightness of the Display Screen before or after a call. You cannot set the display screen brightness during a live captioned call.

To adjust the brightness of the Display Screen:

- 1. On the display screen, touch 💨 (Settings).
- 2. Touch (Display).
- 3. Touch : (Brightness)
- **4.** Touch and drag the slide bar to adjust the brightness to the setting that you would like. Sliding left makes the display screen dimmer. Slide right to make the display screen brighter.
- **5.** When the brightness is set, touch (Accept) to save or (Cancel) to exit.
- **6.** Press (HOME button) to exit.

JOTE

After 60 seconds of non-use, the CapTel display screen will start becoming more dim and gradually go completely dark as an energy-saving measure. To re-illuminate the screen, simply touch the display screen or any button on the dialing keypad.

Getting Help

If you have any questions as you use your new CapTel 2400iBT, we're here to help. Use any of these ways to contact us:

Pick up the handset and press (CUSTOMER SERVICE button) to automatically speed dial directly to our helpful Customer Service team. If you are in an office setting, you may need to dial a number such as "9" first to get an outside line first and then press (CUSTOMER SERVICE **button)**. (Available 24 hours a day, 7 days a week. Customer Service is closed on major holidays.)

Help materials are also available online at our website.

Phone: 1-888-269-7477 (Available 24 hours a day, 7 days a week)

FAX: (608) 204-6167

Mail: CapTel Customer Service

450 Science Drive Madison, WI 53711

Email: CapTel@CapTel.com

Customer Service: 888-269-7477

Online: www.CapTel.com (Live Chat help available at our website)

Spanish Language Customer Service: 1-866-670-9134

section 4

Call History



This section explains how to use Call History to see who has called you, to review captions of past conversations, and to see information about any incoming calls you may have missed.


TOPICS

- **■** Reviewing Previous Calls
- Reviewing Captions from a Previous Call
- Dialing from Call History
- **Erasing Previous Calls**

50 CALL HISTORY

Reviewing Previous Calls


Your CapTel 2400iBT keeps a list of information about recent calls, including calls that you may have missed. You can review captions from the call, the date and time of the call, and any Caller ID information if available. CapTel can even show you a picture of the caller if it is someone in your Contact list.

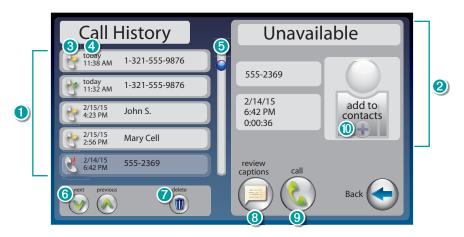

- 1 Previous calls: Touch the call you want information about.
- 2 Information about the call shows up here.
- 3 Indicates whether call was incoming, outgoing, or unanswered.
- 4 Date and time of the call.
- **5** Use the scroll bar to review entries further down the list.
- 6 Next/Previous: Lets you move up/down through entries in the list.
- **7** Delete: Erases the selected call from Call History.
- 8 Review captions: Review captions of the call.
- Ocall: Places a call to this person.
- O Add to Contacts: Adds caller to your Contacts list.

Remember to first touch the call listed in Call History on the left side of display that you want to review. Once you touch the call listing, more information about the call will appear on the right of your screen, including a picture of the caller if you have added a picture for him/her in your Contacts list.

Reviewing Captions from a Previous Call

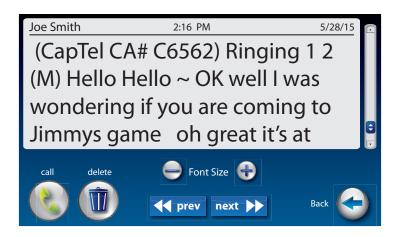

You can look back and see captions from any call in your Call History.

To review captions from a previous call:


- 1. From the display screen, touch (Call History).
- 2. From the list on the left, touch the call that you want to review.
- Touch and drag the scroll bar to see calls further up or down the list.

 You can use (()) (Next/Prev) to move up or down the list.
- **3.** Information about the call that you selected appears in the right of the screen.
- **4.** Touch (() (Review Captions) to review captions of the call.
- **5.** Touch and drag the scroll bar at the right to move backwards and forwards through the captions.
- **6.** When you are finished, press (HOME button) to exit.
- You can increase/decrease the size of the captions font by touching igoplus or igoplus at the bottom of the display.

Customer Service: 888-269-7477

Touch Touch rext (<< prev / next>>) to view captions from a previous/later call.

Dialing from Call History

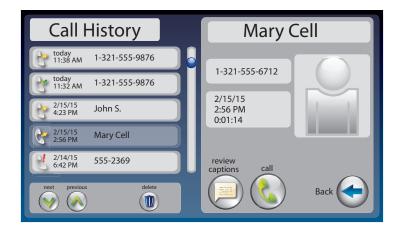

You can use Call History to quickly re-dial someone that has called you before. This is especially helpful if you notice you've missed a call – you can call the person back directly from Call History.

To dial a number from Call History:

- 1. From the display screen, touch (Call History).
- **2.** From the list on the left, touch the call that you want to review.
- Touch and drag the scroll bar to see calls further up or down the list. You can use (Next/Previous) to move up or down the list.
- 3. Information about the call that you selected appears in the right of the screen
- **4.** Touch (Call) to place a call to this person. If you are calling this number for the first time, the display asks you to confirm which number to call. For example, if you need to dial a 1 or the
- **5.** Lift the handset. CapTel dials the number automatically.

area code first, or if a dialing prefix (such as "9") is required.

Erasing Previous Calls



To erase a single call from Call History:

- 1. From the display screen, touch (Call History).
- 2. From the list on the left, touch the call that you want to review.
- Touch and drag the scroll bar to see calls further up or down the list. You can use (()) (Next/Prev) to move up or down the list.
- 3. Information about the call you selected appears in the right of the display.
- **4.** Touch (III) (**Delete**) to erase this call from your Call History.
- **5.** The display asks you to confirm if you want to delete the call. Touch **(yes)** to erase or touch **(no)** to exit.
- **6.** When you are finished, press (HOME button) to exit.

54 CALL HISTORY Customer Service: 888-269-7477

Answering Machine



This section explains how to use the built-in Answering Machine and see captions of messages that people leave for you. You can hear the voice recording and read captions of the message. CapTel can also show you captions of messages that are left on an external answering machine.

TOPICS

- About the Answering Machine
- Turning Answering Machine On/Off
- Playing Your Answering Machine Messages
- Erasing Answering Machine Messages
- Deleting All Messages
- Recording a Personal Greeting
- Seeing Captions of Voice Mail Messages
- Captioning Messages from an External **Answering Machine**
- Setting the Ring Count
- Playing Incoming Messages Aloud/Silent

About the Answering Machine


Your CapTel 2400iBT can take messages for you when you are unavailable, for you to play back later. When checking your messages, you can listen to the messages as well as see captions. The CapTel Answering Machine saves up to 100 messages. Each message can be up to 90 seconds long. If you have more than one answering machine in your home or office, make sure only one answering machine device is turned on/active at a time. If you rely on voice mail service for messages, the answering machine should be turned off.

Turning Answering Machine On/Off




- 1. On the display screen, touch (Ans Machine). The first time you set up your Answering Machine, the display tells you there are no messages available.
- **2.** Touch **(setup internal answering machine)**.
- **3.** The display screen shows you the current setting (on or off). Touch (Answering Machine Off) and select on or off.
- **4.** When you are finished, press (HOME button) to exit.
 - A green ON indicator appears on the Answering Machine icon to let you know at a glance whenever the Answering Machine is turned on.

Playing Your Answering Machine Messages

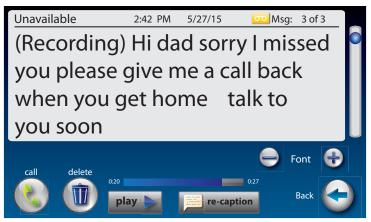


1. The display screen lets you know when you have unread messages waiting.

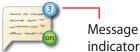

- 2. Touch (Ans Machine).
- 3. Select and touch the call you want to review from the Answering Machine list on the left of the display.
- **4.** Information about the call you selected appears on the right of the display.
- **5.** Touch (PLAY), then lift the CapTel handset to listen to your message. Or touch (speaker) to play messages aloud.
- **6.** When you are done listening/reading messages, you can:
 - Touch (Call) to place a call to this person.
 - Touch (III) (Delete) to erase the Answering Machine message and captions.
 - Touch re-caption (re-caption) to get new captions of this message.
 - Touch Play (PLAY) to play the message/captions again.

If you play your message and there are no captions, press the CAPTIONS button (blue light on). The voice message will play again, this time showing captions on the display.

Erasing Answering Machine Messages

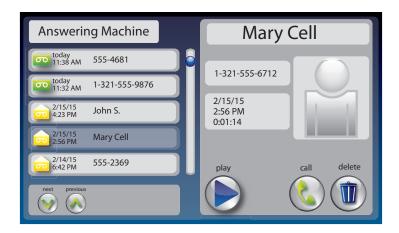

To erase an individual Answering Machine message:

- 1. The display screen lets you know when you have unread messages waiting.
- 2. Touch (Ans Machine).
- **3.** Select and touch the call you want to review from the Answering Machine list on the left side of the display.
- **4.** Information about the call you selected appears on the right side of the display.
- **5.** Touch (PLAY) to listen to your message and see the captions.
- **6.** To delete the message, touch (Delete). The display asks you to confirm you want to delete the message. Touch (yes) to delete or touch (no) to cancel.

When you are finished erasing your messages, press (HOME button) to exit.

Deleting All Messages



In addition to erasing Answering Machine messages one at a time as you play them, you can also erase all your saved Answering Machine messages at one time.

To delete all your Answering Machine messages:

- 1. From the display screen, touch 🚫 (Settings).
- 2. Touch (Answering Machine).
- 3. Touch (Delete All Messages).
- **4.** The display asks you to confirm you want to **delete ALL messages**. Touch (yes) to delete or touch (no) to cancel.
- **5.** Press (HOME button) to exit.

Recording a Personal Greeting



You can record a personal greeting for the CapTel 2400iBT to use whenever the Answering Machine takes messages. If you do not want to record a personal greeting, the Answering Machine will use a pre-recorded greeting that says: "Hello. No one is available to take your call. Please leave a message after the tone."

To record a personal greeting:

- 1. From the display screen, touch 🚫 (Settings).
- 2. Touch (Answering Machine).
- 3. Touch (Greeting Message).

To hear the greeting that is currently being used, lift the handset and touch (play greeting message). You will be able to hear the greeting in the CapTel handset.

Customer Service: 888-269-7477

4. Touch ((record new greeting) to record your personal message.



- **5.** Lift the CapTel handset and touch (press to start recording).
- **6.** Speaking into the CapTel handset, record the personal greeting you would like the Answering Machine to use. When you are finished recording, touch (press to stop recording).
- 7. If you are not satisfied with your personal greeting, you can re-record it. Just repeat steps 4–6 above.
- **8.** When you are satisfied with your personal greeting, touch (Accept). Hang up the CapTel handset.
- **9.** Press (HOME button) to exit.

Seeing Captions of Voice Mail Messages

If you use voice mail to get phone messages instead of the built-in answering machine, your CapTel 2400iBT can show captions of your voice mail messages. Simply use the CapTel dial pad to call into your voice mail system and "log in" the same way you would with any other telephone. Make sure the CAPTIONS light is on. As your voice mail messages are played, you will see captions on the display screen. Use the CapTel dial pad to select options in your voice mail menu system.

Captioning Messages from an External Answering Machine



In addition to the built-in answering machine that comes with your CapTel phone, you can also see captions of voice messages that are left on an external answering machine (a separate answering machine you may own, not the one built into the CapTel phone).

To get captions of your external answering machine messages:

- 1. From the display screen, touch 🗞 (Settings).
- 2. Touch (Answering Machine).
- 3. Touch (Ext Ans Machine).
- **4.** Lift the CapTel handset and place it next to your external answering machine speaker. Make sure the mouth-piece of the handset is next to the speaker of your answering machine. While you do this, your CapTel phone connects to the captioning service.
- **5.** Press the PLAY button on your external answering machine, so that the messages are played aloud.


The display will show captions of the messages as they are played.

OTE

While checking messages, you may operate your external answering machine the same way you normally do, using features such as play, re-play, delete, or save messages.

When you are done listening/reading captions, hang up the CapTel handset.

Setting the Ring Count

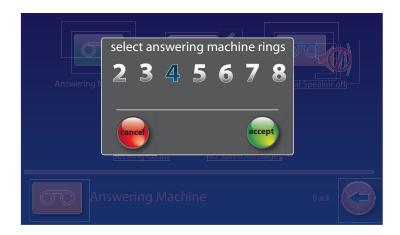


You can set the approximate number of times the CapTel will ring before the Answering Machine answers the call. The default setting is four (4) rings, but you can set it to be any number of rings between two (2) and eight (8).

The ring count may be slightly different in telephone systems that use "distinctive" ring patterns. Please experiment with this setting as needed for your specific telephone system.

To set the number of rings before answering:

- **1.** From the display screen, touch **(Settings)**.
- **2.** Touch (Answering Machine).
- 3. Touch (Set Ring Count).
- **4.** Touch the number of rings you'd like to use, then touch (Accept).
- **5.** Press (HOME button) to exit.

Playing Incoming Messages Aloud/Silent

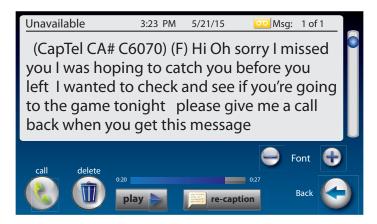

Just like with other Answering Machines, you can "screen" callers by having incoming messages played aloud on the CapTel phone for before you answer the call. While this is a convenient feature, there may be times when you do not want incoming messages to be played aloud, but instead recorded silently for you to review at a later time. The default setting is for incoming messages to be muted (silent).

To make incoming calls audible or set them to be silent:

- 1. From the display screen, touch 🐔 (Settings).
- **2.** Touch (Answering Machine).
- 3. Touch (External Speaker Off).
- **4.** The display asks if you want to (turn on) or (turn off) the answering machine speaker. Touch the option you want.
- **5.** Press (HOME button) to exit.

You can "screen" incoming Answering Machine messages by listening to the caller and reading the captions while the message is recorded. Make sure that the Answering Machine Speaker is set to "On". If you choose to answer the call, just pick up the handset and begin talking.

You can also send an incoming call directly to the answering machine to take a message if you are busy. Just touch ((Send to Ans Machine).

SECTION

Contacts



This section explains how to save frequently called numbers in your contact list for easy and quick dialing.

TOPICS

- Adding a New Contact
- Dialing a Phone Number in Your Contact List
- Editing an Existing Contact
- Removing a Contact
- Adding a Photo to a Contact
- Adding a Contact to Speed Dial

Adding a New Contact



You can save up to 100 names in your contact list for quick and easy dialing.

To add a new contact:

- **1.** From the display screen, touch (Contacts).
- 2. Touch (+) (add new contact).
- **3.** Use the on-screen keyboard to type in the First Name of the person you want to add.
- **4.** Touch the field next to "Last" to move down, then use the on-screen keyboard to enter the last name of the person.
- **5.** You can be specific about which number(s) you are adding: (Mobile), (Home), or (Work). Touch the field(s) next to the number(s) you'd like to add to begin entering.
- **6.** When you are finished entering the Contacts information, touch **(Save)**. The new contact appears on the left side of your Contacts list.
- 7. When you are finished, press (HOME button) to exit.
- You can add a photograph of your contact person to each entry.

 Pictures can be entered at the MyContacts website, then synced to your phone. See Adding a Photo to a Contact on page 70 for instructions.
- You can add contacts to your Speed Dial or Favorites list. See pages 82 and 85 for instructions.

66 CONTACTS Customer Service: 888-269-7477

Dialing a Phone Number in Your Contact List



To easily dial someone in your contact list:

- **1.** From the display screen, touch (Contacts).
- 2. Under the Contacts list on the left side of the display, find and touch the name of the contact you wish to call. To see contacts further down the list, touch and drag the scroll bar.
- Touch (Next/Previous) to move up/down through entries in your Contacts list.
- 3. On the right side of the display, touch the phone number of the person you want to call: [1] (Mobile), [2] (Home), or [1] (Work). CapTel will dial the number automatically for you.
- **4.** Lift the handset to dial.
- You can identify any contact person as a "favorite" for easy dialing. At the Add/Edit Contacts page, touch ((favorite)). See page 85 for instructions.

Editing an Existing Contact



To make changes to a contact in your list:

- **1.** From the display screen, touch (Contacts).
- 2. Under the Contacts list on the left side of display, find and touch the name of the contact you wish to edit. To see contacts further down the list, touch and drag the scroll bar. Information about the contact you selected appears on the right side of the display.
- Touch (Next/Prev) to move up/down through entries in your Contacts list.
- **3.** On the right side of the display, touch **(i)** (**Edit)** to go to the Add/Edit Contact screen.
- **4.** Touch the field that you want to change, then use the on-screen keyboard to type in the new information. Touch the next field you want to edit, or move between fields by touching (return) or Tab (tab). Touch to erase the entire contents of a field.
- **5.** When you are finished editing the contact, touch save (**Save**). The new contact information will be saved in your contact list.
- **6.** When you are finished, press (HOME button) to exit.
- You can add a photograph of your contact person to each entry.

 Pictures can be entered at the MyContacts website, then synced to your phone. See Adding a Photo to a Contact on page 70 for instructions.

68 CONTACTS Customer Service: 888-269-7477

Removing a Contact



To remove a single contact from your list:

- **1.** From the display screen, touch (Contacts).
- 2. Under the Contacts list on the left side of the display, find and touch the name of the contact you wish to remove. Touch and drag the scroll bar to see contacts further down the list.
- Touch (Next/Prev) to move up/down through entries.
- **3.** On the left side of the display, touch (III) (**Delete**) to remove the person from your contacts.
- **4.** CapTel asks you to confirm you want to delete the contact. Touch (yes) to remove the contact or (no) to exit.
- **5.** When you are finished, press (HOME button) to exit.

Adding a Photo to a Contact



Using your computer, you can add photos to your Contact List to make it easy to dial by picture on your CapTel 2400iBT. You can also add, change, or delete Contacts on your computer, then download the changes directly your CapTel 2400iBT phone.

Managing your Contact List is easy to do, but it takes two steps. First, go online and make changes at the MyContacts website. Then, "sync" with your CapTel 2400iBT to download the changes directly to your phone. Please see pages 70–81 for full instructions.

₽

If someone helped set up your CapTel 2400iBT phone, you may want to consult with them as you manage Contact Lists on your computer.

NOTE

If you do not have computer access, you can still add, delete, and change entries in your Contact List by using the touchscreen menu on the CapTel 2400iBT. Access to the MyContacts website is required, however, to add photos to your Contact List.

Managing Your Contacts List


The MyContacts website helps manage the Contacts List on your CapTel phone. You can add new contacts, phone numbers, and even photos, then "sync" them with your CapTel 2400iBT.

Because both the CapTel phone and the MyContacts website need to be set in sync mode, the sync process is easiest when the computer you are using and the CapTel phone are located close together. If the phone is in a different location than the computer, it may be helpful to have two people – one by the phone, one at the computer – to help the sync process go smoothly.


To manage your Contacts List:

Customer Service: 888-269-7477


1. On a computer that has Internet access, go to the website: https://MyContacts.CapTel.com

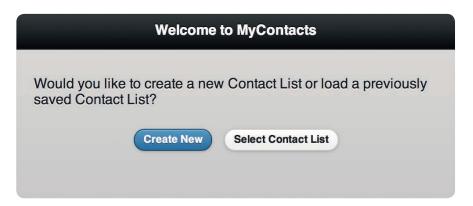

2. The website asks "Would you like to create a new Contact List or load a previously saved Contact List?" Click on Create New (Create New).

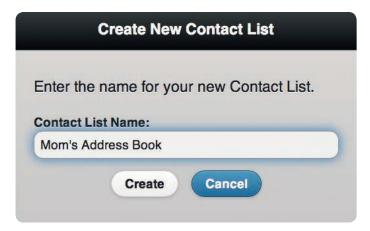

3. Enter the name you would like to use for your new Contact List, then click on **Create**).

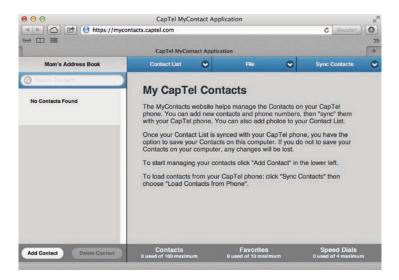

4. Click on Add Contact (Add Contact).



- **5.** Enter the first and last name of your contact, then click on Save (Save).
- 6. Add the phone number information for your contact. Use the tab key to move between fields. Click on (Save Contact) when Save Contact information is entered.




To add a photo to this contact: click on (Add Photo). Add Photo



The website lets you browse your computer to select and save an existing photo. The photo files can be in jpg, png, or gif format.

Click on Select Photo), find and select the photo that you want to add.

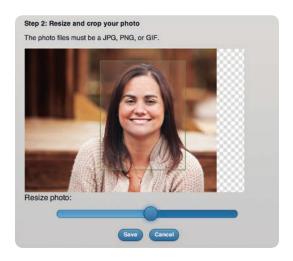

Once your photo is selected, you may re-size or crop the image to fit nicely in the CapTel display window. Only the part of the image inside the green shaded box will appear on your CapTel display.

ΔL

To resize your image, click and drag on the slidebar to make the image bigger or smaller. Click and drag the shaded box area to position it around your picture.

When you have a photo selected and sized, click on (Save).




When you are finished entering information for this contact, click on Save Contact (Save Contact). The new entry appears in your Contact List on the left side of the MyContacts website (in alphabetical order by first name).

Making your contact a Speed Dial number lets you quickly dial-by-picture on your CapTel phone. Click on (speed dial) to add this contact to your Speed Dial list. You may have up to 4 Speed Dial entries. You can also add this number to your Favorites list for easy dialing. To add this contact to Favorites, click on (favorite).

7. Repeat steps 4–6 to add as many contacts as you'd like – up to 100 entries total.

CONTACTS **75**

8. When you are satisfied with your Contact List, you must Sync Contacts in order to save the list to your CapTel phone. Click on Sync Contacts (Sync Contacts), then select Save Contacts to Phone).

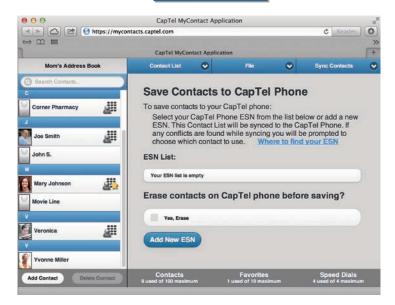

9. The website asks you to select the Electronic Serial Number (ESN) for your CapTel phone. The ESN can be found *on your CapTel phone* by touching (Contacts), then touching (sync).

On your CapTel display screen:



At the MyContacts website:



Click on Add New ESN), then enter the ESN from the CapTel display screen into the field labeled "ESN". Enter a name for the ESN in the field labeled "ESN Name". Click on Add (Add).

On your CapTel display screen:



10. At the window on your CapTel phone that says, "sync your contacts?" touch (Accept).

At the MyContacts website:




TE C


Entries from your online Contact List are added to any contacts you may already have saved on your phone. If you would like to erase the existing contacts on your phone and replace them with the new Contact List from your computer, click on "Yes, Erase" on the MyContacts website before you start the sync process.

The website lets you know when the phone is ready to sync. Make sure the ESN you want is selected, then click on (Start Sync) to begin the process.



Your Contact List is saved from the computer onto your CapTel phone. Watch on your CapTel phone display as the sync completes.

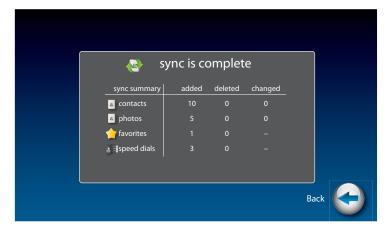

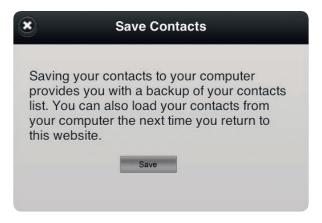

11. Once your Contact List is synced with your CapTel phone, you have the option to save your Contacts on your computer.

Saving a Contact List to your computer allows you to keep a back up and edit your list at a later time. You do not have to save your list on your computer – once the sync is finished, the Contact List is saved on your CapTel phone.

If you choose to save your Contact List, make note of where the xml file is saved on your computer for future reference.

Once the sync process is complete, your Contact List is saved on your CapTel phone. It will not change until you make changes on the phone or sync again with a different Contact List.

12. Exit the MyContacts website by clicking on contact List (Contact List), then selecting exit MyContacts (Exit MyContacts).

Other options available at MyContacts.CapTel.com

Under "Contact List" Contact List

> Manage Favorites – Lets you add or delete contacts in your Favorites list. You can drag & drop any name from your Contact List on the left into your Favorites list.

> Manage Speed Dials – Lets you add or delete contacts in your Speed Dial.

Manage ESN(s) – Lets you add or delete the Electronic Serial Number (ESN) of each CapTel phone.

Create a New Contact List – Lets you create a new Contact List. You can have multiple Contact Lists (i.e., one for business contacts, one for personal numbers, etc.).

Rename this Contact List – Lets you rename the current Contact List.

Exit MyContacts – To leave the MyContacts website. If any changes have been made, you can save the current Contact List to your computer. If you do not save, all information will be lost for security purposes. If the Contact List was already synced to your CapTel 2400iBT, however, the information will already be saved on your phone.

Under "File"

Load Contacts from Computer – Opens any Contact Lists that you may have already saved on your computer, for you to edit or add new entries to.


Save Contacts to Computer – Lets you save a backup copy of the current Contact List to your computer. You can access the saved copy again at a later time to edit or add new entries.

Under "Sync Contacts" Sync Contacts

> **Load Contacts from Phone** – Brings the contacts you already have on your CapTel phone into the MyContacts website for you to edit or add photos to.

> Save Contacts to Phone – Saves the current Contact List on the MyContacts website directly to your CapTel phone.

Adding a Contact to Speed Dial

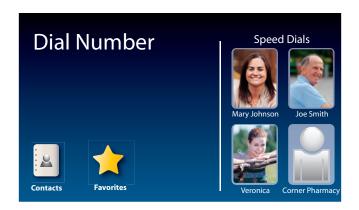

You can set up to four (4) Contacts to be added to Speed Dial. Speed Dial entries appear every time you lift the CapTel handset, for quick one-touch dialing. You can even add photos to your Speed Dial entries for easy dialing by picture.

To add a Contact to Speed Dial:

- 1. From the display screen, touch (Contacts).
- 2. In the Contacts list on the left side of the display, find and touch the name of the person you want to add to Speed Dial. To see contacts further down the list, touch and drag the scroll bar.
- Touch (Next/Prev) to move up/down through entries.
- 3. On the right side of display, touch ((Edit). Touch ((Speed Dial).
- **4.** If there are multiple phone numbers for this contact, the display asks which phone number (mobile, home, or work) you want to make a speed dial. Touch the number(s) you want, then touch (Accept). A little graphic appears by the appropriate phone number to indicate it is added to Speed Dial.
- 5. Touch save (save).
- **6.** When you are finished, press (HOME button) to exit.

Now, when you pick up the handset, this contact will appear automatically on the display screen. Simply touch the person's name or image to dial.

You can add a photograph of your contact person to each Speed Dial entry. See instructions on page 70.

Favorites



This section tells you how to set your most frequently called Contacts as "favorites", for quick dialing and dialing by photo.

TOPICS

- About Favorites
- Adding a Contact to Favorites
- **Dialing Favorites**
- Removing a Contact from Favorites

About Favorites



Favorites let you dial select numbers quickly and easily from the display screen. You can select any entries in your contact list to be included in your favorites list. You can save up to 10 favorites.

JOTE

To add someone to your Favorites list, they must first be entered into your Contacts list. If you want to edit one of your Favorite's information, you must edit their entry in your Contacts list. See Adding a New Contact on page 66.

84 FAVORITES Customer Service: 888-269-7477

Adding a Contact to Favorites



To add a person from your Contact list to Favorites:

- **1.** From the display screen, touch (Contacts).
- 2. In the Contacts list on the left of the display, find and touch the name of the person you want to add to Favorites. To see contacts further down the list, touch and drag the scroll bar or touch (Next/Prev) to move up/down through entries.
- 3. Touch (i) (edit).
- **4.** Touch ((favorite).
- 5. If there are multiple phone numbers for this contact, the display asks which phone number (mobile, home, or work) you want to make a favorite. Touch the number(s) you want, then touch (Accept). A little star appears by the appropriate phone number to indicate it is a favorite.

If your Favorites list is full (10 entries), you need to remove a Favorite first to make room in memory. See page 86 for instructions.

- **6.** Touch save (save).
- 7. When you are finished, press (HOME button) to exit.

Dialing Favorites

To dial a person in your Favorites list:

- 1. From the display screen, touch \uparrow (Favorites).
- **2.** Under the Favorites list on the left of display, find and touch the name of the person you wish to call.
- **3.** On the right side of the display, touch **(Call)**. CapTel dials the phone number automatically.
- **4.** Lift the handset to dial.

Removing a Contact from Favorites

You can remove a person from your Favorites list. Deleting a person from Favorites does NOT delete them from your Contacts list.

To delete a person in your Favorites list:

- 1. From the display screen, touch \uparrow (Favorites).
- **2.** On the left side of the display, find and touch the name of the person you wish to delete.
- 3. Touch (ii) (Delete).
- **4.** CapTel asks you to confirm that you want to remove this contact from Favorites. Touch (yes) to remove the Favorite or (no) to exit.
- **5.** When you are finished, press (HOME button) to exit.

Removing a person from your Favorites list does NOT remove them from your Contacts list.

86 FAVORITES Customer Service: 888-269-7477

SECTION

Settings



This section explains how to adjust settings on your CapTel 2400iBT to fit your preferences.

TOPICS

- Setting the Display **Brightness**
- Screen Timeout
- Night Mode
- Connections
- WiFi Settings
- Bluetooth
- Saving Captions After Hang up
- **Captions Button**
- Ring Flasher
- Saving Conversation Volume

- Keypad Lights
- Language
- Setting the Ringer **Volume**
- Adjusting the Ringer Pitch
- Tone Control
- Time & Date
- About Your CapTel **Phone**
- Updating Your **Software**
- Advanced Settings

Directions for some settings are found in other chapters of this user manual.

Setting the Display Brightness

You can adjust the brightness of the display screen for easy reading in well-lit or dimly-lit areas. The display screen brightness changes as you move the setting so you can see what it will look like.



Screen Timeout

As an energy-saving measure, your CapTel screen will start becoming more dim and gradually go complete dark after 60 seconds of non-use. Touching the display screen restores it to its original brightness. You can set how much time you want the screen to wait before becoming dim.




Night Mode

There may be times that you want your CapTel display to be at a darkened setting, called Night Mode. This is helpful, for example, in a bedroom at night when you don't need the bright display screen. You can set how long Night Mode stays on before the screen goes dark.



Connections

Allows you to connect your CapTel 2400iBT to various networks and devices, including to your WiFi network or to a Bluetooth assistive device such as a headset or neckloop.

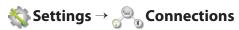

88 SETTINGS Customer Service: 888-269-7477

WiFi Settings

If you use CapTel in a WiFi environment, the Network Settings shows you a listing of all available WiFi networks within range, in order of the strongest signal strength to the weakest. You can change between WiFi networks or save common settings that you use most often.



Bluetooth

You can set up your CapTel 2400iBT to work with a Bluetooth device, giving you the convenience of wireless (cord-free) use of Bluetooth headphones or neckloops. Bluetooth devices sold separately. The Bluetooth Settings lets you "pair" your device to the CapTel phone, and shows you a list of the available Bluetooth devices within range of your phone.



Saving Captions After Hang Up

You can set CapTel to save conversation captions after the call is over for later review, or to erase all captions each time the telephone is hung up. The default setting is Save Captions On.



Captions Button

You can set the CAPTIONS button to automatically stay on or to automatically stay off at the beginning of every call. The default setting is Captions ON. If the CAPTIONS button is set to OFF, captions will not come on automatically. You must press the CAPTIONS button manually in order to turn captions on.



 $frac{4}{4}$ Settings o $frac{4}{4}$ General Settings o captions Button on



Ring Flasher

CapTel includes a bright Ring Flasher that lights up to alert you every time a call comes in or whenever you have answering machine messages waiting. You can adjust the flash rate of the Ring Flasher for different indicators. For example, the Ring Flasher can blink evenly to indicate an answering machine message and can pulse to indicate an incoming call. To adjust the Ring Flash patterns:





Settings → 🏠 General Settings → 📹 Flasher Settings



Saving Conversation Volume

Your CapTel phone amplifies the voice of the caller up to 40dB. Every time you hang up the phone, the volume returns to a mid-range setting. You can set CapTel to remember the conversation volume setting so that it always stays at your desired volume level. The default setting is Save Volume Off.





🗞 Settings → 🖒 General Settings → 👍 Save Volume



90 SETTINGS Customer Service: 888-269-7477

Keypad Lights

The CapTel keypad lights up for easy dialing in dark environments. You can set the keypad lights to go off when your CapTel is not in use. When set to "off" the keypad lights will go off after approximately 30 seconds of non-use. The default setting is Keypad Lights On.



Language

You can set CapTel menus and captions to be in Spanish or English. The default setting is English.



Setting the Ringer Volume

Customer Service: 888-269-7477

You can turn the Ringer Volume up to a setting you hear best. The default setting is five (5). To turn the ringer off, select setting level zero (ringer muted).



Adjusting the Ringer Pitch

You can adjust the frequency of the CapTel ringer to a High, Medium, or Low range that you hear best. The default setting is Med.



Tone Control

You can adjust the frequency of the sound coming through the CapTel handset to a frequency range that you hear best. Select from Low, Medium, High, or Customizable Frequency Ranges. To customize the frequency range, see page 33 for instructions.



Time & Date

You can adjust the time and date displayed on your CapTel phone screen. Touch the white area around the date/number that you want to change.



About Your CapTel Phone

Provides details about how your CapTel phone is set up to help with troubleshooting issues.



Updating Your Software

You can download the most current version of CapTel software to your phone. Once the download is complete, your CapTel phone will re-boot as part of the update process.



92 SETTINGS Customer Service: 888-269-7477

Advanced Settings

Dialing Prefix

Some office environments require you to dial a number in order to get an outside line. For example, you may need to dial a 9 or another number sequence before you can place a call. CapTel 2400iBT lets you add a dialing prefix, then will dial it for you automatically for every call placed from your Contact list, or when using the Speed Dial, Call History, or Answering Machine history. If you dial the phone number directly, you will need to enter the Dialing Prefix at the time you dial. The default setting is no dialing prefix.



Dynamic or Static IP

Using a Dynamic IP address (DHCP) is recommended to avoid IP conflicts. Some networks do not allow DHCP. In those cases, a Static IP address may be used. The default setting is Dynamic IP.



Review Captions on Hang Up

Customer Service: 888-269-7477

You can set your CapTel phone to automatically go to conversation review mode after you hang up. This allows you to review the previous conversation(s) immediately after every call without using the Menu system. The default setting is off.



Line in Use Detector

CapTel can help detect when another telephone in your home or office is off-hook (for example, if someone is talking on another line or another telephone line in your home is not hung up). This can be helpful if multiple people/devices share your telephone line. The default setting is off.



Voice Activity Detector

The built-in answering machine allows one minute for your callers to leave a message. In some cases, your callers' messages may be shorter than one minute. You can set the answering machine to hang up the telephone line once the incoming caller stops talking (stops leaving their message) instead of waiting the full one minute. The default setting is off.



Troubleshooting

Power Reset

On occasion, issues with your Internet connection may impact your CapTel 2400iBT. For example, you may have experienced times when your Internet seems to get "hung up" or things don't look correct on your computer screen. These cases can sometimes impact how your CapTel phone operates. To correct, you need to "reset" your CapTel phone, the same way you might "reset" your computer.

To Reset: Briefly unplug CapTel 2400iBT from the power adapter, wait 10 seconds, then plug it back in. The display screen lights up while the phone powers up. Then the display automatically takes you through the set-up steps again, asks you to confirm your language choice, checks your Internet connection, and confirms your time/date zone. CapTel will also check to make sure you have the most current software, and download any updates as needed.

94 SETTINGS Customer Service: 888-269-7477

CAPTEL 2400 i BT INDEX

Answering a call	Captions
Answering Machine	change font size and style
Audio Jacks22	Contacts65 add new66
Bluetooth answering a call29 connecting a headset/neckloop23–24 ending a call30	add picture to contact66, 68, 70 editing68 erase a contact69 dial from69
Buttons 8, 9 captions 6, 9 customer service 6, 9 dialing keypad 6, 8 flash 6, 8 home 6, 9 mute 6, 9 redial 6, 8 speaker 6, 8, 34–35 volume 6, 8	Conversation Memory
Calls 25 make/answer calls 26–29 use Call History 53	from Call History53 from Contact list67 from Favorites86
Call History 50 dial from 53 review calls 47 review captions from call 51 erase 54 Call waiting 8	from number keypad
<u> </u>	

Flash button 8	Reading captions37
Flasher90	after hang up41
set the ring flasher90	during a call40
Headset22 Bluetooth23–24, 26, 29, 30	from previous call52 Redial button6, 8
use with CapTel25, 29, 30	Reset94
Indicator Lights	Ringer
Internet	Scroll, captions across display
Keypad	Settings
Language	requirements11 using a router15, 16 using wireless/WiFi17–19
Messages, Answering Machine57 erase messages	Sound
Mute button6, 9	customer service in Spanish49, back cover
Neckloop22 use with CapTel22, 23, 29, 30, 32	Speaker button
Network	Speakerphone
available networks17, 89 view saved networks89	Speed dial82 adding contact to82
Power Reset94	adding picture to70 dialing from82

Time & Date	19, 92
Tone Control customize Tone setting	•
Troubleshooting	94
Updating your software	92
Voice mail	61

Volume	8
adjust during a call	31
button	
save conversation volume	90
Ringer volume	91
WiFi/Wireless network	17–19
view networks	18
enter network manually	18

FCC Information

The CapTel has been tested and found to comply with the specifications for a Class B digital device pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates and uses radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the CapTel and the receiver.
- Connect the CapTel to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This equipment complies with Part 68 of the FCC rules and the requirements adopted by the ACTA. A label is located on the underside of the CapTel containing, among other information, an ACTA Product-Labeling Number, US:D8KTE00BCAP2400. If requested, this number must be provided to the telephone company.

A plug and jack used to connect this equipment to the premises wiring and telephone network must comply with the applicable FCC Part 68 rules and requirements adopted by the ACTA. A compliant telephone cord and modular plug is provided with this product. It is designed to be connected to a compatible modular jack that is also compliant. See installation instructions for details.

The REN is used to determine the quantity of devices which may be connected to the telephone line. Excessive RENs on the line may result in the devices not ringing in response to an incoming call. In most, but not all, areas the sum of RENs should not exceed five (5.0). To be certain of the

number of devices that may be connected to a line, as determined by the total RENs, contact the local telephone company. The REN for this product is 0.0B.

Should you experience trouble with this telephone equipment, please contact:

CapTel Customer Service 450 Science Drive Madison, WI 53711 Phone: 888-269-7477 V/TTY Email: CapTel@CapTel.com

For repair or warranty information, please contact CapTel Customer Service at 1-888-269-7477 (V/TTY). If the equipment is causing harm to the telephone network, the telephone company may request that you disconnect the equipment until the problem is resolved.

This equipment cannot be used on public coin phone service provided by the telephone company. Connection to party line service is subject to state tariffs.

If the CapTel causes harm to the telephone network, the telephone company will notify you in advance that temporary discontinuance of service may be required. But, if advance notice is not practical, the telephone company will notify you as soon as possible. Also, you will be advised of your right to file a complaint with the FCC if you believe it is necessary.

Occasionally, your telephone company may make changes in its facilities, equipment, operation, or procedures that could affect the operation of your equipment. If so, you will be given advance notice of the change to give you an opportunity to maintain uninterrupted service.

If your home has specially wired alarm equipment connected to the telephone line, ensure the installation of the CapTel does not disable it. If you have questions about what will disable alarm equipment, consult your telephone company or a qualified installer.

The CapTel telephone does not have any userserviceable parts. Modification or changes to the CapTel not expressly approved by Ultratec, Inc. can void your authority to operate the equipment.

98 FCC INFORMATION Customer Service: 888-269-7477

Compliance Statement (Part 15.19)

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- 1. This device may not cause harmful interference, and
- 2. This device must accept any interference received, including interference that may cause undesired operation.

Warning (Part 15.21)

Changes or modification not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

FCC Interference Statement (Part 15.105b)

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

RF Exposure (OET Bulletin 65)

To comply with FCC/IC exposure requirements for mobile transmitting devices, this transmitter should only be used or installed at location where there is at least 20 cm separation distance between the antenna and all persons.

To comply with FCC/IC RF exposure limits for general population/uncontrolled exposure, the antenna(s) used for this transmitter must be installed to provide a separation distance of at least 20 cm from all persons and must not be co-located or operating in conjunction with any other antenna or transmitter.

Getting Help

If you have any questions as you use your new CapTel 2400iBT, we're here to help. Use any of these ways to contact us:

Pick up the handset and press (CUSTOMER SERVICE button) to automatically speed dial directly to our helpful Customer Service team. If you are in an office setting, you may need to dial a number such as "9" first to get an outside line first and then press (CUSTOMER SERVICE button). (Available 24 hours a day, 7 days a week. Customer Service is closed on major holidays.)

Help materials are also available online at our website.

Phone: 1-888-269-7477 (Available 24 hours a day, 7 days a week)
FAX: (608) 204-6167

Mail: CapTel Customer Service

450 Science Drive Madison, WI 53711

Email: CapTel@CapTel.com

Online: www.CapTel.com
(Live Chat help available at our website)

Spanish Language Customer Service: 1-866-670-9134

CapTel is the latest innovation from **Ultratec**.

Ultratec, Inc. 450 Science Drive Madison, WI 53711 (888) 269-7477 V/TTY

(866) 670-9134 (Spanish CapTel Customer Service)

Email: CapTel@CapTel.com • Fax: (608) 204-6167 • Web: www.CapTel.com

©2016 Ultratec, Inc. All rights reserved.

Ultratec and CapTel are registered trademarks of Ultratec, Inc.

The Bluetooth® word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by Ultratec is under license. Other trademarks and trade names are those of their respective owners.