


Instructions for Use – Central Line Dressing Securement


3 Easy Steps for CVC Dressing Securement

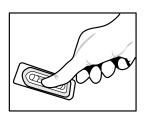

1. Crack: Follow your facility's protocol for CVC site preparation. (Mastisol is compatible with CHG*). Place an individual vial in the blister-pak on a hard surface (label facing down). Gently apply pressure to the vial through the blister-pak to crush the inner ampule, thereby releasing the Mastisol.




2. Peel: Peel back the lid and remove the individual vial using sterile or aseptic technique.

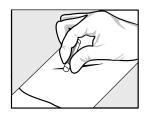

3. Apply: Apply a half-inch strip of Mastisol to the border that will be covered by the dressing. (See correct application in diagram below.) Allow the Mastisol to dry for 20-30 seconds. Using an established technique, place the dressing in contact with dried Mastisol.

For demonstration purpose only, Mastisol is indicated by the color yellow. When Mastisol is actually applied, it will appear clear in color and glisten on the skin.

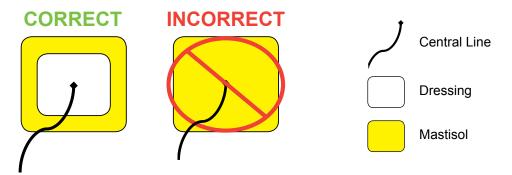

For best results to remove dressing, use Detachol[®] Adhesive Remover. Start at the corner of the dressing and apply Detachol while slowly pulling back.

Irritant or allergic-type application site reactions, especially in patients who have previously developed sensitivity to benzoin or other adhesive components, have been reported. Caution: Flammable. Keep out of reach of children.

Warning: For external use only. Avoid contact with the eyes. Irritation or infection may result if the product comes in contact with the eyes. If accidental contact occurs, rinse immediately with water. Contains: Gum Mastic, Styrax Liquid, SD Alcohol 23A, and Methyl Salicylate.



* Data on file Mastisol and Detachol are registered trademarks of Ferndale IP, Inc., licensed to Eloquest Healthcare, Inc.