



## Mastisol Spray Instructions for Use

Mastisol<sup>®</sup> Liquid Adhesive is used to ensure the integrity of dressings, tapes and devices over an extended period of time. Mastisol is non-water soluble and will enhance adhesive performance.

- ✓ Clean and thoroughly dry the area where Mastisol will be applied.
- ✓ Hold spray pump approximately 6-8 inches from the skin. Spray a fine mist of Mastisol around the area where the bandage will be secured. Do not saturate the skin. Avoid contact with non-intact skin.
- ✓ Allow Mastisol to dry for 20-30 seconds.
- ✓ Place the dressing over the desired area so that the edges come into contact with Mastisol.



Irritant or allergic-type application site reaction, especially in patients who have previously developed sensitivity to benzoin or other adhesive components, have been reported.

**Caution:** Flammable. Keep out of reach of children

**Contains:** Gum Mastic, Styra Liquid, SD Alcohol 23A and Methyl Salicylate.

**Warnings:** For external use only. Avoid contact with the eyes. Irritation or infection may result if the product comes in contact with the eyes. If accidental contact occurs, rinse immediately with water.

**Eloquest**  
HEALTHCARE<sup>®</sup>

780 W. Eight Mile Rd, Ferndale, MI 48220  
877.433.7626

[www.eloquesthealthcare.com](http://www.eloquesthealthcare.com)