


mastisol[®]

LIQUID ADHESIVE


When you need it on firmly,
securely, easily, and safely.



Mastisol is a liquid medical adhesive
used to ensure the integrity of
dressings, tapes, and certain devices
over an extended period of time.

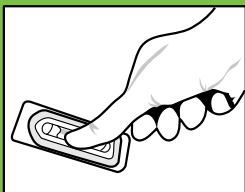

1 CRACK

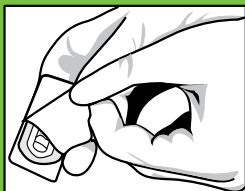

2 PEEL



3 APPLY

3 EASY STEPS FOR SUPERIOR HOLDING POWER

Clean and dry the area where Mastisol will be applied. Place an individual vial in the blister-pak on a hard surface (label facing down) and gently apply pressure to crush the inner ampule, releasing Mastisol.

Peel back the lid and remove the individual vial using sterile or aseptic technique.

Apply Mastisol to the area that will be covered by tape, dressing or device by gently squeezing the vial. Avoid non-intact skin. Allow Mastisol to dry for 20-30 seconds. Using an established technique, place the tape, dressing or device in contact with dried Mastisol.

Irritant or allergic-type application site reactions, especially in patients who have previously developed sensitivity to benzoin or other adhesive components, have been reported.

Caution: Flammable. Keep out of reach of children.

Warning: For external use only. Avoid contact with the eyes.


Irritation or infection may result if the product comes in contact with the eyes. If accidental contact occurs, rinse immediately with water.

Contains: Gum Mastic, Styrax Liquid, SD Alcohol 23A, and Methyl Salicylate.


COMMON MASTISOL APPLICATION AREAS




**CENTRAL
LINE IV
DRESSINGS**

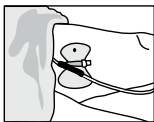

**CATHETER
ANCHOR
DEVICE**

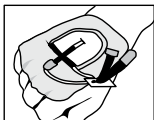

**PERIPHERAL
IV DRESSINGS**

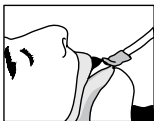

**ENDOTRACHEAL
TUBE**

Eloquest
HEALTHCARE™

780 West Eight Mile Road, Ferndale, MI 48220
877-433-7626 • www.eloquesthealthcare.com

Mastisol is a registered trademark of Ferndale IP, Inc.
MA41 4/12