

[Click here to view detailed product descriptions and prices.](#)

Call 800-397-5899 to order today!



IV3000[®]

Moisture Responsive
Catheter Dressing

IV3000 Application

IV3000 10cm x 12cm

Suitable for dressing central line catheters

Step A



Clean and thoroughly dry the application area.

First remove section **1** from the end of the dressing. This contains 2 strips and a date label.


Step B



The strips can be used on the catheter wings or body for improved security, to secure tubing or to stabilise catheter lumens.

Peel off the backing paper marked **2**.

Step C

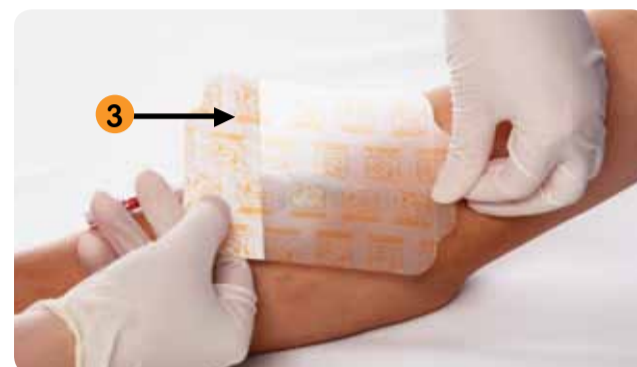

Centre the dressing over the catheter insertion site.

Place the dressing on the catheter hub and adhere the dressing to the patient's skin.

Remove the remaining protector paper marked **3**.

Step D



Using the tab gently peel off the top carrier marked **4**.

Step E



Mould the film around the catheter, ensuring the dressing is secure and the patient is comfortable.

Finally place the date label from section **1** on the skin or the dressing (without obscuring the view of the catheter site).

Removal

Take one corner of the film and gently pull parallel to the skin (stabilising the patient's skin and catheter with the other hand).

Release and repeat the process.



Ordering codes

Dressing						
Size	5cm x 6cm	6cm x 7cm	7cm x 9cm	9cm x 12cm	10cm x 12cm	11cm x 14cm
S&N Code	66004011	4007	4006	66004009	4008	66800512
NHS Code	ELW011	ELW032	ELW046	ELW112	ELW054	ELW363
Carton	100	100	100	50	50	25

Wound Management
Smith & Nephew
Healthcare Ltd
Healthcare House
101 Hessele Road
Hull HU3 2BN

T 01482 222200
F 01482 222211

www.smith-nephew.com/wound
www.iv3000.com

[®]Trademark of Smith & Nephew
© Smith & Nephew May 2011
27284


IV3000 Application

IV3000 Ported 11cm x 14cm

Suitable for PICC and midline catheters


Application with a stabilisation device

Step A

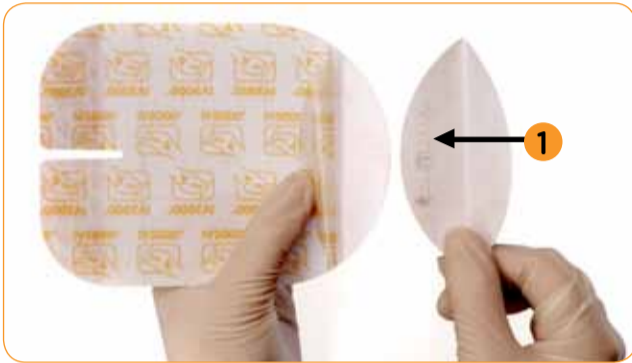

Clean and thoroughly dry the application area. First remove section 1 from the rounded end of the dressing. This contains 2 strips and a date label.

Step B

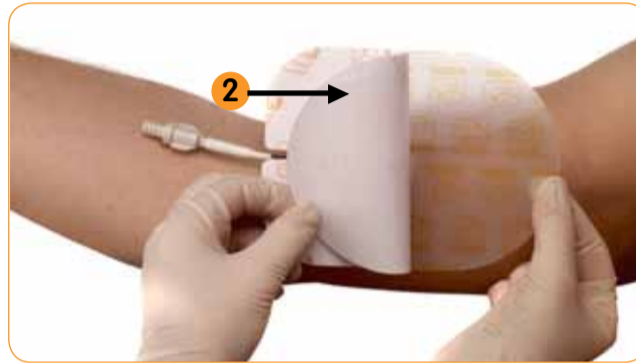


Peel off the backing paper marked 2

Step C



Place the dressing over the cannula site, ensuring that the entry site is as close to the centre of the dressing (clear section) as possible.

Step D

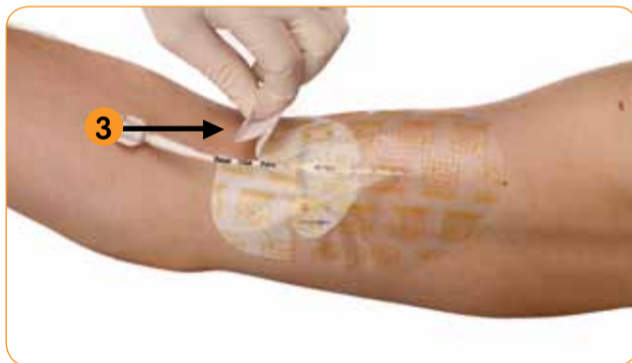

Arrange the ported area around the lumen and remove the backing papers marked 3 from the non-woven wings.

Step E



Using the tab gently peel off the top carrier marked 4. Mould the film around the catheter, ensuring that the dressing is secure and the patient is comfortable.

Step F



The securing strips from section 1 can be used on the catheter for improved security to secure tubing or to stabilise catheter lumens. Finally place the date label from section 1 on the skin or the dressing (without obscuring the view of the catheter site).



Removal

Gently peel back the non-woven wings to the point at which they reach the film section (stabilising the patient's skin and catheter with the other hand). Take one corner of the film and gently pull parallel to the skin. Release and repeat the process.

Ordering codes

Dressing						
Size	5cm x 6cm	6cm x 7cm	7cm x 9cm	9cm x 12cm	10cm x 12cm	11cm x 14cm
S&N Code	66004011	4007	4006	66004009	4008	66800512
NHS Code	ELW011	ELW032	ELW046	ELW112	ELW054	ELW363
Carton	100	100	100	50	50	25

Wound Management

Smith & Nephew
Healthcare Ltd
Healthcare House
101 Hessle Road
Hull HU3 2BN

T 01482 222200
F 01482 222211

www.smith-nephew.com/wound
www.iv3000.com


[®]Trademark of Smith & Nephew
© Smith & Nephew May 2011
27279

IV3000 Application

IV3000 Ported 5cm x 6cm

Suitable for dressing paediatric catheters

Step A

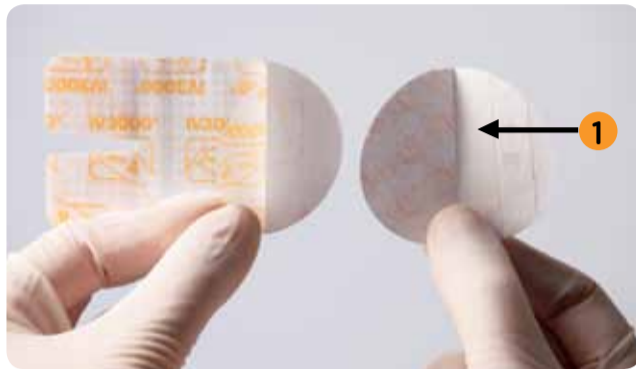

Clean and thoroughly dry the application area.

First remove section **1** from the rounded end of the dressing.

This contains 2 strips and a date label.

Step B



The strips can be used on the catheter wings or body for improved security, to secure tubing or to stabilise catheter lumens.

Step C



Remove the backing paper marked **2**.

Step D



Position the dressing over the catheter port. Remove the backing papers marked **3** from the non woven sections and adhere over the catheter wings.

Using the tab gently peel off the top carrier marked **4**.

Step E



Mould the film around the catheter, ensuring that the dressing is secure and the patient is comfortable.

Finally place the date label from section **1** on the skin or the dressing (without obscuring the view of the catheter site).









Removal

Gently peel back the non-woven wings to the point at which they reach the film section (stabilising the patient's skin and catheter with the other hand).

Take one corner of the film and gently pull parallel to the skin.

Release and repeat the process.

Ordering codes

Dressing						
Size	5cm x 6cm	6cm x 7cm	7cm x 9cm	9cm x 12cm	10cm x 12cm	11cm x 14cm
S&N Code	66004011	4007	4006	66004009	4008	66800512
NHS Code	ELW011	ELW032	ELW046	ELW112	ELW054	ELW363
Carton	100	100	100	50	50	25

Wound Management
Smith & Nephew
Healthcare Ltd
Healthcare House
101 Hesse Road
Hull HU3 2BN

T 01482 222200
F 01482 222211

www.smith-nephew.com/wound
www.iv3000.com


[®]Trademark of Smith & Nephew
© Smith & Nephew May 2011
27280

IV3000 Application

IV3000 6cm x 7cm


Suitable for dressing peripheral catheters

Step A

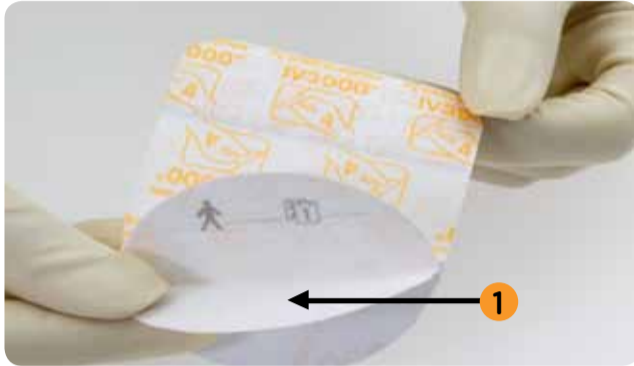


Clean and thoroughly dry the application area.
First remove section **1** from the rounded end of the dressing.
This contains 2 strips and a date label.

Step B

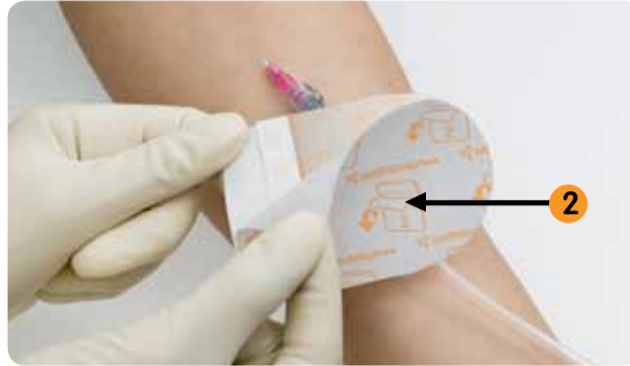


The strips can be used on the catheter wings or body for improved security, to secure tubing or to stabilise catheter lumens.
Remove the backing paper marked **2**.

Step C

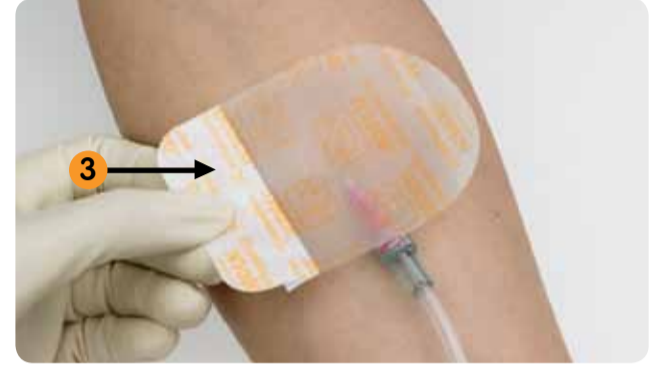

Centre the dressing over the catheter insertion site.
Place the dressing on the catheter port and adhere the dressing to the patients skin.
Remove the remaining protector paper marked **3**.

Step D

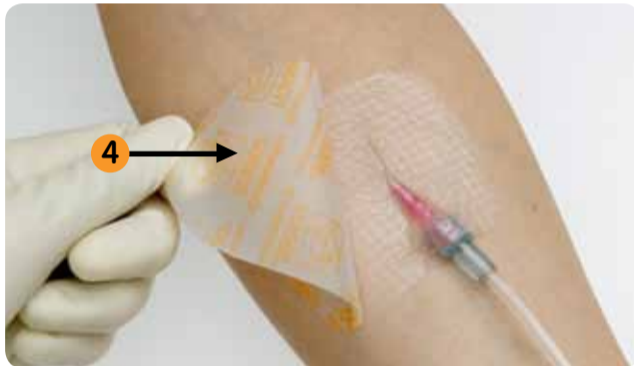





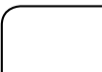

Using the tab gently peel off the top carrier marked **4**.
Mould the film around the catheter to complete dressing application.
Finally place the date label from section **1** on the skin or the dressing (without obscuring the view of the catheter site).

Removal

Take one corner of the film and gently pull parallel to the skin.
Release and repeat the process.



Ordering codes

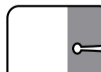Dressing						
Size	5cm x 6cm	6cm x 7cm	7cm x 9cm	9cm x 12cm	10cm x 12cm	11cm x 14cm
S&N Code	66004011	4007	4006	66004009	4008	66800512
NHS Code	ELW011	ELW032	ELW046	ELW112	ELW054	ELW363
Carton	100	100	100	50	50	25

Wound Management

Smith & Nephew
Healthcare Ltd
Healthcare House
101 Hesse Road
Hull HU3 2BN

T 01482 222200
F 01482 222211

www.smith-nephew.com/wound
www.iv3000.com


[®]Trademark of Smith & Nephew
© Smith & Nephew May 2011
27281

IV3000 Application

IV3000 Ported 7cm x 9cm

Suitable for dressing peripheral catheters

Step A

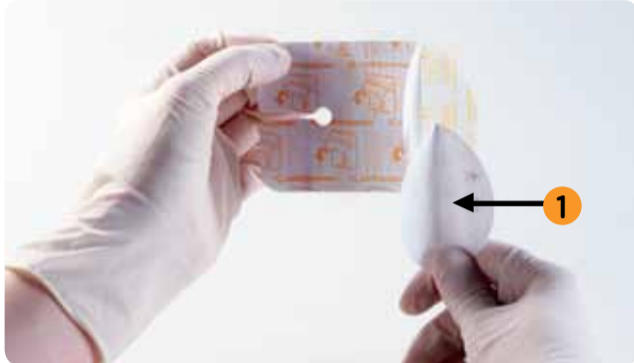

Clean and thoroughly dry the application area.
First remove section **1** from the rounded end of the dressing.
This contains 2 strips and a date label.

Step B



The strips can be used on the catheter wings or body for improved security, to secure tubing or to stabilise catheter lumens.
Remove the backing paper marked 2.

Step C



Position the dressing over the catheter port.
Remove the backing papers marked **3** from the non woven sections and adhere over the catheter wings.

Step D



Using the tab, gently peel off the top carrier marked **4**

Step E




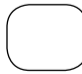
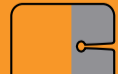
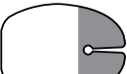
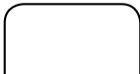
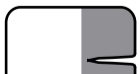
Mould the film around the catheter, ensuring that the dressing is secure and the patient is comfortable.
Finally place the date label from section **1** on the skin or the dressing (without obscuring the view of the catheter site).



Removal

Gently peel back the non-woven wings to the point at which they reach the film section (stabilising the patient's skin and catheter with the other hand).
Take one corner of the film and gently pull parallel to the skin. Release and repeat the process.

Ordering codes

Dressing						
Size	5cm x 6cm	6cm x 7cm	7cm x 9cm	9cm x 12cm	10cm x 12cm	11cm x 14cm
S&N Code	66004011	4007	4006	66004009	4008	66800512
NHS Code	ELW011	ELW032	ELW046	ELW112	ELW054	ELW363
Carton	100	100	100	50	50	25

Wound Management
Smith & Nephew
Medical Ltd
101 Hesse Road
Hull HU3 2BN

T +44 (0) 1482 225181
F +44 (0) 1482 328326

www.smith-nephew.com/wound
www.iv3000.com


®Trademark of Smith & Nephew
© Smith & Nephew May 2011
27282

IV3000 Application

IV3000 Ported 9cm x 12cm

Suitable for dressing jugular and central line catheters

Step A

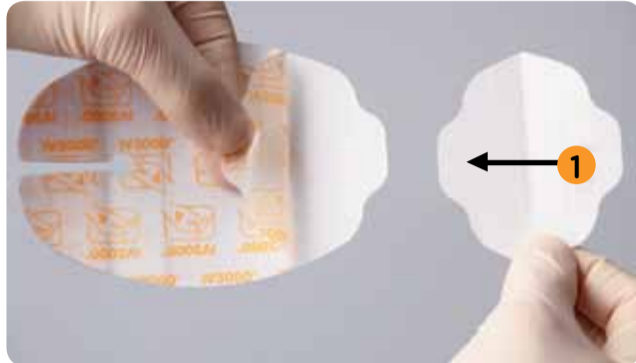

Clean and thoroughly dry the application area.
First remove section **1** from the rounded end of the dressing.
This contains 2 strips and a date label.

Step B



The strips can be used on the catheter wings or body for improved security, to secure tubing or to stabilise catheter lumens.
Peel off the backing paper marked **2**. Place the dressing over the cannula site, ensuring that the entry site is as close to the centre of the dressing (clear section) as possible.

Step C



Arrange the ported area around the lumen and remove the backing papers marked **3** from the non-woven wings.
The non-woven wings can be gently pulled together and overlapped to ensure the weight of the lumen is well secured.

Step D



Peel off the top film carrier marked **4** using the tab at the base of the dressing, pulling gently towards the direction of the port.

Step E




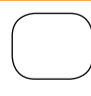
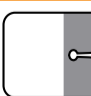



Mould the film around the catheter, ensuring that the dressing is secure and the patient is comfortable.
Finally place the date label from section **1** on the skin or the dressing (without obscuring the view of the catheter site).



Removal

Gently peel back the non-woven wings to the point at which they reach the film section (stabilising the patient's skin and catheter with the other hand).
Then take one corner of the film and gently pull parallel to the skin.
Release and repeat process.

Ordering codes

Dressing						
Size	5cm x 6cm	6cm x 7cm	7cm x 9cm	9cm x 12cm	10cm x 12cm	11cm x 14cm
S&N Code	66004011	4007	4006	66004009	4008	66800512
NHS Code	ELW011	ELW032	ELW046	ELW112	ELW054	ELW363
Carton	100	100	100	50	50	25

Wound Management
Smith & Nephew
Healthcare Ltd
Healthcare House
101 Hessle Road
Hull HU3 2BN
T 01482 222200
F 01482 222211

www.smith-nephew.com/wound
www.iv3000.com

[®]Trademark of Smith & Nephew
© Smith & Nephew May 2011
27283