| | |
|---------------------------------------|--------------------------------------|
| Part # | D454 |
| Revision | E |
| Description | LITTLE SUCKER DFU |
| Artwork # | AW4454 |
| Material # (PM, DM, LM) & description | DM466 50# opaque smooth offset White |
| Effective date | |
| CO # | |

NEOTECH™
Products, Inc.

Little Sucker®

Oral and Nasal Suction Device

REF Catalog No.

N204 Premie
N204TP Premie
N205 Standard
N205TP Standard**DIRECTIONS FOR USE**
INSTRUCTIONS D'UTILISATION
NÁVOD K POUŽITÍ
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
ZALECENIA DOTYCZĄCE
UŻYTKOWANIAManufactured by
NEOTECH PRODUCTS, INC.
28430 Witherspoon Pkwy, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.comEMERGO EUROPE
Molenstraat 15
2513 BH, The Hague, The Netherlands

Made in USA

D454 Rev E

English

Little Sucker®
Oral and Nasal Suction Device**FEATURES & BENEFITS**

- Designed for single handed suctioning
- Thumb port for intermittent suctioning
- Soft, flexible tip similar to a bulb syringe
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

INDICATIONS FOR USE

The Little Sucker is indicated for use as a vacuum powered suction device to aspirate fluids from the mouth and nares.

DIRECTIONS FOR USE

Step 1 Connect barbed end of Little Sucker to suction line.

Step 2 Set suction to appropriate pressure per hospital protocol or physician recommendation.

Step 3 Block thumb port with thumb to begin suctioning.

Step 4 Suction mouth or nares with intermittent suctioning.

Note: For intermittent suctioning, unblock thumb port to stop suctioning. Block thumb port again to continue suctioning.

Step 5 Suction sterile water through Little Sucker after each use.

Step 6 Replace Little Sucker every 24 hours or per hospital protocol, whichever is sooner.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

For oral and nasal use only.

Inserting the Little Sucker too far into the mouth or nares can cause tissue trauma.

The Little Sucker should remain free of debris.

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