# UNIVERSAL ALUMINIUM CRUTCH

## MULETAS UNIVERSALES DE ALUMINIO

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#### MOBILITY • MOVILIDAD • MOBILITÉ

#### LIFETIME LIMITED WARRANTY

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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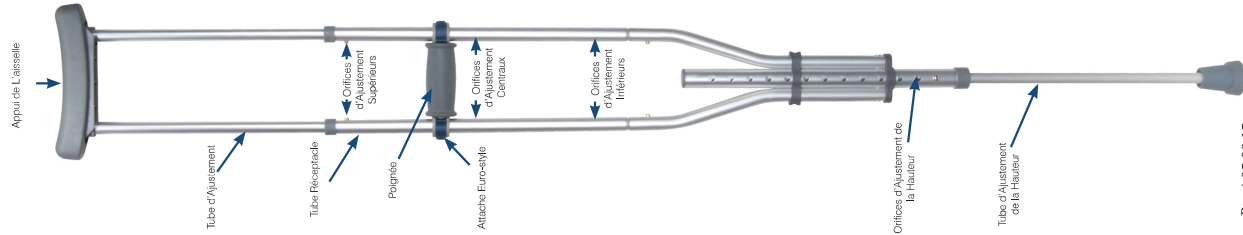
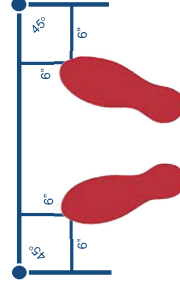
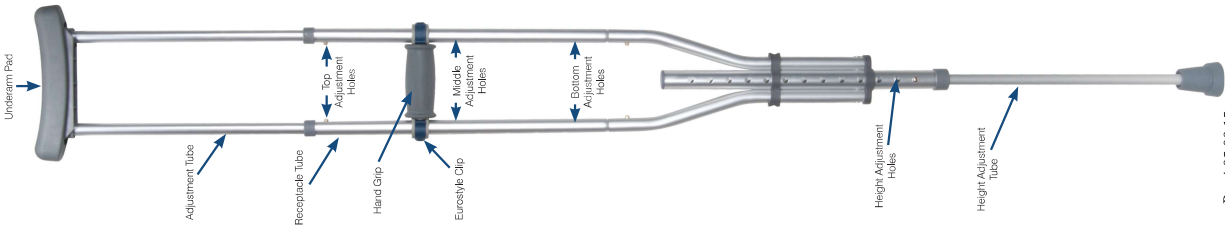
**POIDS MAXIMAL**  
300 LB / 136 KG

## ASSEMBLY INSTRUCTIONS

For best fit and assembly, see instructions below.

1. Remove contents from package.
2. With one hand, place bottom section on floor with tip securely on the ground. Insert top section into tubing as shown. Depress both brass push pins simultaneously with thumbs and slide top half into bottom half until the push pins "pop" through holes. You will hear an audible click when top section is secure.

## HEIGHT ADJUSTMENT

1. Unlock both blue euro-style clips on hand grip by using thumbs and pressing outward on thumb tab.
2. With both thumbs, push hand grip down in order to have full access to brass push pins.

**FOR INDIVIDUALS BETWEEN 4'6" AND 5'2"**

3. Set underarm adjustment tube in first (lowest from ground) set of **INSIDE** adjustment holes on the top crutch section, making sure push pins protrude all the way through the adjustment holes. Continue to step 4.

**FOR INDIVIDUALS BETWEEN 5'3" AND 5'10"**

3. Turn crutch upside down so underarm pad is on the floor. With foot, hold underarm pad to the floor and pull up on receptacle tube while depressing push pins. Set underarm adjustment tube in the second (middle) set of **INSIDE** adjustment holes on the top crutch section, making sure push pins protrude all the way through the adjustment holes. Continue to step 4.

**FOR INDIVIDUALS BETWEEN 5'11" AND 6'6"**

3. Turn crutch upside down so underarm pad is on the floor. With foot, hold underarm pad to the floor and pull up on receptacle tube while depressing push pins. Set underarm adjustment tube in the third (highest from ground) set of **INSIDE** adjustment holes on the top crutch section, making sure push pins protrude all the way through the adjustment holes.

4. Adjust bottom height adjustment tube by depressing both silver push pins and rotating slightly so push pins flow freely. Slide to desired height and turn until silver push pins protrude through holes.
5. Set hand grip by placing underarm pad under amput and sliding hand grip so that the elbow has a 10° bend.
6. When correct position is achieved, lock euro-style clips in place by inserting into positioning hole and locking push tab.
7. Repeat for second crutch.

## CAUTIONS

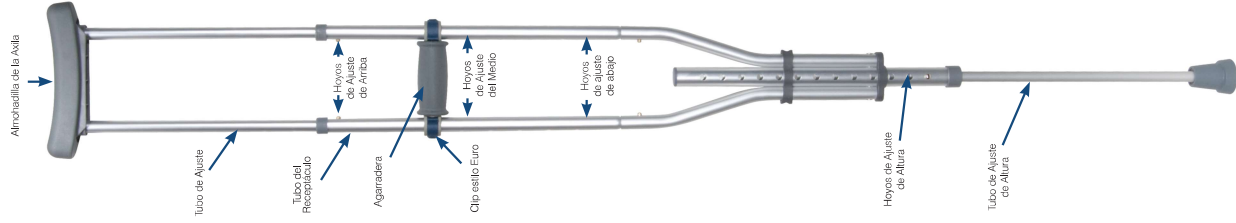
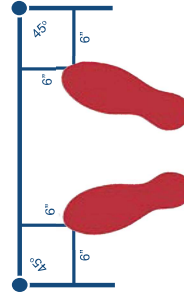
- Use after proper training under the direction of your doctor or physical therapist.
- Replace Crutch Tips if worn.
- Use extra caution on wet and slippery surfaces.

## SAFETY CHECK

With crutches positioned per diagram, confirm patient can lean forward comfortably while resting on the underarm supports. Patient should not have to stand uncomfortably erect. Ensure both push-buttons are firmly in place and handgrips are secure.

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