

PROCARE®



MaxTrax™
Walking Braces

*An Evolution
In Comfort*

ProCare MaxTrax™

Developing a walking brace while maintaining maximum performance and patient comfort is a collective effort. Unique, innovative features. High quality materials. World-class manufacturing. Not just one, but the sum of all the parts is how we developed the MaxTrax walker.

Every MaxTrax Walker Features:

- Pediatric sizing through extra large
- Wider footbed for increased stability
- A one-piece liner design to keep sewn seams away from critical post-surgical areas.
- Swiveling D-rings to allow for criss-cross or parallel strapping
- Cushioned inner/outer sole designed to absorb shock upon heel strike and improve patient comfort
- Rocker Sole to help reduce plantar pressures
- Malleable uprights that breakdown on all standard height walkers to accommodate patients with shorter tibia/fibula
- Low profile non-slip outer sole promotes a natural gait

MaxTrax Air Walker Liner Features:

- Adjustable air chambers to accommodate changes in edema during rehab and maximize patient compliance
- Integrated pump designed to make the liner simple to inflate and adjust
- A dual air-cell system that inflates the medial and lateral air chambers equally to maintain limb contact for improved stability

Experience a higher standard for patient care with MaxTrax walkers.

MaxTrax Walker Intended Use

The MaxTrax walkers are intended to increase stability following trauma or post-operative procedures including:

- Acute ankle sprains
- Stress fractures of the lower leg
- Soft tissue injuries (grade 2 and 3 sprains)
- Stable fractures and injuries of the foot and ankle
- Bunionectomies
- Metatarsal fractures
- Achilles tendon repair

MaxTrax Air Walkers Also Provide:

- Edema Control
- Maintain limb contact for increased stability



Unique Strapping



Cushioned Sole



Breakdown Uprights

MaxTrax Air Walkers

The MaxTrax Air walkers have all the features of the standard walkers plus the benefit of a pneumatic liner, providing support and compression to help promote fracture healing. Designed to increase stability and help reduce pain and edema following trauma or post-operative procedures, the MaxTrax Air walkers also provide a cost-effective option to serial casting.

MaxTrax Range-of-Motion Walkers

A patented hinge design is incorporated into the MaxTrax ROM walkers providing protected range-of-motion throughout rehabilitation in 7.5° increments from 45° plantar-flexion to 30° dorsi-flexion. The foot and ankle may also be completely immobilized at 0°, 7.5°, 15°, 22.5°, and 30° of plantar or dorsi-flexion.

MaxTrax Diabetic Walker

The MaxTrax Diabetic walker features a wider foot bed, malleable uprights and a shock absorbing heel, making it an ideal solution for the treatment of plantar ulcers. The tri-laminate insole designed with the Impax Grid Technology, allows for removal of foam under the ulcer thus reducing pressure at the wound site. The bottom two layers of foam can be easily removed around the affected areas, redistributing load away from healing ulcers. The key benefit in the design is that the top layer of foam stays in contact with the plantar surface of the foot, not allowing "hot spots."



MaxTrax Air



MaxTrax ROM Walker



MaxTrax Diabetic Walker



MaxTrax Impax Grid



The MaxTrax Air walkers are designed to increase circulation and help decrease pain and edema.

Support
Comfort
Stability



MaxTrax Air Walker

Ideal following bunionectomies, acute ankle sprains and stable fractures of the lower leg and forefoot.

TALL AIR	REPLACEMENT LINER	ANKLE AIR	REPLACEMENT LINER	SHOE SIZE MALE	SHOE SIZE FEMALE	SHOE SIZE JUNIOR	SIZE
79-95412	79-95362	79-95422	79-95372		≤ 4	≤ 5	XSmall, Ped.
79-95413	79-95363	79-95423	79-95373	≤ 5	4.5 – 6		Small
79-95415	79-95365	79-95425	79-95375	5.5 – 10	6.5 – 11		Medium
79-95417	79-95367	79-95427	79-95377	10.5-13.5	11.5 – 14.5		Large
79-95418	79-95368	79-95428	79-95378	14 – 17			XLarge



MaxTrax Walker

Ideal following metatarsal fractures, bunionectomies, acute ankle sprains, stress and stable fractures of the foot.

TALL WALKER	REPLACEMENT LINER	ANKLE	REPLACEMENT LINER	SHOE SIZE MALE	SHOE SIZE FEMALE	SHOE SIZE JUNIOR	SIZE
79-95322	79-95192	79-95342	79-95182		≤ 4	≤ 5	XSmall, Ped.
79-95323	79-95193	79-95343	79-95183	≤ 5	4.5 – 6		Small
79-95325	79-95195	79-95345	79-95185	5.5 – 10	6.5 – 11		Medium
79-95327	79-95197	79-95347	79-95187	10.5 – 13.5	11.5 – 14.5		Large
79-95328	79-95198	79-95348	79-95188	14 – 17	NA		XLarge



MaxTrax ROM Walker

Provides protected range-of-motion following trauma or post-operative procedures. ROM settings between 45° plantar-flexion and 30° dorsi-flexion in 7.5° increments. Uprights will lock in fixed positions of 0°, 7.5°, 15°, 22.5°, and 30° plantar- and dorsi-flexion. Malleable uprights breakdown if needed for custom fit. Available in Pneumatic Tall, Standard Tall and Ankle.

TALL ROM AIR	REPLACEMENT LINER	SHOE SIZE MALE	SHOE SIZE FEMALE	SHOE SIZE JUNIOR	SIZE
79-95432	79-95362		≤ 4	≤ 5	XSmall, Ped.
79-95433	79-95363	≤ 5	4.5 – 6		Small
79-95435	79-95365	5.5 – 10	6.5 – 11		Medium
79-95437	79-95367	10.5-13.5	11.5 – 14.5		Large
79-95438	79-95368	14 – 17			XLarge



TALL ROM	REPLACEMENT LINER	ANKLE ROM	REPLACEMENT LINER	SHOE SIZE MALE	SHOE SIZE FEMALE	SHOE SIZE JUNIOR	SIZE
79-95332	79-95192	79-95352	79-95182		≤ 5	≤ 5	XSmall, Ped.
79-95333	79-95193	79-95353	79-95183	≤ 5	4.5 – 6		Small
79-95335	79-95195	79-95355	79-95185	5.5 – 10	6.5 – 11		Medium
79-95337	79-95197	79-95357	79-95187	10.5-13.5	11.5 – 14.5		Large
79-95338	79-95198	79-95358	79-95188	14 – 17			XLarge



MaxTrax Diabetic Walker

Ideal for replacement of total contact casting; treatment of pre-ulcerative or ulcerative conditions of the plantar surface of the foot; charcot foot. Included, optional Toe Cover and a set of Upright Pads for additional patient comfort.

TALL WALKER	REPLACEMENT LINER	IMPAX FOAM INSOLE	SHOE SIZE MALE	SHOE SIZE FEMALE	SIZE
79-95453	79-95473	79-95463	≤ 5	≤ 4	Small
79-95455	79-95475	79-95465	5.5 – 10	4.5 – 6.5	Medium
79-95457	79-95477	79-95467	10.5-13.5	6.5 – 11	Large
79-95458	79-95478	79-95468	14 – 17	11.5 – 14.5	X-Large

Accessories

PART NUMBER	DESCRIPTION	SIZE
79-95402	Anterior/Posterior and Medial/Lateral Shells	XSmall – Small
79-95403	Anterior/Posterior and Medial/Lateral Shells	Medium – XL
79-95450	MaxTrax Diabetic Toe Cover	U
79-95460	MaxTrax Upright Pads (1pair)	U
79-95130	Achilles Wedge	U
79-95093	Weather Cover	Small
79-95095	Weather Cover	Medium
79-95097	Weather Cover	Large



1430 Decision Street
Vista, CA 92081-8553, U.S.A.

T 800.793.6065

F 760.734.4712

djoglobal.com

DJO Global proudly sponsors:

