

**Recommended Wound Conditions**

- Shallow   ● No/Minimal drainage   ● Primary dressing
- Deep   ● Moderate/Heavy drainage   ● Secondary dressing

# Optifoam<sup>®</sup>

## Foam Wound Dressing

FOAM

### About Optifoam

- Moisture vapor transmission rate adjusts to fluid level
- Will not curl at edges (*adhesive*)
- Highly absorbent
- Helps create ideal healing environment
- Waterproof outer layer protects wound

### Indications

- Pressure ulcers
- Partial and full-thickness wounds
- Leg ulcers
- Donor sites
- Lacerations and abrasions
- Skin tears
- First and second-degree burns

### Contraindications

- Third-degree burns
- Lesions with active vasculitis


### Change Frequency

- Optifoam may be left in place for up to 7 days
- Dressing change frequency will depend on amount of drainage

### Recommended Secondary Dressings

- Medfix Tape (*for Optifoam Non-Adhesive*)
- Elastic Net (*for Optifoam Non-Adhesive*)
- Optifoam Thin can be used as a secondary dressing

### Fluid Handling Comparative Study<sup>1</sup>

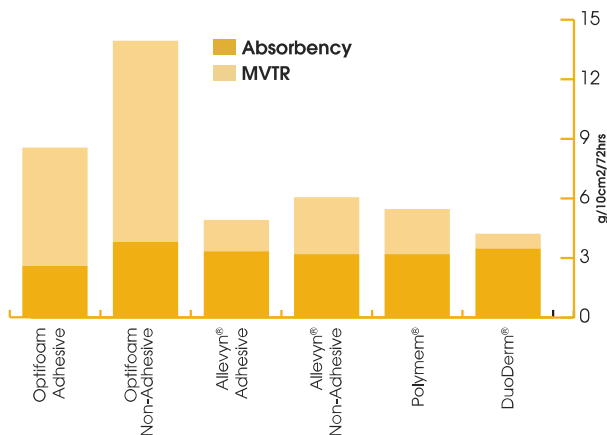

Powerful ability to manage (absorb + transpire) wound fluids.

**Evidence Based References**


1. Data on file.


Debridement  
Infection/Inflammation  
Moisture Balance  
Edge/Environment  
Support





## Ordering Information


Soft, conformable foam dressing.

<b>Optifoam Adhesive</b> An all-in-one dressing for fluid handling				
	Item Number	Description	HCPCS	Packaging
	MSC1044EP	4" x 4" (10.16 x 10.16 cm), 2.5" x 2.5" (6.35 x 6.35 cm) pad	A6212	10/bx, 10 bx/cs
	MSC1066EP	6" x 6" (15.24 x 15.24 cm), 4.5" x 4.5" (11.43 x 11.43 cm) pad	A6213	10/bx, 10 bx/cs
	MSC1065EP	6.1" x 5.6" (15.49 x 14.22 cm), sacral	Pending	10/bx, 10 bx/cs

<b>Optifoam Non-Adhesive</b> Superb fluid handling with a variety of applications				
	Item Number	Description	HCPCS	Packaging
	MSC1244EP	4" x 4" (10.16 x 10.16 cm)	A6209	10/bx, 10 bx/cs
	MSC1266EP	6" x 6" (15.24 x 15.24 cm)	A6210	10/bx, 10 bx/cs

<b>Optifoam Thin</b> Extremely conformable, protective dressing				
	Item Number	Description	HCPCS	Packaging
	MSC1523	2" x 3" (5.08 x 7.62 cm)	Pending	10/bx, 20 bx/cs
	MSC1544	4" x 4" (10.16 x 10.16 cm)	Pending	10/bx, 10 bx/cs

<b>Optifoam Basic</b> For wounds or tubes				
	Item Number	Description	HCPCS	Packaging
	MSC1133	3" x 3" (7.62 x 7.62 cm)	A6209	10/bx, 10 bx/cs
	MSC133F	3" x 3" (7.62 x 7.62 cm) with fenestration	A6209	10/bx, 10 bx/cs
	MSC1145	4" x 5" (10.16 x 12.7 cm)	A6210	10/bx, 10 bx/cs

<b>Optifoam Site</b> Designed specifically for tube sites				
	Item Number	Description	HCPCS	Packaging
	MSC1104	4" (10.16 cm) round, 2" (5.08 cm) pad	A6212	30/bag, 4 bags/cs

For more information, visit [www.medline.com/woundcare](http://www.medline.com/woundcare) or contact your sales specialist.