



Instructions for Use

(Read carefully before using the device)

Descriptions

Every Milliken Ultra foam dressing utilizes Active Fluid Management® technology created by Milliken Healthcare Products. The three part design combines two layers of high performance fabric using a micro-knit process, creating a powerful moisture transfer mechanism that pulls away and traps excess exudate in the highly absorbent foam layer. The combination of these fabrics keeps the periwound skin dry, the wound bed moist, and harmful exudate away from a healing wound.

Directions for Use

Ultra dressings can be used throughout the healing process. Prepare the wound bed according to clinical protocol such as debridement and/or cleaning and rinsing with sterile saline or water. Remove the dressing from the package and apply to the wound with the printed side of the dressing away from the wound. Optimal performance of Ultra dressings requires direct contact with the wound. Ultra dressings should not be folded. If required, Ultra dressings can be cut to various shapes and sizes. Ultra dressings can be secured with an appropriate dressing such as roll gauze, adhesive border gauze, composite island dressing, transparent film or tape. Ultra dressings can be used under compression.

Directions for Removal

Ultra dressings should be changed when clinically indicated or as prescribed by a healthcare professional. Carefully remove the fixative and the dressing from the wound. Normally the dressing does not adhere to the wound. If adherence is observed, the dressing can be moistened with sterile saline or water to ease removal.

Indications

The Ultra dressings are indicated for low to highly exuding acute and chronic wounds such as partial thickness burns, incisions, skin grafts, donor sites, lacerations, abrasions, Stage II-IV pressure ulcers, venous ulcers, and diabetic ulcers.

Precautions and Observations

Ultra dressings are not approved for use directly on full thickness burns.

The Active Fluid Management® performance of Ultra dressings may be impaired by use with petrolatum based ointments.

Storage

Ultra dressings should be stored at temperatures less than 131°F. Keep dry

Sterilization

Ultra dressings are sterile as provided. Do not use Ultra dressings if the primary package is visibly damaged or broken. Do not resterilize Ultra dressings. For single use only.

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner (U.S.A. only).

Milliken Healthcare Products, LLC
920 Milliken Road
Spartanburg, SC 29303 USA
Tel: 866-491-6556
8 am – 5 pm E.S.T.

healthcare.milliken.com
medproducts@milliken.com

Made in U.S.A.
2013 Copyright Milliken Healthcare Products, LLC, All rights reserved. Active Fluid Management®, and AFM® are trademarks of Milliken Healthcare Products, LLC.
Document #3000006840PPDIGr7