SAVE THESE INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

⚠ WARNING

- **DO NOT** use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- You should **ALWAYS** consult with your physician or therapist to determine proper adjustment and usage.
- Items should not protrude from the walker basket - otherwise, injury or damage may occur.
- **ALWAYS** test to see that the walker and attachments are properly and securely locked in place before using.
- **DO NOT** carry or place weight exceeding five pounds in the walker basket when attached to walker. This may cause the walker to tip forward.
- **NEVER** use the walker tray, walker basket and the walker pouch at the same time.
- **NEVER** use the walker basket as a means of support.
- **DO NOT** sit or push down on the walker basket when in use. This may cause the basket to collapse.
- Use extreme care when using this walker basket on a walker with swivel casters. The walker may tip resulting in possible injury or damage.
- **DO NOT** use with heavy duty walkers.

⚠ ACCESSORIES WARNING

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

Installing/Removing the Walker Basket

NOTE: Walker basket model 6096 may be used on 6240-A, 6240-JR, 6240-5F and 6240-JR5F walkers.

NOTE: For this procedure, refer to FIGURE 1.

NOTE: To remove the walker basket, reverse this procedure.

NOTE: The installation procedures differ for Adult and Junior walkers.

For Adult Walker Models 6240-A and 6240-5F:

1. With the walker basket on the outside of the walker, put the top hook on the upper pivot brace.
2. Engage the lower S-hooks onto the lower pivot brace on the Adult walker. Refer to Detail "B".

NOTE: Make sure that the hooks are completely seated on the upper and lower pivot braces and the walker basket sits level.

3. Wrap the strap around the lower pivot brace and through the walker basket. Continue wrapping until the entire length of the strap is around the lower pivot brace. Apply slight pressure to the end of the strap to secure it in place.

NOTE: Make sure that the strap is wrapped completely around the lower pivot brace and that the end of the strap is secure before use.

For Junior Walker Models 6240-JR and 6240-JR5F:

1. With the walker basket on the outside of the walker, put the top hook on the upper pivot brace.
2. Engage the upper S-hooks onto the lower pivot brace on the Junior walker. Refer to Detail "B".


NOTE: Make sure that the hooks are completely seated on the upper and lower pivot braces and the walker basket sits level.

3. Unfasten and move the strap to the second wire from the top of the walker basket. Refer to Detail "C". Secure the strap around the wire. Wrap the strap around the lower pivot brace and through the walker basket. Continue wrapping until the entire length of the strap is around the lower pivot brace. Apply slight pressure to the end of the strap to secure it in place.

NOTE: Make sure that the strap is wrapped completely around the lower pivot brace and that the end of the strap is secure before use.

DETAIL "B" - BASKET DETAILS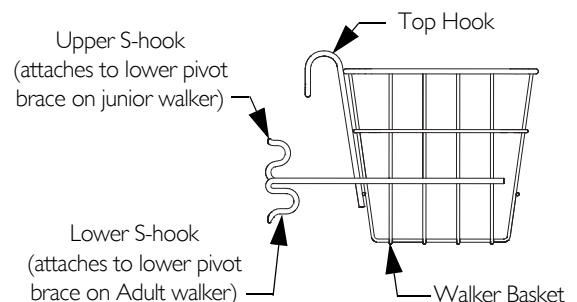

DETAIL "C" - STRAP LOCATION

FIGURE 1 Installing/Removing the Walker Basket

For more information regarding Invacare products, parts, and services, please visit www.invacare.com

Limited Warranty

Invacare warrants this product to be free from defects in materials and workmanship.

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants this product to be free from defects in materials and workmanship for one year for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will use a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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All rights reserved. Trademarks are identified by the symbols [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
© 2009 Invacare Corporation