


# CARE & USAGE INSTRUCTIONS for your PURPLE® SEAT CUSHION

The smooth, black cover included with your Purple® Seat Cushion will extend the life, look, and comfort of your cushion.

## HOW TO PUT YOUR SEAT CUSHION INTO THE COVER


STEP 1

Lay the cover flat, with the gripping side down. Face the zipper toward you. Unzip the cover.

  
UNZIP COVER →


STEP 2

Lay Purple cushion flat. If your cushion is **rounded**, the curved edges should face up. If your cushion is **square**, the smaller grid should face up. Align the cushion with the cover.

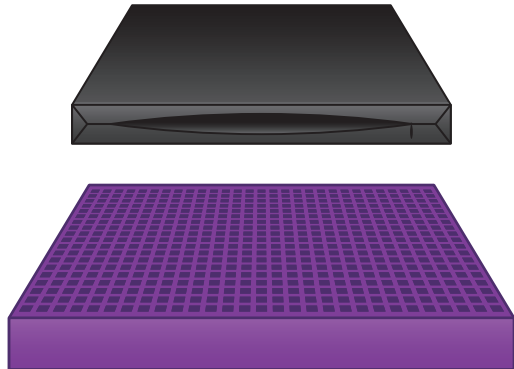


STEP 3

Fold the Purple in half, with the top folded in. Slide the cushion into the open cover.



STEP 4

Push the Purple all the way in and then align all four corners within the cover. Start with the back two, then the front two. The zipper should slide closed easily. Sit and let your bum enjoy!

  
← ZIP CLOSED

### Is it getting hot in here, or is it just your cushion?

Be careful leaving your Purple cushion in the car when it's hot. Like the Wicked Witch of the West, it can melt under the wrong circumstances.


If your seat cushion gets above 105° F, do not sit on it or move it. This could permanently damage your seat cushion! Wait until the Purple is below 105° F (up to 30 minutes) before moving or sitting on it.

Placing your seat cushion in the shade on the car floor or under the seat should keep it from getting too hot.

### Keeping Your Cushion Clean

The cover is washable. Make sure you take the Purple out of the cover before washing. Use a gentle warm or cold cycle with gentle detergent. Air dry or tumble dry without heat.

You can clean your Purple by hand using warm water and a gentle liquid soap (like dish soap). Air dry.

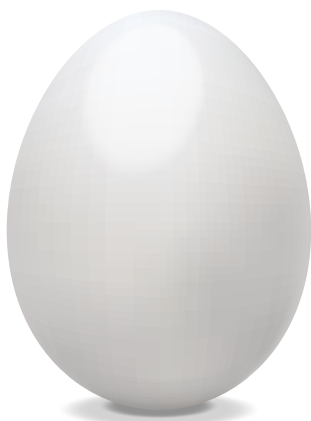

### Fancy yourself a scientist? Try the egg test at home!

Any seat cushion 2-inches tall or more can pass the raw egg test! Simply take the cover off (this is important!), put the egg on the cushion, and sit! You may want to put the egg in a plastic bag to keep any accidental messes from happening.

If your seat cushion is shorter than 2-inches tall, we don't recommend the egg test. But you can replace the egg with your keys to explore the comfort technology in Purple. It's still impressive and rarely messy.

**purple**

Learn about more Purple-tastic products for your booty and body at [OnPurple.com](https://OnPurple.com)