

Quick Start Guide



myAerX.com

1

Place the Aer X into the Aer X bag, ensure the filter inlet is not covered and air can flow freely

Figure B
POWER
BUTTON

Figure A
MAGNETIC
OXYGEN
PORT

Figure C
BATTERY
COVER

Figure D
CHARGING PORT
& AIR FILTER
COVER

2

Slide battery into battery compartment by opening cover located on the top of your Aer X.

The charging port is located just above the filter inlet. Your Aer X may be charged during use.



3

Insert the power cord into the Aer X if not using battery power or when charging battery.

CONTINUED ON NEXT PAGE

4

Connect cannula to the oxygen port. **NOTE:** the oxygen port connects magnetically, this is for your safety (figure A)



7

An extended beep indicates your device is powering on. Please wait until start up is complete before changing any settings.

5

Place the nasal cannula in a comfortable position, with the prongs inside your nostrils

8

Your Aer X will now be providing oxygen when you inhale

6

Press the blue Power button to begin oxygen flow (figure B)



3B Medical, Inc.

QUICK TROUBLE SHOOTING TIPS

If the 3B™ AER X should shut down during start up or, the Yellow light is illuminated after start up;

1. Check to make sure the 3B AER X cannula is attached. **USING A DIFFERENT CANNULA THAN THE CANNULA SHIPPED WITH YOUR 3B AER or HAVING NO CANNULA ATTACHED** may cause the 3B AER X to shut down during startup.
2. Make sure that the battery is charged. A **DEPLETED BATTERY** will cause the 3B AER X to shut down during startup.
3. Power down your 3B AER X and repeat the startup process.

If the 3B AER X fails to start again;

4. Disconnect the battery and connect the power supply.
5. Repeat the startup process.

If the 3B™ AER X continues to fail to start up, please contact your equipment provider for assistance.