## USER MANUAL

*Aromatherapy*  
**PERSONAL OXYGEN BAR**


Model # OXY03

# QUICK START GUIDE


## BEFORE USING THE OXYGEN BAR FOR THE FIRST TIME, FOLLOW THESE 3 STEPS:

### STEP 1: CUT OFF ALL SAFETY TIES FROM BOTTOM

- Using scissors, cut off the two(2) white safety ties from the bottom of the unit, making sure to remove all ties from the unit. Then discard the ties.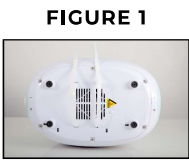

Find the two (2) white ties on bottom of machine

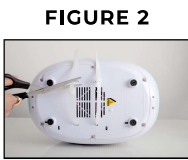Cut both white ties, remove all ties from machine, and discard


Air vents should be clear of obstruction

### STEP 2: PLACE THE UNIT ON A HARD, FLAT SURFACE AND ATTACH HEADSET

- Place the unit on a stable, hard and flat surface such as a tabletop, tile floor, or desk.
- Do not place the unit on a carpet, rug, towel, or other soft surface to avoid overheating and damage.
- Attach the airflow tube to the front of the unit by gently pushing the end of the tube over the oxygen output nozzle (see Figure 4)

### STEP 3: TURN ON BOTH THE POWER SUPPLY **AND** OXYGEN FLOW

- Plug the AC adapter into the back of the unit (see Figure 5), and then plug into a nearby electrical outlet.
- Turn on the power supply using the switch on the back of the unit.
- To turn on the flow of oxygen, press the "OXYGEN ON/OFF" button on the LCD screen display panel or on the remote control.
- Both the power supply of the unit, **AND** the "OXYGEN ON/OFF" setting must be turned on for the oxygen flow to start.

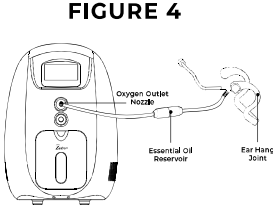**ZADRO®**  
*Aromatherapy*  
**PERSONAL OXYGEN BAR**  
Model Number : OXY03

## Important Safety Instructions

### READ ALL INSTRUCTIONS BEFORE USE

ALL OF THE SAFETY AND OPERATING INSTRUCTIONS SHOULD BE READ, ADHERED TO AND FOLLOWED BEFORE THE UNIT IS OPERATED.

THIS IS NOT A MEDICAL DEVICE , AND SHOULD NOT BE USED AS A REPLACEMENT FOR PRESCRIBED MEDICAL TREATMENTS.

### FOR HOUSEHOLD USE ONLY SAVE THESE INSTRUCTIONS

#### WHAT'S IN THE BOX

- Oxygen Bar (1)
- Remote Control (1)
- Headset (1)
- Nasal Tube Attachment (1)
- AC Adapter (1)
- Essential Oil (1)

#### SPECIFICATIONS

- Adjustable Oxygen Output Rate (at Sea Level):
  - 1L\* = 90% Oxygen Concentration
  - 2L\* = 75% Oxygen Concentration
  - 3L\* = 55% Oxygen Concentration
  - 4L\* = 45% Oxygen Concentration
  - 5L\* = 30% Oxygen Concentration
- Noise: < 45dB
- Rated Voltage: 120V
- Wattage Maximum: 130W
- Product Dimensions: 14.1" x 9.10" x 12.64"
- Weight: 14.33 lbs

#### SAFETY INFORMATION

-  **Caution** : Advice on how to get the best performance from the unit.
-  **Warning** : Important safety information for hazards that might cause serious injury or unit breakdown.
-  **Danger** : Urgent safety information for hazards that may cause serious injury or death.

## Caution

- THIS UNIT IS NOT FOR MEDICAL USE. It is designed for health maintenance.
- Do not block the ventilation port located on the bottom of unit.
- Do not set base on carpet or other soft surface. This could cause air flow restriction and damage to unit.
- For best performance, regularly clean the Intake Filter and panel located in the back of the unit.

## Warning

- DO NOT use the unit when there are any hazardous vapors or pesticide spray in the room.
- Do not leave the unit near heated areas. High heat will cause the plastic case to deform.
- During operation, do not press hard upon the external plastic tube as this will result in air flow restriction and damage to unit.
- Do not leave the unit unattended when turned on
- Do not use accessory attachments not recommended by Zadro as this may cause injury
- Do not alter the polarized plug in any way

## Caution

Check the electrical rating on the unit label and wall outlet before connecting the power cord.

When the power cord is found damaged or broken, return it to Zadro Products for examination or repair.

Infant, child, or disabled people should use under the watch of other adult to avoid accidental injury.

Do not reach for the unit in the event the unit falls into water. Unplug the unit immediately, then return it to Zadro Products for examination or repair.

Please remove the plug from electric outlet when not in use.

Keep the unit and all electrical cords away from heated surfaces or appliances. The oxygen will make the material flammable.

Do not use while bathing or with wet hands.

Do not to smoke or have open flames near the unit.

## INTRODUCTION

Restore your sense of calm, relaxation and overall well-being with Zadro's Aromatherapy Personal Oxygen Bar. It's quiet, compact, portable, and modeled after oxygen bars found in luxury resorts and spas that help increase self-care and overall well-being. Zadro's oxygen bar produces oxygen-enriched air through a quadruple air filtration system and French-imported molecular membrane that increases oxygen concentration up to 90% (compared to 21% oxygen concentration of normal air\*). Adjustable oxygen concentration from 30% oxygen up to 90% oxygen lets you control your relaxation experience. Choose to add essential oils to the oxygen for instant spa-like aromatherapy at home (lavender essential oil included)\*\*. Easily change the settings during your session with the convenient remote control. Enjoy peace of mind with 8 timer settings from 15 minutes up to 2 hours with auto-shutoff. Then enhance the purification of the oxygen with the optional negative ions booster function\*\*\*. If you're in a dry climate, use the optional water reservoir to infuse the purified oxygen with humidity. Relax with the oxygen bar while reading, at your desk, watching TV, in bed, or enjoying your hobbies. Simply wear the included headset or nose tube attachment for a gentle stream of oxygen.

## FEATURES

1. Experience luxurious spa-like aromatherapy during your session using essential oil of your choice or with the included lavender essential oil\*\*
2. 5 adjustable oxygen concentration settings from 30% oxygen up to 90% oxygen
3. Quadruple air filtration system and French-imported molecular membrane produces highly purified oxygen of up to 90% concentration
4. Auto shutoff timer with 8 timer settings from 15 minutes up to 2 hours
5. Personalize your experience using the digital LCD screen display panel or easily change the settings during your session with the convenient remote control (CR2025 battery included)
6. Enhance the oxygen purification with the optional negative ions booster function\*\*\*
7. If you're in a dry climate, use the optional water reservoir which infuses the purified oxygen with humidity
8. Enjoy less noise (<45 dB) from the unit for a peaceful experience
9. Internal overheat protection will automatically shut off the unit for safety
10. Plugs into a standard electrical outlet

\*\*Average 21% oxygen concentration of ambient air at sea level but can vary depending on location


\*\*Consult with a doctor or health care provider first before using any essential oils while pregnant or nursing

\*\*\*Consult with a doctor or health care provider first before incorporating negative ions into your oxygen session

## CONTROL PANEL DIAGRAM

## EXTERIOR DIAGRAM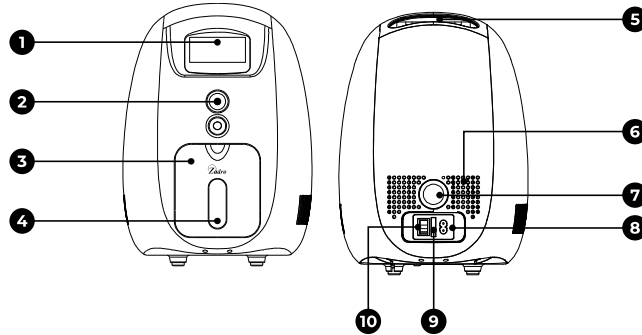


- |                         |                         |
|-------------------------|-------------------------|
| 1. LCD Screen Display   | 6. Heat Vents           |
| 2. Oxygen Outlet Nozzle | 7. Air Filter           |
| 3. Water Tank Panel     | 8. Power Jack           |
| 4. Water Reservoir      | 9. Fuse                 |
| 5. Top Handle           | 10. Power Supply Switch |


## 2 WAYS TO WEAR

- Choose either the headset or nasal tube and attach it to the end of the air flow tube.
- Place the headset or nasal tube attachment on your head and adjust to a comfortable position.
- If using the Headset: place it over your ears and adjust the flexible arm to be directly in front of your nose.
- If using the Nasal Tube Attachment: position the nasal tube with the nasal prongs facing upward and curved toward the face. Then insert the two nasal prongs into your nostrils and position until it's comfortable.

RECOMMENDED POSITIONING  
OF HEADSET

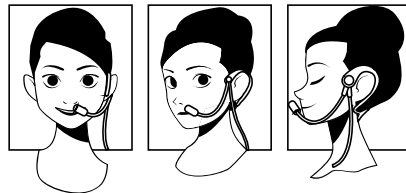RECOMMENDED POSITIONING  
OF NASAL TUBE
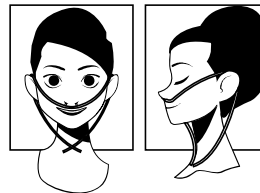
## ADJUSTABLE OXYGEN CONCENTRATION

1. Choose from 5 adjustable oxygen concentrations (based at sea level):
  - a. 1L\* = 90% Oxygen (highest concentration)
  - b. 2L\* = 75% Oxygen
  - c. 3L\* = 55% Oxygen
  - d. 4L\* = 45% Oxygen
  - e. 5L\* = 30% Oxygen (lowest concentration)
2. If this is your first time using an oxygen bar, we recommend starting at the lowest concentration (5L setting) then increasing the concentration over time.


\*L = Liter of air per minute

## TIMER SETTINGS

1. When the OXYGEN ON/OFF is turned on, press the up and down TIMER buttons on the LCD screen display panel on the front of the unit, or on the remote control.
2. Choose between the following 8 timer settings (the unit will auto shut off when the timer setting is finished):
  - o 15 minutes
  - o 30 minutes
  - o 45 minutes
  - o 1 hour
  - o 1 hour 15 minutes
  - o 1 hour 30 minutes
  - o 1 hour 45 minutes
  - o 2 hour

## AROMATHERAPY

- Use the aromatherapy reservoir to infuse the oxygen with your choice of essential oils (optional).
- Unscrew the reservoir located in the middle of the air tube by twisting until it completely detaches, add your desired amount of essential oil (2-4 drops are recommended), then screw the reservoir back together to lock into place.
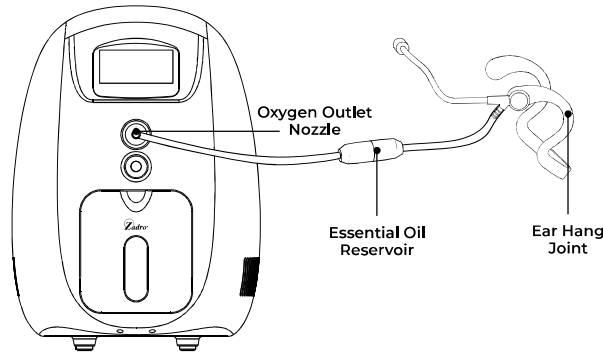


## BOOST A SETTING

1. Enhance the oxygen's purification with the Boost A setting (optional) which emits negative ions to further filter the air.
2. After the oxygen airflow is turned on, press the "BOOST A" button on the LCD screen display panel on the front of the unit, or on the remote control.

## HUMIDITY

1. If you're in a dry climate, you can increase the humidity within the oxygen flow by utilizing the humidity feature (optional).
2. Before turning on the unit, pull out the humidifier door and slide out the water tank cup. Then fill the cup with water up to the "MAX" fill line.
3. Turn on the power supply switch in the back of the unit, then press the "OXYGEN ON/OFF" button on the LCD screen display panel on the front of the unit, or on the remote control.
4. The water tank will automatically start when the OXYGEN ON/OFF button is pushed to turn on the airflow.
5. To refill the water tank, turn off the oxygen flow and power supply first before sliding out the water cup.

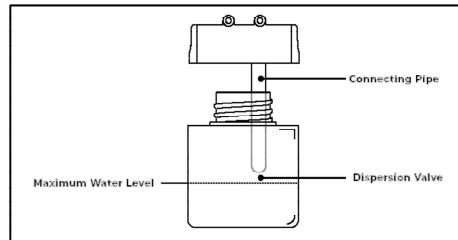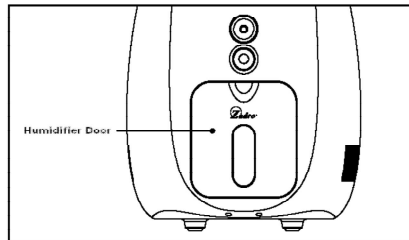## MAINTENANCE

**DANGER**

TURN OFF THE UNIT AND REMOVE THE POWER CORD FROM THE ELECTRICAL OUTLET BEFORE MAINTENANCE TO AVOID ELECTRICAL SHOCK.

### CLEANING THE HEADSET AND TUBES

1. Water condensation may form in the air flow tube and essential oil reservoir if using the oxygen bar in high humidity locations. Clean both pieces before storing the unit for long periods of time.
2. Rinse the airflow tube and essential oil reservoir with warm water and allow to air dry before storage or re-attaching the parts to the unit.

### CLEANING THE UNIT

1. Unplug the unit from electrical outlet. Wipe the unit with a soft clean cloth that's slightly damp with water, then wipe dry with a dry cloth. Do not use paper towels (fibers can get caught in the unit's vents).
2. Do not use washing detergents, benzene, thinner or alcohol.

### CLEANING THE WATER RESERVOIR

1. Open the water tank panel and slide out the water cup from the unit.
2. Rinse the cup with warm water and dry with a cloth before attaching back to the unit. Do not use washing detergents, benzene, thinner or alcohol.

### CLEANING THE ESSENTIAL OIL RESERVOIR

1. Twist to unlock the essential oil reservoir.
2. Rinse each piece of the reservoir with warm water. Do not use washing detergents, benzene, thinner or alcohol.

## TROUBLESHOOTING

### 1. The unit doesn't turn on.

- Check that the AC adapter is plugged into the back of the unit, and that it's also securely plugged into a nearby electrical outlet.
- Check that the AC adapter cord is undamaged.
- Check that your wall outlet is live and undamaged.
- Check if the fuse in the back of the unit is black or the wire is broken. If so, replace the fuse with a standard 5 amp fuse from a local hardware supply store.

### 2. The unit's motor is running, but no oxygen is emitted.

- Check that the OXYGEN ON/OFF button has been pressed.
- Check that all tube connections are secure.
- Check for damage or pinching of the hose, headset and nasal tube.

### 3. THERE IS LIMITED OXYGEN FLOW.

- Check that all tube connections are secure.
- Check for damage or pinching to the hose, headset and nasal tube.

### 4. THE AROMATHERAPY DOESN'T WORK.

- Check that the aromatherapy reservoir is screwed tightly, there are no cracks, and that it's not leaking.
- Check that there is enough essential oil inside the reservoir.
- Check that the essential oil of choice is of high enough concentration.

## **1 YEAR LIMITED WARRANTY**

Zadro products include a 1 Year limited warranty from date of purchase against defects in materials and workmanship. This warranty does not include damage to the product as a result of misuse or accident.

Should your product become defective during the warranty period, Zadro, Inc.<sup>™</sup> will repair or replace (at our sole discretion) your product, at no charge. However, you are responsible for properly packing your product and any applicable shipping charges to Zadro, Inc.<sup>™</sup>

Please contact our Customer Service at 800-468-4348, 714-892-9200 or email [customersupport@zadroinc.com](mailto:customersupport@zadroinc.com) on how to obtain an RMA (return authorization number). Please have available and/or include a copy of your receipt, model number (OXY03) and the issue you are experiencing with your product.

CUSTOMERS OUTSIDE THE UNITED STATES OF AMERICA, PLEASE CONTACT THE RETAILER OF PURCHASE DIRECTLY TO RESOLVE ANY PRODUCT WARRANTY ISSUES.

Any product returned for preference is subject to a 20% restocking fee of the product and no shipping charges will be refunded.

Zadro, Inc.<sup>™</sup> shall not be liable for any incidental or consequential damages arising out of defects in this product, whether or not caused by its negligence.

Some states do not allow the exclusion or limitation of incidental or consequential damages or allow limitations on how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights and you may also have other rights, which vary from state to state.

---

Thank you for your purchase! Please visit us online to leave comments, reviews, & see more exciting new products from [www.ZadroInc.com](http://www.ZadroInc.com)

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© Zadro, Inc.™