
Contour®

BLOOD GLUCOSE MONITORING SYSTE



QUICK REFERENCE GUIDE

Your CONTOUR® Meter:

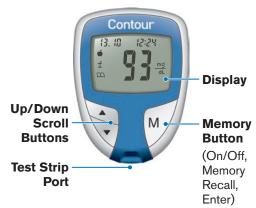

Your CONTOUR® Test Strip:

pulled in here.



Grey End: Insert this end into the test

strip port.

Basic Testing

See your CONTOUR® meter user guide and lancing device insert for complete instructions. CAUTION: The meter and lancing device are intended for use by a single patient.


Getting Ready to Test:

WARNING


Always wash your hands well with soap and water and dry them well before and after testing,

CAUTION: The lancing device provided with your kit is intended for self testing by a single patient. It must not be used on more than one person due to the risk of infection.

- 1. Remove the endcap from the lancing device.
- 2. Loosen the round protective cap on a lancet by rotating it 1/4 turn, but do not remove it.

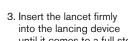

until it comes to a full stop.

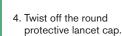






5. Replace the endcap.



- 6. Adjust the endcap dial to your preferred puncture depth setting.
- 7. Insert the arev end of the test strip into the test strip port on the meter.

No coding is required.

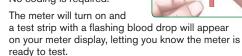

Getting the Blood Drop:

. Press the lancing device firmly against the puncture site and press the release



2. Stroke your hand and finger towards the puncture site to form a drop of blood.

Testing Your Blood:

I. Touch the tip of the test strip to the drop of blood. The blood is pulled into the test strip.

Hold it in the blood drop until the meter beeps.



Your result* is displayed after a 5-second countdown.

To turn your meter off, remove

the test strip. *Your meter is preset and locked to display results in mg/dL. If your result is in mmol/L, contact Customer Service at 1-800-348-8100, Available Monday through Sunday, 8:00 am



Ejecting and Disposing of the Used Lancet:

1. Do not use your fingers to remove the lancet from the lancing device. The lancing device provided with your kit has an automatic lancet ejection feature.



2. Refer to the separate lancing device insert, if provided with your kit, for instructions on automatic ejection of the lancet.

WARNING

 Dispose of the used lancet as medical waste or as advised by your health care professional.

Need Help? Call: 1-800-348-8100 Available Monday through Sunday, 8:00 am through 12:00 Midnight, Eastern Time. www.diabetes.ascensia.com



Using Advanced Mode

See your user guide for complete instructions.

Setting Your Meter to Advanced Mode

To set your meter to Advanced Mode, and take advantage of meal markers and other available features:

- 1. Press **M** to turn your meter on.
- 2. Press and hold ▲ or ▼ for 3 seconds.
- 3. L-1 will flash.
- 4. Press ▲ or ▼ to change to L-2.
- 5. Press M to set.



Find out how these additional features can help you manage your diabetes. Go to www.diabetes.ascensia.com or call Customer Service:

1-800-348-8100

Available Monday through Sunday, 8:00 am through 12:00 Midnight, Eastern Time.

Using the Markers

See your user guide for complete instructions.



Pre-meal =

Marks a test result that was taken before a meal



Post-meal =

Marks a test result that was taken after a




Logbook = Marks any result as being unique or different in some way.



Alarm = Sets an alarm to remind you to run a post-meal test.



Contour

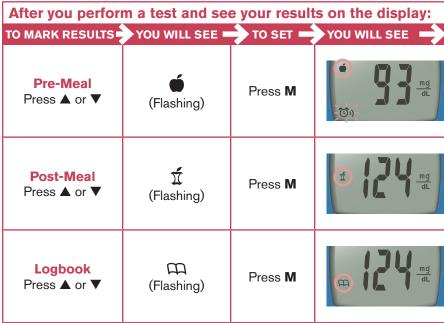

WHAT IT MEANS **USING THE ALARM**

Result is marked

as a pre-meal test.

Press **M** again to

Result is marked as

Result is marked in

memory as unique.

this in your Clinilog®

You should note

logbook.

a post-meal test.

set an alarm.**

When the alarm sounds, the meter turns on, and you will see your pre-meal test result.



Press any button once to silence alarm and twice to turn the meter off.

Run another blood test and mark it with a $\sqrt{7}$ post-meal marker (see chart at left).

E11 Abnormal result. Retest using a new test strip.

Distributed by: Ascensia Diabetes Care US, Inc. 5 Wood Hollow Road Parsippany, NJ 07054 www.diabetes.ascensia.com

For information on patents, and related licenses, see: www.patents.ascensia.com


© 2016 Ascensia Diabetes Care Holdings AG. All rights reserved.

Ascensia, the Ascensia Diabetes Care logo. Clinilog, Contour, Glucofacts, and the No Coding

logo are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

**Alarm is factory set at 2 hours. See your user guide for complete instructions on how to adjust alarm times.

Error Codes and Symbols See your user guide for a complete list of

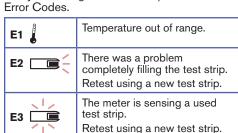



85556347 Rev 10/16