Cooling Sensation Seat Cushion
Cojín para asiento con sensación
refrescante
Coussin de siège sensation
rafraîchissante
RTL2017CTS

Drive DeVilbiss Healthcare
99 Seaview Blvd.
Port Washington, NY 11050
1-877-224-0946
www.drivemedical.com

Cooling Sensation Seat Cushion

COMFORT TOUCH™ Cushions by Drive™ are designed to help satisfy your therapeutic needs while immersing you in comfort with an ergonomic design, high-quality foam and luxurious look and feel.

Beneficial Uses

The Comfort Touch Cooling Sensation Seat Cushion combines high density foam with cooling properties to provide superior comfort across multiple indoor and outdoor settings. This versatile cushion can be used on any seating surface including home or office chairs, wheelchairs, benches, stadium seating, and more. The carry handle allows for convenient portability.

How to Use

Place Comfort Touch Cooling Sensation Seat Cushion on seating surface. The gel top layer on the foam will provide a cooling sensation until the body temperature naturally alters and reduces the cooling properties. The cooling properties will return with the next use.

CAUTION & WARNINGS

1. The gel top layer on the foam should last a long time if properly cared for. In the event the gel topper is punctured or torn, the cushion should be discarded.
2. If there is exposure to the gel contents, take the following action:
 - a. If gel contents come in contact with the eyes, flush with water.
 - b. If gel contents come in contact with the skin, wash off with soap and water.
 - c. If gel contents are inhaled, move to an area with fresh air ventilation.
 - d. The Gel substance is a Non-Toxic material, however if contents of the Gel is swallowed, give one or two glasses of water and induce vomiting and consult a physician.
3. Do not freeze or heat this seat cushion.

Care and Cleaning Instructions

Cover: Remove the poly-spandex cover by unzipping and carefully removing from the cushion. Wash it on a cold gentle cycle with mild soap as needed. Do not bleach. Tumble dry on low heat or allow to air dry.

Foam and Gel Top Layer: If the high density foam or gel top has any odor or stains, surface clean it with a moist cloth and air it out until it's completely dry. No industrial solvents should be used at any time. Any lingering odor will dissipate over time. Never put foam with gel layer in washer or dryer.

**Note- foam is a porous material. Small holes seen on the foam itself are not a defect in the product but rather the texture and composition of foam.*

Weight Capacity:

300 lbs. (136 kg)

Size:

18 x 16 x 2 pulgadas
45.7cm x 40.6cm x 5.1cm

One Year Limited Warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for 1 year from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited 1 Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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