# Installation and Operating Instructions

## ***Model FA3-MC***

### *Fire Alarm with Dry Contact Input 418MHz RF Transmitter*

#### **Installation:**

Please contact a licensed fire alarm installer for installation and working with this product. Improper installation can damage your fire alarm and/or prevent proper operation of the fire alarm. Technical support is available at (800) 572-5227.

The FA3-MC in conjunction with a Silent Call Medallion Series receiver is considered an *auxiliary alerting system*.

The FA3-MC is designed to interface with a dry contact output. It will transmit as long as the contact is closed and will stop transmitting when the contact opens.

1. Using the supplied Velcro strip, attach the Fire transmitter to the wall next to the alarm panel. You can also hang the transmitter on a nail or screw using the opening in the back of the case. Do not attach inside of the alarm panel box! The metal of the box will detune the transmitter.

#### **Operation:**

1. The FA3-MC will transmit as long as the contacts are closed. When the contacts are closed, the Red LED on the face of the transmitter will light up. The transmitter will then send a signal to any Silent Call Medallion Series Receiver activating the receiver.
2. Transmission range with a Silent Call Medallion Series receiver is about 500 feet.
3. This unit is powered by two AA size alkaline batteries (included). Actual battery life is determined by the amount of time the transmitter is transmitting. In normal use the batteries may be expected to last several years.
4. There is a Yellow LED (low battery indicator light) on the face of the transmitter to let you know when the battery is getting low and will need to be changed soon.

#### **Transmitter Settings:**

The Silent Call Medallion system is digitally encoded. All Silent Call Medallion Series receivers are capable of receiving one million different codes and the transmitters are preset at the factory to send one of the million different codes. Please refer to your receiver manual for instruction on teaching the transmitter code to the receiver.

#### **Replacement Batteries:**

Use two AA 1.5V Alkaline Batteries. Use Only Panasonic Alkaline Batteries Type LR6XWA. Batteries may be purchased Online at [www.silentcall.com](http://www.silentcall.com).

#### **REGULATORY INFORMATION NOTICE:**

Unauthorized changes or modifications could void the user's authority to operate the equipment.

This device complies with Industry Canada license-exempt RSS standard(s).

Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation of the device. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and the receiver
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/television technician for help

## Installation and Operating Instructions

# **Model FA3-MC**

*Fire Alarm with Dry Contact Input 418MHz RF Transmitter*

### **Technical Support**

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at [support@silentcall.com](mailto:support@silentcall.com)

### **Limited Warranty**

Your transmitter is warranted to be free of defects in material and workmanship for five years from the date of initial purchase. During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. This warranty is void if the defect is caused by customer abuse or neglect.

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