


IV3000[®] STANDARD

Moisture responsive catheter dressing

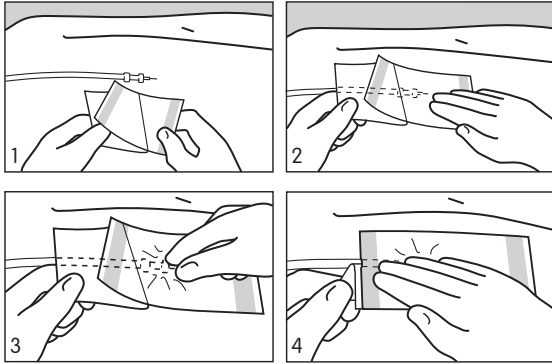

STERILE **EO** Single use. Ethylene oxide sterilised. Do not use if pack is open or damaged. Store in a dry place (<25°C/77°F).

Description

IV3000 Standard is a dressing based on Smith & Nephew's REACTIC[®] moisture responsive transparent film, specifically designed to meet the needs of catheter fixation.

Mode of Action

The REACTIC film in IV3000

Standard has a unique molecular structure which is significantly more permeable to water vapour than ordinary film. The film is not porous and it therefore helps prevent bacterial contamination to the IV site and is waterproof. These properties result in drier catheter sites, reduced skin maceration and lower skin colonisation, thus reducing the risk of catheter related infection. Further, it can be applied easily and aseptically, and due to the pattern spread adhesive, pain on removal is reduced. IV3000 Standard is spread with a non-irritating non-sensitising adhesive. Due to these properties IV3000 Standard offers overall advantages over ordinary films, tapes or gauze, and is believed to be a 'technological advance for IV site care'.

Instructions for use

Clean and thoroughly dry application area.

1. Remove approximately half of the protector paper and centre the dressing over the insertion site.
2. Smooth the dressing down onto the patient's skin working from the catheter outwards and remove the remaining protector paper. Be careful not to apply the dressing under tension.
3. The protector paper may be removed from the orange tape handles and the entire handle adhered to the patient's skin.

4. Alternatively, a portion of the tape handles can be torn off with the protector paper still in place. This can then be removed and the tape handles used to help secure the catheter hub or tubing. They may also be used for documentation.

5. If using more than one dressing, do not allow the dressings to overlap.
6. Remove dressing by loosening the orange handles and stretching the dressing parallel to the patient's skin while stabilising the patient's skin and catheter with the other hand. Alternatively, use an alcohol wipe to break the adhesive seal. If the skin is particularly fragile then warm soap and water may be used to break the adhesive bond and to ease the dressing off.

Indications

The above mentioned products are suitable for dressing peripheral, central line and epidural catheters directly onto the skin.

Contraindications

IV3000 Standard should be used on IV sites only. If a film dressing is required for open wounds such as pressure sores, burns or donor sites, OPSITE[®] FLEXIGRID[®] should be used.

Precautions

Use on prepared dry skin only. Do not stack dressings, or allow dressings to overlap. Periodically monitor the dressing and catheter site to confirm secure attachment and continued proper infusion, especially after bathing, showering, or if the dressing and catheter site becomes wet. If the dressing comes off, evaluate to ensure proper catheter placement, and then apply a new dressing. As with all adhesive products, apply and remove carefully from sensitive or fragile skin.

MANUFACTURER:

Bard Australia Pty Ltd

ABN 50 001 468 935

22 Lambs Road, Artarmon,

NSW 2064 Australia

Ph +61 2 8875 4000

Fax +61 2 8875 4050

UK Smith & Nephew Healthcare Limited, Healthcare House, 101 Hessel Road, Hull HU3 2BN. Tel: 01482 222200.

AUSTRALIA Smith & Nephew Pty Ltd., 315 Ferntree Gully Road, Mount Waverley, PO Box 242, Victoria 3149. Tel: (03) 8540 6777.

AUSTRIA Smith & Nephew GmbH, Concorde Business Park C3, A-2320 Schwechat. Tel: +43 (0)1 707 91 02.

BELGIUM Smith & Nephew SA-NV, Av. du Four à Briques 3b - 1140 Brussels - Karelevoenlaan 3b - 1140 Brussels. Tel: (02) 702 29 11.

CANADA Smith & Nephew Inc., 2250 Alfred Nobel, St. Laurent (Québec) H4S 2C9. Tel: 1 514 956 1010.

CHINA Smith & Nephew Limited, 19F, Harbour Ring Hiang Pu Centre, No. 98 Liu He Road, Shanghai 200001. Tel: (21) 6350 3100.

DENMARK Smith & Nephew A/S, Slotsmarken 14, 2970 Hørsholm, Tel: +45 45 80 61 00.

EASTERN EUROPE Smith & Nephew GmbH, Concorde Business Park C3, A-2320 Schwechat.

FRANCE Smith & Nephew SAS, 25 bd Alexandre Oyon 72019 LE MANS Cedex 2. Tel: 02 43 83 23 23.

FINLAND Smith & Nephew Oy, Äyrittie 12 C, 01510 Vantaa, (0) 10 218 7300.

GERMANY Smith & Nephew GmbH, Wound Management, Osterbrooksweg 71, D-22869 Schenefeld. Tel: 040 879744-0.

GREECE Costas A. Papaellinas (Hellas) SA, 26th km, Paeonias - Markopoulou Ave, PO Box 182, 19400 Koropi. Tel: 662 6201-7.

HONG KONG Smith & Nephew Ltd., Unit 813-816, 8/F, Delta House, 3 On Yiu Street, Shatin, N.T. Tel: (852) 2648 7700.

INDIA Smith & Nephew Healthcare Limited, 361 Solitaire Corporate Park, Andheri-Kurla Road, Andheri (East), Mumbai 400 093. Tel: (22) 2826 6406.

INDONESIA PT Smith & Nephew Healthcare, Total Building, 5th Floor, Jl. Let. Jend. S. Parman Kav106A, Jakarta 11440. Tel: 21 5680521.

IRELAND Smith & Nephew Limited, Carraig Court, George's Avenue, Blackrock, Co. Dublin, Tel: 01-217 0444.

ITALY Smith & Nephew S.r.l., - Via De Capitani 2A, 20041 Agrate Brianza (Milano). Tel: +39 039 60941.

JAPAN Smith & Nephew Wound Management KK, 2-4-1 Shiba Park, Minato-ku, Tokyo 105-0011 Japan. Tel: 03-5403-8930.

KOREA Smith & Nephew Ltd, 6th Floor, Kwangmyung Building, 237-2 Neung-dong, Kwangjin-ku, Seoul, Korea (143-180). Tel: 2343 60460.

MALAYSIA Smith & Nephew Healthcare Sdn.Bhd, Menara Merais, 9th Floor, 1, Jalan 9/13, Petaling Jaya 46300, Selangor. Tel: 03-7958 7103.

MALTA V J Salamone Ltd., PO Box 55, Valletta.

MEXICO Smith & Nephew, SA de CV, Patricio Sanz No. 442, Colonia del Valle, C.P. 03100, México D.F., Tel: 53 40 22 60.

MIDDLE EAST Smith & Nephew FZE, Level 7 Convention Tower, Dubai International Exhibition Complex, P.O. Box 9715, Dubai, United Arab Emirates. Tel: 971 4 3116 300.

NETHERLANDS Smith & Nephew BV, Kruisweg 659, 2132 NG Hoofddorp, Tel: (0031) 020 - 6543999.

NEW ZEALAND Smith & Nephew Ltd, PO Box 442, 621 Rosebank Road, Avondale, Auckland, Tel: 9 828 4059.

NORWAY Smith & Nephew A/S, PO Box 224, Nye Vakas vei 64, 1379 Nesbru, Tel: 66842020.

PAKISTAN Smith & Nephew Pakistan (Private) Ltd, A/69 SITE, Manghopir Road, SITE, PO Box 3659, Karachi 75700. Tel: 21 2561176.

PORTUGAL Smith & Nephew, Lda, Lagoas Park, Edificio 7, 2740-299 Porto Salvo. Tel: 21-446 06 50.

SINGAPORE Smith & Nephew Pte Ltd, 1 Jalan Kilang Timor, #08-03/05 Pacific Tech Centre, 159303. Tel: 270 0552.

SOUTH AFRICA Smith & Nephew Pharmaceuticals (Pty) Limited, 30 The Boulevard, West End Office Park, Westville, 3629. Tel: 031 710811.

SPAIN Smith & Nephew SAU, Fructuós Gelabert, 2-4, 08970 Sant Joan Despí, Barcelona. Tel: 93 373 73 01.

SWEDEN Smith & Nephew AB, Box 143, Kråketorsgatan 20, 43122 Mölndal. Tel: 031-746 58 00.

SWITZERLAND Smith & Nephew AG, Glutz Blotzheim-Str.1, CH-4502 Solothurn, Tel: 32 624-5660.

TAIWAN (R.O.C.) Smith & Nephew (Overseas) Ltd, Taiwan Branch, 9F-4, No.6, Lane 28, Long-an Rd., Xizhi City, Taipei County 221. Tel: 886 2 2649-2206.

THAILAND Smith & Nephew LTD, 344/3 Soi Rongrien Yupun, Rama IX Road, Kwang Bangapi, Khet Huay Kwang, Bangkok 10310. Tel: 2 719 6221-27.

TURKEY Interfarma KS, Mithatpasa C, No 24/7, Yenisehir, Ankara.

UNITED STATES Smith & Nephew, Inc., Wound Management Division, 970 Lake Carillon Drive, Suite 110, St. Petersburg, FL 33716. **Customer Care Center:** 1 800 876-1261 or 727 392-1261.

VENEZUELA Eurociencia C.A., Laboratorios Ball C.A., Marcas Asociadas (Marcasa) SA, Calle Santa Ana, Edificio Centro Penafiel, Piso 3, Urb. Boleíta Sur, Caracas. Tel: 2 234 8802.