

# Tranquility® Product Family

Incontinence Catalog and Care Guide



  
**TRANQUILITY®**  
PREMIUM PROTECTION

**SELECT**  
Incontinence Products

**ComfortCare™**  
Incontinence Products

**swimmates®**  
Disposable Swimwear

**PBE** Principle  
Business  
Enterprises, Inc.

# Tranquility® Product Family Incontinence Catalog and Care Guide

## About this Guide

The purpose of this guide is to provide help for navigating the best path of care, for individuals or care providers, dealing with incontinence that can challenge normal, productive, fulfilled living and/or pose healthcare risks. Selecting just the right product for each person is vital.

This guide will answer most of your questions and provide you with a peace of mind about incontinence. If you still have questions, please contact our experienced staff at 1-800-467-3224.

## Contents

Company Introduction and Mission.....	2
Product Positioning.....	3
Understanding Absorbency.....	4
The Tranquility Advantage .....	5
Peach Mat Guarantee.....	5
Tranquility® Product Family Listing	
Tranquility Incontinence Products.....	6
Select Incontinence Products .....	14
ComfortCare Incontinence Products.....	17
Swimmates Disposable Swimwear.....	18
Useful Reference Materials	
Moisture Management Standards .....	19
Frequently Asked Questions .....	19
Product Selection Chart .....	20
Sizing Guide .....	21
Quick Order Reference Guide.....	22



## Welcome to PBE

Principle Business Enterprises, (PBE) is a woman-owned, three generation, family enterprise celebrating over 55 years of service to the healthcare field and over 30 years as a pioneer in the development of incontinence and moisture management solutions. Founded by Lee and Jim Mitchell in 1961, the company's name and corporate charter clearly speak to its ethical foundation and purpose:

"to operate its business affairs in such a manner as will accord with that fixed principle which will destine it to do nothing but that which is honorable and will promote a high standard of business ethics in its own affairs and may tend to raise and leaven the standard of business ethics of the community."

PBE is the acknowledged innovative leader in high performance niche products especially in the area of incontinence care and fall prevention. Its products have been designed to address risk factors and care protocols that can materially affect the quality of life for individuals in hospitals, nursing homes, clinics and homecare settings while providing exceptional customer service.

## Our Mission

"To be a principle-centered business that **uplifts,** **enlightens,** and **enriches** the lives of those it serves so that they can live more abundantly."



# Tranquility<sup>®</sup> Product Family

## Product Positioning

**TRANQUILITY<sup>®</sup>**  
PREMIUM PROTECTION

**When Performance Matters Most.**  
Maximum Protection for greatest needs.  
The highest performing incontinence products  
available anywhere.

**SELECT<sup>®</sup>**  
Incontinence Products

Heavy Protection, Excellent Value.  
High quality products with proven performance.

**ComfortCare<sup>™</sup>**  
Incontinence Products

Moderate Protection, Quality Design.  
"Best in Class" of mass retail products.

**swimmates<sup>®</sup>**  
Disposable Swimwear

Providing adults with affordable, discreet disposable  
swimwear designed for bowel containment.





# Understanding Absorbency

Capacity claims can be confusing. Some brands show “drops of fluid” or use terms, such as; “super,” “extra,” or “supreme” to indicate product capacity. Two measurable indicators are C.U.P. and ISO. The Tranquility Product Family uses C.U.P. and displays actual product fluid capacity in standard 8 ounce measuring cups.

**ISO (International Organization for Standardization)** is a test method that measures capacity that only records the amount of fluid a product can absorb without any pressure applied (free-swell). The product is placed into saline solution for 30 minutes. After removal from the saline, it is allowed to drip for 5 minutes. The level of fluid that is retained is the ISO capacity.

**C.U.P. (Capacity Under Pressure) is the recommended way PBE tests because it gives a more “real world” indication of what the product will hold in actual use.**


PBE tests all absorbent products using the ISO method and also tests using the C.U.P. which takes it a step further. After the product has been soaked in synthetic urine, pressure is applied to simulate the kind of pressure a wearer would exert on the saturated product. The amount of fluid remaining after pressure has been applied is defined as the C.U.P. capacity.

The reality is that wearers of incontinence products are sitting, standing, walking, or lying down – all of which exerts pressure on the product. Experienced caregivers are acutely aware that a product’s RETENTION is the key factor in incontinence management.

Easy to understand graphics on packaging help in selecting the right product and capacity to meet the challenge.



Experienced caregivers are acutely aware that a product’s **RETENTION** is the key factor in incontinence management.

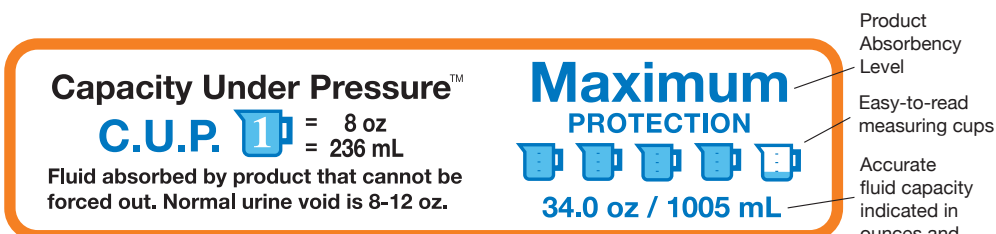

*Absorbent capacity indicator as shown on packaging.*



“The fact that Tranquility actually states the number of ounces it absorbs says it all. Other companies that advertise ‘maximum’ or ‘ultimate’ don’t compare ... I have not seen any other brand diaper that can boast a 34 oz. capacity for their overnight underwear! Dry, plus cost effective – so glad we tried them!”

– D.K., Caregiver, Whippany, NJ



# The Tranquility® Advantage

## Quality of Life

- Effective core design eliminates the feeling of wetness, coldness, and discomfort.
- Confidence of no leakage and no odor ends isolation, and restores normal socialization.
- Uninterrupted sleep patterns contribute to healing and well-being.
- Ending leakage means less laundry and less frustration for caregivers.
- Caregivers experience less fatigue especially with complex cases: dementia, mobility limitation, bowel incontinence, and obesity.

## Clinical Benefits

- Tranquility superabsorbent technology protects fragile skin from irritation and rashes.
- By inhibiting bacterial growth, urinary tract infections, and skin breakdown can be reduced.
- Injury from incontinence related slips and falls can be lowered.
- By decreasing the number of changes needed, the risk of strain/injury to caregivers is reduced.

## Economic Benefits

- Total cost savings: high absorbent capacity means fewer changes, fewer units purchased, fewer units to dispose, and less labor required.
- Laundry cost savings: fewer clothing/bedding changes due to leakage, less detergent and water usage, longer linen life, and less labor.

**TRANQUILITY**  
PREMIUM PROTECTION



## Peach Mat Guarantee®

**Tranquility Premium Protection Products** are carefully engineered to provide premium protection. The Peach Mat construction uses a proprietary composite structure of tissues, superabsorbent polymers, cellulose, and liquid dispersing materials. The peach-colored core assures the product is the Tranquility brand. The clinically proven design guarantees four key factors:

1. Unsurpassed capacity – protects against leakage and skin irritation.
2. Odor reduction – due to polymer selection and product construction odor is reduced.
3. Urine pH neutralization – protects delicate skin from the caustic effect of urine.
4. Inhibition of bacterial growth – deprives bacteria of a place or conditions to thrive.

You can rely on Tranquility's promise of unsurpassed overall performance in the areas of...

- skin dryness
- odor reduction
- urine neutralization
- inhibition of bacterial growth.

*In fact, we guarantee it!*

**Peach Mat Guarantee®**

## CONSUMER

Uninterrupted sleep &  
reduced health risks for  
wounds and falls



## CAREGIVERS

Improved morale &  
productivity



## FACILITIES

Reduced product usage,  
laundry & odors



# WHEN PERFORMANCE MATTERS MOST.

Caregivers have peace of mind they are giving their loved ones the best care and a sense of relief from daily household chores. Wearers have confidence to pursue normal daily activities.

Tranquility products guarantee unsurpassed, overall performance in the areas of skin dryness, odor reduction, urine pH neutralization, and inhibition of bacterial growth.

**TRANQUILITY®**  
PREMIUM PROTECTION

## Look for these high performance features in Tranquility products:

- PMG Peach Mat Guarantee®**  
Exclusive to Tranquility Premium Protection Products
- WI Wetness Indicator**  
Yellow indicator lines turn to blue when moisture is present eliminating unnecessary changes
- BR Breathable**  
Allows air to circulate around the body, promoting skin integrity
- RT Refastenable Tapes**  
Advanced tape system allows multiple openings of the briefs
- KG Kufguards®**  
Inner leg cuffs channel fluids into the core, helping to contain high volume episodes
- AS Adhesive Strip**  
Secures product in place
- CL Cloth-Like Backsheet**  
Soft, rustle free outer layer
- LF Latex-Free**
- MH Micro-Hook Closure Tabs**  
Refastenable tabs grip the soft, cloth-like outer layer to securely fasten the brief
- Absorbency**  
Product fluid capacity represented by standard 8-ounce measuring cup



Breathable  
Sides

Absorbency: 34 oz/1005 mL, size Large



## SmartCore®

Breathable sides help eliminate heat and moisture, providing an ideal micro-climate for skin integrity. Color coded by size.

Product Features:



ITEM	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2311	Small (Green)	24" - 32"	85-100 lbs	22 oz/650 mL	10 bags of 10 (100)	T4521
2312	Medium (White)	32" - 44"	100-150 lbs	28 oz/828 mL	8 bags of 12 (96)	T4522
2313	Large (Blue)	45" - 58"	150-200 lbs	34 oz/1005 mL	8 bags of 12 (96)	T4523
2314	X-Large (Beige)	56" - 64"	200-250 lbs	34 oz/1005 mL	6 bags of 12 (72)	T4524
2315	2XL/Bariatric (White)	60" - 80"	200-250 lbs	34 oz/1005 mL	4 bags of 8 (32)	T4543



Absorbency: 33 oz/975 mL, size Large



## ATN™ (All-Through-the-Night)

A full-fit, poly-backed brief with expanded surface area providing longer wear-time by absorbing multiple voids.

Product Features:



ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2183	X-S/Youth 6-7*	18" - 26"	42-90 lbs	18.5 oz/547 mL	10 bags of 10 (100)	T4533
2184	Small	24" - 32"	85-100 lbs	22.2 oz/657 mL	10 bags of 10 (100)	T4521
2185	Medium	32" - 44"	100-150 lbs	27.5 oz/812 mL	8 bags of 12 (96)	T4522
2186	Large	45" - 58"	150-200 lbs	33 oz/975 mL	8 bags of 12 (96)	T4523
2187	X-Large	56" - 64"	200-250 lbs	34 oz/1005 mL	6 bags of 12 (72)	T4524

\* Cloth-like backsheet, Micro-hook closure tabs



Absorbency: 21.4 oz/632 mL, size Large



## SlimLine® Original

Classic full-fit, poly-backed brief with a slimmer profile makes it versatile for everyday use.

Product Features:



ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2112	Youth 5-6*	16" - 22"	28-42 lbs	10.2 oz/302 mL	10 bags of 12 (120)	T4530
2166	X-S/Youth 6-7*	18" - 26"	42-90 lbs	10.9 oz/322 mL	10 bags of 10 (100)	T4533
2120	Small*	24" - 32"	85-100 lbs	12.7 oz/374 mL	10 bags of 10 (100)	T4521
2122	Medium	32" - 44"	100-150 lbs	20.4 oz/602 mL	8 bags of 12 (96)	T4522
2132	Large	45" - 58"	150-200 lbs	21.4 oz/632 mL	8 bags of 12 (96)	T4523
2134	X-Large	56" - 64"	200-250 lbs	23.7 oz/702 mL	6 bags of 12 (72)	T4524

\* Cloth-like backsheet, Micro-hook closure tabs



MAXIMUM

PROTECTION

Stretchable  
Side Panels

Absorbency: 34 oz/1005 mL



## AIR-Plus™ Bariatric

100% breathable allowing for heat and moisture vapor to escape enhancing skin integrity. Fits up to a 108" waist.

Product  
Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2195	4-5XL	70" - 108"	250+ lbs	34 oz/1005 mL	4 bags of 8 (32)	T4543

MAXIMUM

PROTECTION

Stretchable  
Side Panels

Absorbency: 34 oz/1005 mL



## HI-Rise™ Bariatric

Full-fit brief with higher coverage in the front and back for a secure fit. Fits up to a 96" waist. Breathable, stretchy side panels.

Product  
Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2192	3XL	64" - 96"	250+ lbs	34 oz/1005 mL	4 bags of 8 (32)	T4543

MAXIMUM

PROTECTION

Stretchable  
Side Panels

Absorbency: 34 oz/1005 mL



## Bariatric

Full-fit brief with breathable, stretchy side panels comfortably fits up to a 96" waist.

Product  
Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2190	3XL	64" - 96"	250+ lbs	34 oz/1005 mL	4 bags of 8 (32)	T4543

## Premium OverNight™

Maximum absorbency provides extended wear-time for a full night of uninterrupted sleep.



Absorbency: 34 oz/1005 mL, size Large



Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2113	X-Small/Youth Large	17" - 28"	65-85 lbs	20.3 oz/600 mL	4 bags of 22 (88)	T4534
2114	Small/Youth X-Large	22" - 36"	80-125 lbs	20.3 oz/600 mL	4 bags of 20 (80)	T4525/T4534
2115	Medium*	34" - 48"	120-175 lbs	34 oz/1005 mL	4 bags of 18 (72)	T4526
2116	Large*	44" - 54"	170-210 lbs	34 oz/1005 mL	4 bags of 16 (64)	T4527
2117	X-Large*	48" - 66"	210-250 lbs	34 oz/1005 mL	4 bags of 14 (56)	T4528
2118	2XL/Bariatric*	62" - 80"	250+ lbs	34 oz/1005 mL	4 bags of 12 (48)	T4544

\* Blue leg elastics denote Premium OverNight from Premium DayTime.

## Premium DayTime™

Discreet daytime protection for active individuals with heavier urine voids.



Absorbency: 25 oz/740 mL, size Large



Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2105	Medium	34" - 48"	120-175 lbs	25 oz/740 mL	4 bags of 18 (72)	T4526
2106	Large	44" - 54"	170-210 lbs	25 oz/740 mL	4 bags of 16 (64)	T4527
2107	X-Large	48" - 66"	210-250 lbs	25 oz/740 mL	4 bags of 14 (56)	T4528
2108	2XL/Bariatric	62" - 80"	250+ lbs	25 oz/740 mL	4 bags of 12 (48)	T4544

For sizes X-Small and Small, try Premium OverNight.



"As a full time caregiver to my wife, I have painfully witnessed a steady cognitive and physical decline over the past six years. Last fall she started with incontinence. This winter, I finally gave in to not getting up with her to go to the toilet because it was literally wearing me out. I discovered your underwear, which I put on her at bedtime. They work! I am able to get a reasonable night's sleep, have more patience and don't have to change the sheets!"

— M.J., Husband and Caregiver, Belvidere, NJ

MAXIMUM  
PROTECTION



NEW!

## AIR-Plus™ Extra-Strength Breathable

Strong backsheet for boosting, turning and repositioning.\*  
100% breathable; allows for heat and moisture vapor to escape.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2711	30" x 36"	34 oz/1005 mL	4 bags of 10 (40)	T4541/A4554

\*Product should be used in accordance with Safe Patient Handling Guidelines.  
Lay printed side down against bed, chair, or other surface.

Absorbency: 34 oz/1005 mL



MAXIMUM  
PROTECTION



## AIR-Plus™ Breathable

100% breathable backsheet allows for heat and moisture vapor to escape. Ideal for use with low-air-loss bed systems.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2709	23" x 36"	29 oz/850 mL	4 bags of 10 (40)	T4541/A4554
2710	30" x 36"	34 oz/1005 mL	4 bags of 10 (40)	T4541/A4554

Lay printed side down against bed, chair, or other surface.

Printed Backsheet with Trilingual Placement Instructions

Absorbency: 34 oz/1005 mL, size 30" x 36"



MAXIMUM  
PROTECTION



## Peach Sheet

Poly-backed underpad has four adhesive tapes to secure it to wheelchairs, bedding and furniture.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2074	21.5" x 32.5"	34 oz/1005 mL	8 bags of 12 (96)	T4541/A4554

Absorbency: 34 oz/1005 mL



MAXIMUM  
PROTECTION



## Heavy-Duty

Strong cloth-like backsheet resists tearing as an individual moves on the underpad.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2088	30" x 36"	34 oz/1005 mL	6 bags of 10 (60)	T4541/A4554

Absorbency: 34 oz/1005 mL





## Male Guard

Absorbent core locks in fluid and controls odor. Contoured shape with side elastics forms a cup that conforms to a man's body.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2385	One Size	12.25" x 5.25"	9.5 oz/280 mL	2 bags of 52 (104)	T4535

Absorbency: 9.5 oz/280 mL



## Personal Care Pads

Features two embossed channels to direct fluid into the pad and elastics to form a cup shape. Worn in regular underwear.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2380	Super*	10.5" x 5.5"	12.6 oz/375 mL	4 bags of 24 (96)	T4535
2381	Ultimate*	13.5" x 6.5"	15.2 oz/450 mL	4 bags of 24 (96)	T4535
2382	OverNight	16.5" x 7.25"	16.9 oz/500 mL	4 bags of 24 (96)	T4535

Absorbency: 15.2 oz/450 mL, size Ultimate



\* Individually packaged for hygienic protection

## Adult Liner

Thin, moisture-proof backed liner is worn in regular underwear for a discreet appearance.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF


ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2078	Adult	24" x 9"	15.5 oz/458 mL	4 bags of 30 (120)	T4535

Absorbency: 15.5 oz/458 mL



## What is a booster pad?

Booster pads have a flow-through design that allows the pad to fill to capacity first and then pass additional fluid to the host (primary) garment. Booster pads are placed inside any disposable undergarment with a moisture barrier, boosting absorbency to extend wear-time, reduce changes and help contain bowel incontinence. Strategically placed over the hip helps prevent leaking for side sleepers.

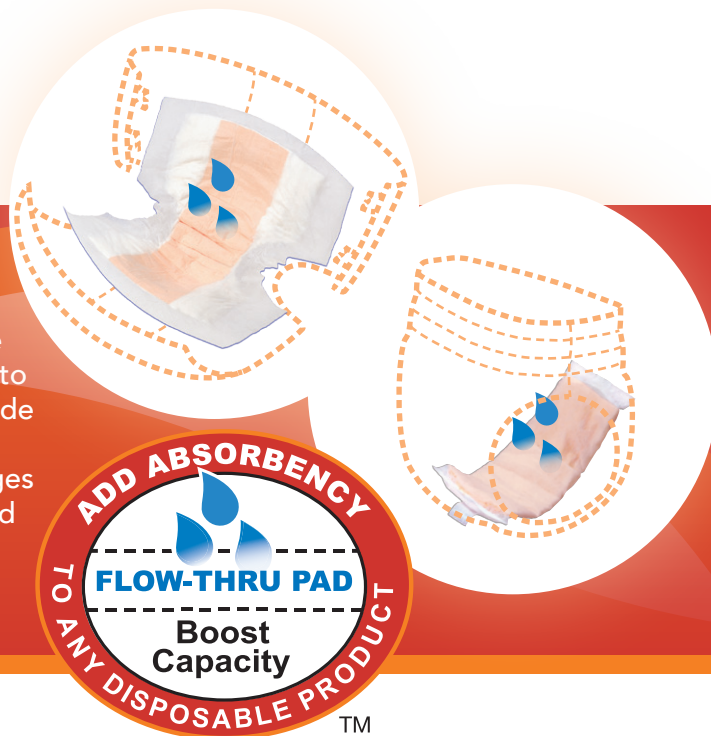

ADDED

PROTECTION



**Absorbency:** 14.2 oz/420 mL, size Super



## TopLiner® Booster Pads

Thin design allows for placement in target areas where additional absorption is needed.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2072	Mini	10.5" x 2.75"	5.7 oz/169 mL	8 bags of 25 (200)	T4535
2070	Regular	14" x 4"	10.9 oz/322 mL	8 bags of 25 (200)	T4535
2060	Super	15" x 4.25"	14.2 oz/420 mL	8 bags of 25 (200)	T4535

ADDED

PROTECTION



**Absorbency:** 13.6 oz/403 mL, size Contour



## TopLiner® Booster Contours

Wide hourglass shape helps contain bowel. Soft texture and no adhesive strip makes cleanup easier.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
3096	Contour	21.5" x 13.5"	13.6 oz/403 mL	10 bags of 12 (120)	T4535
3097	Super-Plus*	32" x 14"	27.5 oz/813 mL	8 bags of 12 (96)	T4535

\* Ideal for use with sizes XL and greater.



## ThinLiner Moisture Management® Sheets

100% breathable, soft cloth-like sheets reduce skin-on-skin friction. Collects, absorbs and retains moisture.

Product Features:



ITEM #	LENGTH/WIDTH	ABSORBENCY*	PACKAGING	HCPC
3190	6"x 10"	6 oz/177 mL*	20 bags of 10 (200)	A9999
3191	6"x 14"	8.5 oz/251 mL*	20 bags of 10 (200)	A9999
3092	20"x 22"	15 oz/450 mL*	4 bags of 25 (100)	A9999

\* Moisture absorbency (not C.U.P.)



Note: NEVER CUT, only fold, ThinLiner Sheets to accommodate various sizes and applications.

### Possible Applications:



## Cleansing Wipes

Paraben and alcohol free, hypoallergenic with a mild scent. Generous 9" x 13" size allows for quicker cleanup.

Product Features:



ITEM #	SIZE	LENGTH/WIDTH	PACKAGING	HCPC
3101	Full Case	9"x 13"	12 packs of 56 (672)	A4335
3121	Mini Case	9"x 13"	4 packs of 56 (224)	A4335



"Haven't had any leaks since using these products. Had to get up 2-3 times a night with my elderly mother in the past. Now we can both get an uninterrupted night's sleep and not have to change bedding, etc. Thank you. TopLiners are inexpensive and save having to change disposable underwear every time. Thanks for giving us back our rest."

— B.T., Daughter and Caregiver, Rowe, NM





# EXCELLENT CAPACITY, OUTSTANDING VALUE.



Heavy Protection, Excellent Value.  
High quality products with proven performance.

**SELECT**  
Incontinence Products

## Look for these high performance features in Select products:

- BR Breathable**  
Allows air to circulate around the body, promoting skin integrity.
- KG Kufguards®**  
Inner leg cuffs channel fluids into the core, helping to contain high volume episodes.
- CL Cloth-Like Backsheet**  
Product outer layer is soft and rustle free.
- MH Micro-Hook Closure Tabs**  
Refastenable tabs grip the soft, cloth-like outer layer to securely fasten the brief.
- WI Wetness Indicator**  
Yellow indicator lines turn to blue when moisture is present eliminating unnecessary changes.
- RT Refastenable Tapes**  
Advanced tape system allows multiple openings of the briefs.
- AS Adhesive Strip**  
Secures product in place.
- LF Latex-Free**
- Absorbency**  
Product fluid capacity represented by standard 8-ounce measuring cup.



Absorbency: 19.3 oz/571 mL, size Large



## Briefs

A full-fit, poly-backed brief with color coding for easy size identification.

Product Features:



ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
3665	Youth 5-6* (White)	16" - 22"	28-42 lbs	9.8 oz/289 mL	10 bags of 10 (100)	T4530
3666	X-S/Youth 6-7* (White)	18" - 26"	42-90 lbs	10.4 oz/308 mL	10 bags of 10 (100)	T4533
2620	Small (Green)	24" - 32"	85-100 lbs	11.8 oz/348 mL	10 bags of 10 (100)	T4521
2624	Medium (White)	32" - 44"	100-150 lbs	15.8 oz/468 mL	8 bags of 12 (96)	T4522
2634	Large (Blue)	45" - 58"	150-200 lbs	19.3 oz/571 mL	6 bags of 12 (72)	T4523
2635	X-Large (Beige)	56" - 64"	200-250 lbs	23.0 oz/681 mL	8 bags of 8 (64)	T4524

\* Cloth-like backsheets, Micro-hook closure tabs



Absorbency: 19.3 oz/571 mL, size Large



## Soft n' Breathable™

Contoured core with breathable sides allow for air circulation. Color coded for easy size identification.

Product Features:



ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2627	Medium (White)	32" - 44"	100-150 lbs	15.8 oz/468 mL	8 bags of 12 (96)	T4522
2628	Large (Blue)	45" - 58"	150-200 lbs	19.3 oz/571 mL	6 bags of 12 (72)	T4523
2629	X-Large (Beige)	56" - 64"	200-250 lbs	23.0 oz/681 mL	8 bags of 8 (64)	T4524

## Disposable Absorbent Underwear

Highly absorbent underwear with breathable sides and full-rise waist panel for a comfortable, secure fit.



Absorbency: 19 oz/562 mL, size Large



Product Features:



ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2602	Youth/Youth Medium	15" - 25"	38-65 lbs	14.5 oz/429 mL	8 bags of 12 (96)	T4534
2603	X-S/Youth Large	17" - 28"	65-85 lbs	14.5 oz/429 mL	4 bags of 24 (96)	T4534
2604	Small/Youth X-Large	22" - 36"	80-125 lbs	14.5 oz/429 mL	4 bags of 22 (88)	T4525/T4534
2605	Medium	34" - 48"	120-175 lbs	19.0 oz/562 mL	4 bags of 20 (80)	T4526
2606	Large	44" - 54"	170-210 lbs	19.0 oz/562 mL	4 bags of 18 (72)	T4527
2607	X-Large	48" - 66"	210-250 lbs	19.0 oz/562 mL	4 bags of 14 (56)	T4528
2608	2XL/Bariatric	62" - 80"	250+ lbs	19.0 oz/562 mL	4 bags of 12 (48)	T4544



MODERATE

PROTECTION



Absorbency: 11.8 oz/349 mL, size Regular



## Personal Care Pads

Contoured pads with moisture barrier fit securely into regular underwear for easy changes.

Product Features:



ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2880	Light*	10.5" x 5.5"	10.1 oz/299 mL	4 bags of 24 (96)	T4535
2881	Regular*	13.5" x 6.5"	11.8 oz/349 mL	4 bags of 24 (96)	T4535
2882	Extra	16.5" x 7.25"	12.6 oz/375 mL	4 bags of 24 (96)	T4535

\* Individually packaged for hygienic protection

ADDED

PROTECTION



Absorbency: 6.7 oz/198 mL, size Medium



## Booster Pads

A soft, flow-through pad designed to fill to capacity and then pass additional fluid into the primary garment.

Product Features:



ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2770	Small	12" x 3.25"	5.5 oz/163 mL	6 bags of 30 (180)	T4535
2760	Medium	12" x 4.25"	6.7 oz/198 mL	8 bags of 25 (200)	T4535
2762	Extended	15" x 4.25"	8.0 oz/236 mL	8 bags of 25 (200)	T4535

ADDED

PROTECTION



Absorbency: 8.4 oz/248 mL



## Ultra-Thin

Ultra-thin, composite mat is fluff-free (no shake-out). Consistently smooth, uniform construction resists bunching during use.


Product Features:



ITEM #	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2717	22.5" x 30"	8.4 oz/248 mL	5 bags of 30 (150)	T4541/A4554

ADDED

PROTECTION

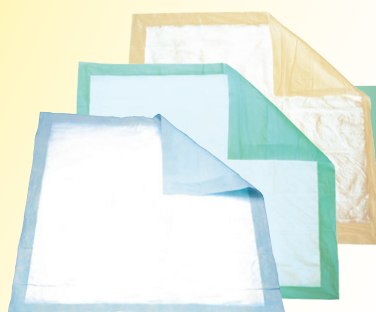

Absorbency: 18.5 oz/550 mL, size Ultra Large



## Standard

Perfect for protecting bedding, furniture and wheelchairs from leakage.

Product Features:



ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2675	Large (Blue)	23" x 36"	8.1 oz/239 mL	6 bags of 25 (150)	T4541/A4554
2677	Extra Large (Green)	28" x 30"	10.6 oz/314 mL	10 bags of 10 (100)	T4541/A4554
2679	Ultra Large (Beige)	36" x 36"	18.5 oz/550 mL	5 bags of 10 (50)	T4541/A4554





# MODERATE PROTECTION, QUALITY DESIGN.

## ComfortCare™

Incontinence Products

"Best in Class" of mass retail products.



Absorbency: 13.5 oz/400 mL, size Large



### Briefs

Provides breathable comfort in a value brief where cost containment is the key focus.

Product Features:



ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2965-100	Medium (White)	32" - 44"	100-150 lbs	13.5 oz/400 mL	4 bags of 25 (100)	T4522
2966-100	Large (Blue)	45" - 58"	150-200 lbs	13.5 oz/400 mL	4 bags of 25 (100)	T4523
2967-100	X-Large (Beige)	56" - 64"	200-250 lbs	13.5 oz/400 mL	4 bags of 25 (100)	T4524

MODERATE  
PROTECTION



Absorbency: 13.5 oz/400 mL, size Large



### Disposable Absorbent Underwear

Ideal for active individuals. Best absorption in the economy class.

Product Features:



ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2974-100	Small/Youth XL	22" - 36"	80-125 lbs	10.1 oz/299 mL	4 bags of 25 (100)	T4525/T4534
2975-100	Medium	34" - 48"	120-175 lbs	13.5 oz/400 mL	4 bags of 25 (100)	T4526
2976-100	Large	44" - 54"	170-210 lbs	13.5 oz/400 mL	4 bags of 25 (100)	T4527
2977-100	X-Large	48" - 66"	210-250 lbs	13.5 oz/400 mL	4 bags of 25 (100)	T4528

MODERATE  
PROTECTION



Absorbency: 5.3 oz/157 mL



### Booster Pads

Flow-through pad adds absorbency to any disposable undergarment.


Product Features:



ITEM #	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
19244	Booster	12" x 4"	5.3 oz/157 mL	8 bags of 25 (200) T4535

ADDED  
PROTECTION

For Aqua Therapy or  
Any Day in the Pool

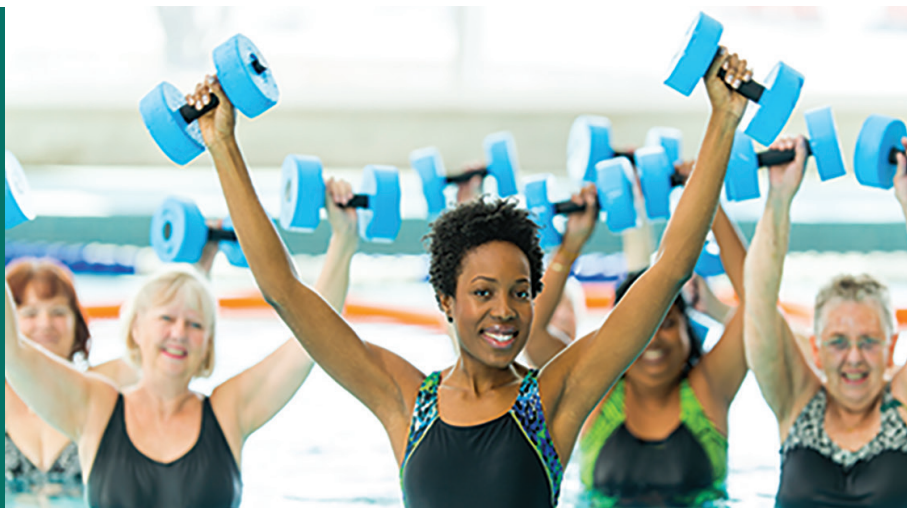

## Disposable Swimwear

This discreet unisex disposable underwear is worn under a bathing suit. Features Kufguards® (inner leg cuffs) and moisture barrier to help contain bowel incontinence. Designed for minimal swelling and does not break apart when wet.

**Product Features:**

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	WEIGHT	PACKAGING	HCPC
2844	Small/Youth XL	22" - 36"	80-125 lbs	4 bags of 22 (88)	T4525/T4534/A9999
2845	Medium	34" - 48"	120-175 lbs	4 bags of 20 (80)	T4526/A9999
2846	Large	44" - 54"	170-210 lbs	4 bags of 18 (72)	T4527/A9999
2847	X-Large	48" - 66"	210-250 lbs	4 bags of 14 (56)	T4528/A9999
2848	XXL-Plus/Bariatric	62" - 80"	250+ lbs	4 bags of 12 (48)	T4544/A9999



BR Breathable

KG Kufguards

CL Cloth-Like

MH Micro-Hook

WI Wetness Indicator

RT Refastenable Tapes

AS Adhesive Strip

LF Latex-Free

= 8 oz.

# Moisture Management Standards



Effective products dealing with moisture need to meet exacting performance criteria. The objectives are always to maintain or restore the ideal “micro-climate” for skin, the safety and quality of life for the individuals and caregivers involved. **The following are fundamental principles and criteria that need to be considered:**

1. Sufficient absorbent capacity to provide 6-8 hours of protection to allow uninterrupted activity (such as sleep, working, shopping, etc.).
2. Protection from risks to the skin caused by shear, friction, heat, wetness and unbalanced pH.
3. Protection from the risk of falls associated with incontinence and/or wound exudate that leaves the feeling of wetness/cold from the absorbent products.
4. Design and absorbent capacity that eliminates the waste and costs incurred arising from product leakage and the resulting excessive cost and labor demands resulting from wet bedding and/or wet clothing.
5. Ease of use and effectiveness that minimizes the burden and frustration for caregivers or professional staff that leads to “burnout.”
6. The high performance that allows care-providers to focus on individual-centered therapies and life-enhancement strategies for care recipients.

## Frequently Asked Questions

### **1. Will drinking less fluid decrease my challenges with incontinence?**

No. Cutting back on fluid intake can lead to dehydration, constipation and irritation of the bladder. It is true that certain types of fluids should be avoided such as alcohol and caffeine that irritate the bladder.

### **2. What can I do to stop leaking with my current incontinence products?**

The main factors to take into consideration when choosing the correct incontinence product are:

- Sizing and Fit: Make sure you have the right size product. Leg openings should be snug fitting (no gaps) and should not create redness around the leg/groin area.
- Style and absorption capacity required: Make sure that you are wearing the correct product style and absorbency level to meet your needs. Refer to the Product Selection Chart (page 20) to assist in selection of the right style of product to meet ones mobility and incontinence level.

### **3. Can two briefs/pads be used at the same time (also called “diaper doubling”)?**

No, doubling up on products that contain a moisture barrier will not increase absorption. Briefs, disposable underwear and pads/liners

are typically designed with a moisture barrier that stops the urine from passing through. This approach to adding more capacity or addressing leakage concerns works only if the second product is a “flow-through” design with no moisture barrier (Refer to booster pads on pages 12, 16 & 17).

### **4. Why are some incontinence products so much more expensive?**

Higher performing incontinence products (that may cost more) typically contain more super absorbent capacity and are made with superior quality materials. The Tranquility Product Family keeps skin drier, neutralizes urine pH, and reduces odors. It is often possible to save money by using the more expensive products because they require fewer changes, result in fewer clothing/bedding changes, less waste, less laundry due to leakage, and better quality of life for both the care provider and the care recipient.

### **5. How is bowel incontinence managed?**

Booster pads are designed to contain the bowel, assist in clean-up and can extend the wear time of the “host garment.” The focus for bowel incontinence is on containment not absorption. (Refer to booster pads on page 12).








# Product Selection Chart

Refer to the chart below for product selections/recommendations based on type of incontinence, frequency and absorbency level needed. Incontinence types are defined below for reference.

## Incontinence Level:

	1	2	3	4	5
Incontinence Type (see below for descriptions)	Stress	Stress Urge Mixed	Overflow Reflex Functional	Overflow Reflex Functional	Overflow Reflex Functional
Incontinence (Urine/Bowel)	Occasionally Incontinent Partial voids No Fecal	Occasionally Incontinent Partial Voids Occasional Fecal Smears	Frequently Incontinent Full Void Fecal Incontinent	Frequently Incontinent Multiple Voids Fecal Incontinent	Frequently Incontinent Multiple Voids Fecal Incontinent
Mental Orientation/ Mobility Level	Oriented/Clear Active No Assistance	Oriented/Clear Active Minimal Assistance	Oriented/Clear to Slightly Confused Active; Semi-Active Minimal to Moderate Assistance	Oriented/Clear to Confused Semi-Active; Inactive Moderate Assistance to Immobile	Oriented/Clear to Confused Inactive Moderate Assistance to Immobile

## Absorbency Level:

	Light	Moderate	Heavy	Maximum	Maximum-Plus
Pads/Liners 		Tranquility® Personal Care Pads Tranquility® Male Guard Select® Personal Care Pads Tranquility® Adult Liner			
Briefs 		ComfortCare™	Tranquility® SlimLine® Original Select® Soft n' Breathable™ Select® Briefs	Tranquility® SmartCore® Tranquility® ATN™ Tranquility® Bariatric	
Underwear 		ComfortCare™	Tranquility® DayTime™ Select® Underwear	Tranquility® OverNight™	
Booster Pads 	Booster pads can be added to any pad, liner, brief or underwear to assist with bowel incontinence or to increase absorption.				
Underpads 		Select® Large Select® XL Select® Ultra-Thin	Select® Ultra Large	Tranquility® AirPlus™ Extra-Strength Breathable Tranquility® AirPlus™ Breathable Tranquility® Peach Sheet Tranquility® Heavy-Duty	

## Definitions: Incontinence types

### Stress:

Leakage of small amounts of urine due to sudden pressure on the bladder, such as coughing, sneezing, laughing or exercising.

### Urge:

A sudden urge to urinate that is so strong it cannot be held long enough to reach the bathroom.

### Mixed:

A combination of the symptoms of stress and urge incontinence.

### Overflow:

Involuntary urination. Bladder muscles are unable to contain urine under pressure. Feeling of inability to empty bladder completely.

### Reflex:

Involuntary urination without sensation of a full bladder. Unaware of the need to urinate.

### Functional:


Inability to get to the bathroom fast enough due to mobility or cognitive impairments.

# Sizing Guide

Suggested sizes are a guideline. When in doubt, best practice is to use a tape measure and obtain an accurate hip measurement. **For sizes greater than an XL, obtain an accurate hip measurement for best fit.**

## How to Measure

Use a tape measure to measure in inches the widest part of your body between waist and hip. Use this number to determine product size.

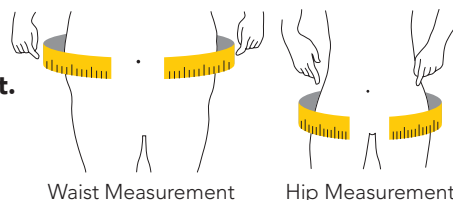


## Disposable Briefs

Size Description	Waist/Hips (inches)	Weight (pounds)	Women's Pant Size	Men's Pant Size
Youth	16" - 22"	28 - 42 lbs	N/A	N/A
X-Small	18" - 26"	42 - 90 lbs	0-3	18-26
Small	24" - 32"	85 - 100 lbs	4-6	24-32
Medium	32" - 44"	100 - 150 lbs	8-12	32-44
Large	45" - 58"	150 - 200 lbs	14-18	44-58
X-Large	56" - 64"	200 - 250 lbs	18-24	56-64
2XL	60" - 80"	250+ lbs	26-28	60+
3XL	64" - 96"	250+ lbs	28+	64+
4-5XL	70" - 108"	250+ lbs	28+	70+

Briefs are color-coded for easy size identification.


### Correct Fitting Brief

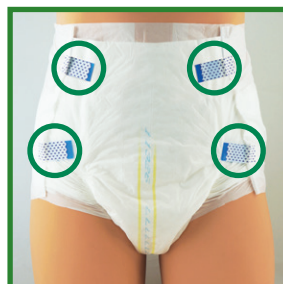

#### Correct Fit


Product should fit snugly with the placement of the tape tabs on the hips and no gaps around the legs.

### Poor Fitting Brief

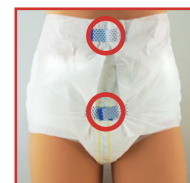

#### Too big

Brief is too large if tape tabs overlap, will cause leakage.

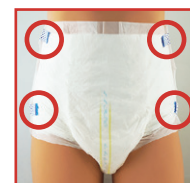

#### Too small

Brief is too small if tape tabs reach side panels, may cause irritation.

## Disposable Absorbent Underwear

Size Description	Waist/Hips (inches)	Weight (pounds)	Women's Pant Size	Men's Pant Size
Youth	15" - 25"	38 - 65 lbs	N/A	N/A
X-Small	17" - 28"	65 - 85 lbs	0-3	17-28
Small	22" - 36"	80 - 125 lbs	4-6	22-36
Medium	34" - 48"	120 - 175 lbs	6-12	34-48
Large	44" - 54"	170 - 210 lbs	12-18	44-54
X-Large	48" - 66"	210 - 250 lbs	18-24	48-66
2XL	62" - 80"	250+ lbs	24+	62+

### Correct Fitting DAU



#### Correct Fit


Product should fit snugly, like underwear, touching the crotch with no gaps around the legs.

### Poor Fitting DAU



#### Too big

Gaps around legs, underwear not touching the crotch will cause leakage.

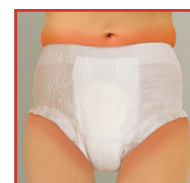

#### Too small

Too tight around waist and legs, may cause irritation.

# Quick Order Reference Guide

Item #	Product Description	Unit/ Case	Pkg/ Case	HCPC	Printed Pkg	Page #
2060	Tranquility® TopLiner® Booster Pad - Super	200	8/25	T4535	Yes	12
2070	Tranquility® TopLiner® Booster Pad - Regular	200	8/25	T4535	Yes	12
2072	Tranquility® TopLiner® Booster Pad - Mini	200	8/25	T4535	Yes	12
2074	Tranquility® Peach Sheet Underpad	96	8/12	T4541/A4554	Yes	10
2078	Tranquility® Adult Liner	120	4/30	T4535	Yes	11
2088	Tranquility® Heavy-Duty Underpad	60	6/10	T4541/A4554	No	10
2105	Tranquility® Premium DayTime™ Underwear - Medium	72	4/18	T4526	Yes	9
2106	Tranquility® Premium DayTime™ Underwear - Large	64	4/16	T4527	Yes	9
2107	Tranquility® Premium DayTime™ Underwear - X-Large	56	4/14	T4528	Yes	9
2108	Tranquility® Premium DayTime™ Underwear - 2XL/Bariatric	48	4/12	T4544	Yes	9
2112	Tranquility® SlimLine® Original Brief - Youth 5/6	120	10/12	T4530	Yes	7
2113	Tranquility® Premium OverNight™ Underwear - X-Small / Youth Large	88	4/22	T4534	Yes	9
2114	Tranquility® Premium OverNight™ Underwear - Small / Youth X-Large	80	4/20	T4525/T4534	Yes	9
2115	Tranquility® Premium OverNight™ Underwear - Medium	72	4/18	T4526	Yes	9
2116	Tranquility® Premium OverNight™ Underwear - Large	64	4/16	T4527	Yes	9
2117	Tranquility® Premium OverNight™ Underwear - X-Large	56	4/14	T4528	Yes	9
2118	Tranquility® Premium OverNight™ Underwear - 2XL/Bariatric	48	4/12	T4544	Yes	9
2120	Tranquility® SlimLine® Original Brief - Small	100	10/10	T4521	Yes	7
2122	Tranquility® SlimLine® Original Brief - Medium	96	8/12	T4522	Yes	7
2132	Tranquility® SlimLine® Original Brief - Large	96	8/12	T4523	Yes	7
2134	Tranquility® SlimLine® Original Brief - X-Large	72	6/12	T4524	Yes	7
2166	Tranquility® SlimLine® Original Brief - X-Small / Youth 6/7	100	10/10	T4533	Yes	7
2183	Tranquility® ATN™ (All-Through-the-Night) Brief - X-Small / Youth 6/7	100	10/10	T4533	Yes	7
2184	Tranquility® ATN™ (All-Through-the-Night) Brief - Small	100	10/10	T4521	Yes	7
2185	Tranquility® ATN™ (All-Through-the-Night) Brief - Medium	96	8/12	T4522	Yes	7
2186	Tranquility® ATN™ (All-Through-the-Night) Brief - Large	96	8/12	T4523	Yes	7
2187	Tranquility® ATN™ (All-Through-the-Night) Brief - X-Large	72	6/12	T4524	Yes	7
2190	Tranquility® Bariatric Brief - 3XL	32	4/8	T4543	Yes	8
2192	Tranquility® HI-Rise™ Bariatric Brief - 3XL	32	4/8	T4543	Yes	8
2195	Tranquility® AIR-Plus™ Bariatric Brief - 4-5XL	32	4/8	T4543	Yes	8
2311	Tranquility® SmartCore® Brief - Small	100	10/10	T4521	Yes	7
2312	Tranquility® SmartCore® Brief - Medium	96	8/12	T4522	Yes	7
2313	Tranquility® SmartCore® Brief - Large	96	8/12	T4523	Yes	7
2314	Tranquility® SmartCore® Brief - X-Large	72	6/12	T4524	Yes	7
2315	Tranquility® SmartCore® Brief - 2XL/Bariatric	32	4/8	T4543	Yes	7
2380	Tranquility® Personal Care Pad - Super	96	4/24	T4535	Yes	11
2381	Tranquility® Personal Care Pad - Ultimate	96	4/24	T4535	Yes	11
2382	Tranquility® Personal Care Pad - OverNight	96	4/24	T4535	Yes	11
2385	Tranquility® Male Guard	104	2/52	T4535	Yes	11
2602	Select® Absorbent Underwear - Youth/Youth Medium	96	8/12	T4534	Yes	15
2603	Select® Absorbent Underwear - X-Small / Youth Large	96	4/24	T4534	Yes	15
2604	Select® Absorbent Underwear - Small / Youth X-Large	88	4/22	T4525/T4534	Yes	15
2605	Select® Absorbent Underwear - Medium	80	4/20	T4526	Yes	15
2606	Select® Absorbent Underwear - Large	72	4/18	T4527	Yes	15
2607	Select® Absorbent Underwear - X-Large	56	4/14	T4528	Yes	15
2608	Select® Absorbent Underwear - 2XL/Bariatric	48	4/12	T4544	Yes	15



Item #	Product Description	Unit/ Case	Pkg/ Case	HCPC	Printed Pkg	Page #
2620	Select® Disposable Brief - Small	100	10/10	T4521	Yes	15
2624	Select® Disposable Brief - Medium	96	8/12	T4522	Yes	15
2627	Select® Soft n' Breathable™ Brief - Medium	96	8/12	T4522	Yes	15
2628	Select® Soft n' Breathable™ Brief - Large	72	6/12	T4523	Yes	15
2629	Select® Soft n' Breathable™ Brief - X-Large	64	8/8	T4524	Yes	15
2634	Select® Disposable Brief - Large	72	6/12	T4523	Yes	15
2635	Select® Disposable Brief - X-Large	64	8/8	T4524	Yes	15
2675	Select® Underpads - Large	150	6/25	T4541/A4554	Yes	16
2677	Select® Underpads - Extra Large	100	10/10	T4541/A4554	No	16
2679	Select® Underpads - Ultra Large	50	5/10	T4541/A4554	No	16
2709	Tranquility® AIR-Plus™ Breathable Underpad 23 x 36	40	4/10	T4541/A4554	Yes	10
2710	Tranquility® AIR-Plus™ Breathable Underpad 30 x 36	40	4/10	T4541/A4554	Yes	10
2711	Tranquility® AIR-Plus™ Extra-Strength Breathable Underpad 30 x 36	40	4/10	T4541/A4554	Yes	10
2717	Select® Ultra-Thin Underpad	150	5/30	T4541/A4554	No	16
2760	Select® Booster Pad - Medium	200	8/25	T4535	Yes	16
2762	Select® Booster Pad - Extended	200	8/25	T4535	No	16
2770	Select® Booster Pad - Small	180	6/30	T4535	Yes	16
2844	Swimmates® - Small/Youth X-Large	88	4/22	T4525/T4534/ A9999	Yes	18
2845	Swimmates® - Medium	80	4/20	T4526/A9999	Yes	18
2846	Swimmates® - Large	72	4/18	T4527/A9999	Yes	18
2847	Swimmates® - X-Large	56	4/14	T4528/A9999	Yes	18
2848	Swimmates® - XXL-Plus/Bariatric	48	4/12	T4544 A9999	Yes	18
2880	Select® Personal Care Pads - Light	96	4/24	T4535	Yes	16
2881	Select® Personal Care Pads - Regular	96	4/24	T4535	Yes	16
2882	Select® Personal Care Pads - Extra	96	4/24	T4535	Yes	16
3092	Tranquility® ThinLiner Moisture Management® Sheets 20 x 22	100	4/25	A9999	Yes	13
3096	Tranquility® TopLiner® Booster Contour Pad	120	10/12	T4535	Yes	12
3097	Tranquility® TopLiner® Booster Contour Pad- Super-Plus	96	8/12	T4535	Yes	12
3101	Tranquility® Cleansing Wipes 9 x 13	672	12/56	A4335	Yes	13
3190	Tranquility® ThinLiner Moisture Management® Sheets 6 x 10	200	20/10	A9999	Yes	13
3191	Tranquility® ThinLiner Moisture Management® Sheets 6 x 14	200	20/10	A9999	Yes	13
3121	Tranquility® Cleansing Wipes 9 x 13	224	4/56	A4335	Yes	13
3665	Select® Disposable Brief - Youth 5/6	100	10/10	T4530	Yes	15
3666	Select® Disposable Brief - X-Small / Youth 6/7	100	10/10	T4533	Yes	15
2965-100	ComfortCare™ Briefs - Medium	100	4/25	T4522	Yes	17
2966-100	ComfortCare™ Briefs - Large	100	4/25	T4523	Yes	17
2967-100	ComfortCare™ Briefs - X-Large	100	4/25	T4524	Yes	17
2974-100	ComfortCare™ Disposable Underwear - Small / Youth X-Large	100	4/25	T4525/T4534	Yes	17
2975-100	ComfortCare™ Disposable Underwear - Medium	100	4/25	T4526	Yes	17
2976-100	ComfortCare™ Disposable Underwear - Large	100	4/25	T4527	Yes	17
2977-100	ComfortCare™ Disposable Underwear - X-Large	100	4/25	T4528	Yes	17
19244	ComfortCare™ Booster Pad	200	8/25	T4535	Yes	17



## Tranquility® Product Family

WHEN PERFORMANCE MATTERS MOST.

# Try it!

### Product Sampling/Promotional Programs

PBE offers a generous sample program, allowing the customer to experience the high quality performance of these unique incontinence products.



### Dealers/Distributors

Dealers and distributors can contact PBE at 1-800-467-3224 for information about Tranquility products' wide range of marketing services, consumer sampling programs, and support materials.

### Care Center

The Tranquility Care Center, 1-800-467-3224 ext. 7, provides FREE product samples, private consultation, and product information to consumers.

### Website

The Tranquility website, [www.TranquilityProducts.com](http://www.TranquilityProducts.com), features technical articles, product information, dealer locator assistance, and much more.

**PBE** Principle Business Enterprises, Inc.

P.O. Box 129  
Dunbridge, OH 43414  
419.352.1551

[www.PrincipleBusinessEnterprises.com](http://www.PrincipleBusinessEnterprises.com)

**Incontinence Products**  
**Proudly Made in the U.S.A.**  
from domestic and imported materials  
(except item #2602)



PBE is the acknowledged leader in the following fields of care:

- **Incontinence Care** – Tranquility® family of absorbent products provide premium protection for moisture and risk management. [www.TranquilityProducts.com](http://www.TranquilityProducts.com)
- **Fall Prevention** – Pillow Paws® brand slippers deliver protection from the risk of slips and falls, as well as hygienic protection and comfort. [www.PillowPaws.com](http://www.PillowPaws.com)



© Tranquility, Select, Swimmates, and Pillow Paws are registered trademarks of Principle Business Enterprises, Inc. | © Copyright 2020 | 1TRQ-802-SL20

800-467-3224 [www.TranquilityProducts.com](http://www.TranquilityProducts.com)