Sizing Guide

Suggested sizes are a guideline. When in doubt, best practice is to use a tape measure and obtain an accurate hip measurement (refer to sizing instruction document on reverse side). For sizes greater than XL, obtain an accurate hip measurement for best fit.

Disposable Brief

Size Description	Waist/Hips (inches)	Weight (pounds)	Women's Pant Size	Men's Pant Size
Youth	N/A	28-42 lbs	N/A	N/A
X-Small	18"-26"	42-90 lbs	0-3	18-26
Small	24"-32"	85-100 lbs	4-6	24-32
Medium	32"-44"	100-150 lbs	8-12	32-44
Large	45"-58"	150-200 lbs	14-18	44-58
X-Large	56"-64"	200-250 lbs	18-24	56-64
2XL	60"-80"	250+ lbs	26-28	60+
3XL	64"-96"	250+ lbs	28+	64+
4XL	70"-106"	250+ lbs	28+	70+

Briefs are color-coded for easy size identification.

Correct Fitting Brief



Correct Fit

- Correct tab placement
- Should fit like underwear
- No gaps

Poorly Fitting Brief



Too big

- Tape tabs overlapping
- Brief may leak



Too small

- Tape tabs only reach the side panels
- May cause skin irritation

Disposable Absorbent Underwear

Size Description	Waist/Hips (inches)	Weight (pounds)	Women's Pant Size	Men's Pant Size
Youth	15"-25"	38-65 lbs	N/A	N/A
X-Small	17"-28"	65-85 lbs	0-3	17-28
Small	22"-36"	80-125 lbs	4-6	22-36
Medium	34"-48"	120-175 lbs	6-12	34-48
Large	44"-54"	170-210 lbs	12-18	44-54
X-Large	48"-66"	210-250 lbs	18-24	48-66
2XL	62"-80"	250+ lbs	24+	62+

Correct Fitting DAU



Correct FitProduct should fit snugly touching the crotch with no gaps around the legs.

Poorly Fitting DAU



Too big

 Gaps around legs, underwear not touching the crotch will cause leakage



Too small

• Too tight around waist and legs, may cause skin irritation







For additional information or questions Call: 1-800-467-3224 x 340 or email: clinical@pbenet.com