



Hypervolt quick start instructions.

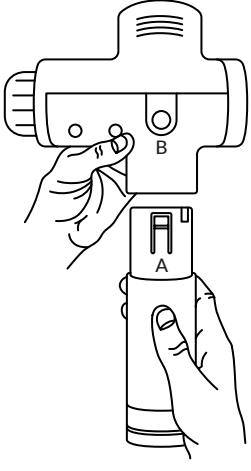


Aligning your battery.

Align the battery by lining up the anchor (A) in a straight path with the battery release button (B). Insert the battery into the body of the Hypervolt.

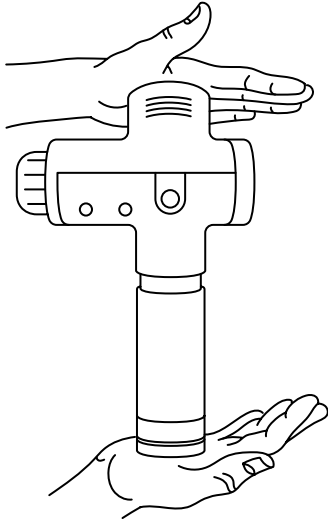


Securing your battery.

After inserting the battery in to the body of the Hypervolt, press firmly together so that the unit becomes one.

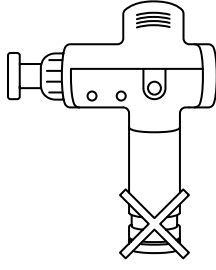

Caution

Ensure that the rubber grip on the handle is flush with the base of the battery. There should not be any "gap" of plastic showing below the rubber grip.

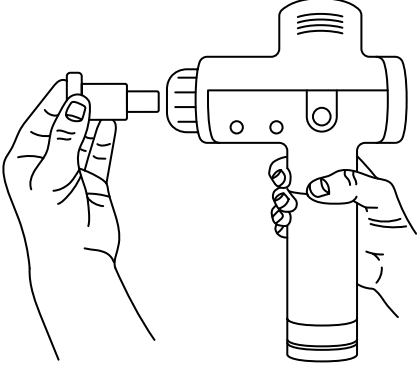


Inserting head attachment.

Choose desired head attachment, insert directly into the shaft opening while pushing firmly. Test by pulling out to ensure the head attachment does not release.

Removing head attachment

Grip the head attachment tightly and twist while pulling.

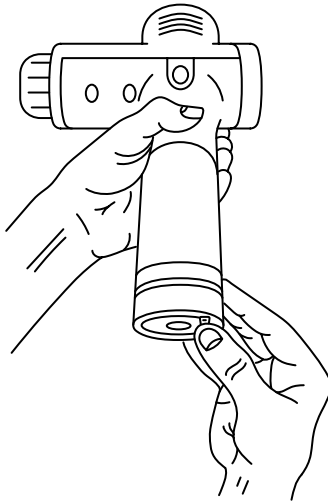


Powering ON.

Switch the Hypervolt to the ON position at the base (bottom) of the battery.


Powering OFF.

Simply switch from ON to OFF to power off the Hypervolt.

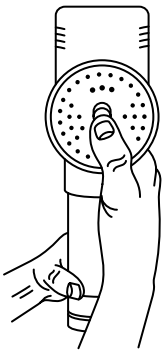


Select your speed.

Choose your desired speed (1-3) by pressing the speed setting button. Each additional speed can be reached by pressing the speed setting button once more.

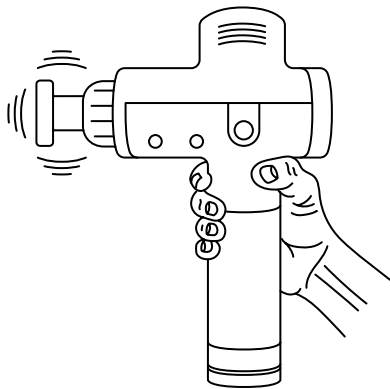

Ready for use.

Your Hypervolt will is now ready for use.

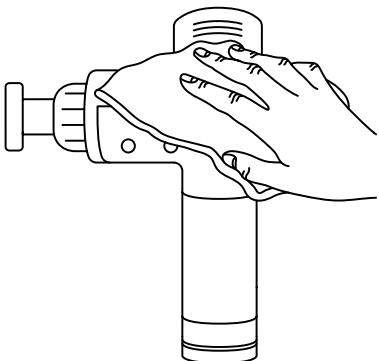

Caring for your Hypervolt.

Use a damp, clean, cloth and gently wipe down your Hypervolt. Make sure the power is OFF and the battery charger is not attached. Store in a clean, cool environment.