**GLISSANDO® / Softform®
PREMIER™ / Softform®
EXCEL™ / ESSENTIAL®**

EN **Mattresses**
User Manual 3

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[Essential product description and price.](#)

[Glissando product description and price.](#)

[Softform Excel product description and price.](#)

[Softform Premier product description and price.](#)

Call 800-397-5899 to order today!

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

© 2014 Invacare® Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Contents

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

1	General	4
1.1	General information	4
1.2	Symbols	4
1.3	Limited Warranty	5
1.4	Quality and Flame Retardancy	7
1.5	Intended use	7
1.6	Service life	7
2	Safety	8
2.1	Safety information	8
3	Components	10
3.1	Product description	10
3.2	Components	10
4	Accessories	12
4.1	Raised Side Rail Cover (optional)	12
5	Setup	13
5.1	Safety information	13
5.2	Fitting the Raised Side Rail Cover (optional)	14
6	Usage	15
6.1	Safety information	15
7	Transport	17
7.1	Safety information	17
8	Maintenance	18
8.1	Inspection	18

8.2	Cleaning and Care	18
8.3	Cleaning Raised Side Rail Covers	20
8.4	Compression test	21
9	After Use	22
9.1	Storage	22
9.2	Re-Use	22
9.3	Disposal	22
10	Technical data	23
10.1	Mattress	23
10.2	Environmental Parameters	25
10.3	Materials	25

I General

I.1 General information

Essential nursing care is pivotal in pressure ulcer prevention. These mattresses will positively contribute to the outcome of a pressure ulcer prevention care plan.

Education, clinical judgement and action based planning based on vulnerability are fundamental factors in the prevention of pressure ulcers.

A range of assessment scales can be used as a formal method of assessing risk from pressure ulcer development, and should be used in conjunction with an informal assessment (informed nursing judgement). Informal assessment is considered to be of greater importance and clinical value.

Please heed all the notes, particularly the safety information, and act accordingly.

Should you require more information on the product please refer to the *Invacare Softform®* Brochure and contact details on the back page of this User Manual.

I.2 Symbols

Symbols in this manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

DANGER

Indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury

WARNING

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

Indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

Manufacturer

Symbols on the product

	Do not pierce or cut		Line dry
	Do not dry clean		Refer to user manual
	Do not put near flame		Recommended 176 °F / 80 °C

	User weight limit*		Do not iron
	Do not bleach		Tumble dry low heat (less than 104 °F / 40 °C)
	Date of manufacture		

* Maximum user weight as per section 10 Technical data, page 23.

1.3 Limited Warranty

THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products. This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the mattress and cover when purchased new and unused to be free from defects in materials, workmanship, and assembly for the following time periods:

Product	Foam	Cover
GLISSANDO®	8 years	4 years
SOFTFORM® PREMIER™	8 years	4 years
SOFTFORM® EXCEL™	6 years	2 years
ESSENTIAL®	2 years	1 year

If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced with new

1185740-A

or refurbished product components, or a refund corresponding to the proper service time shall be offered, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Moreover, the warranty period of the repaired or replaced product shall terminate with the termination of the warranty period for the original product component. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair or replacement. For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide the dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Invacare Corporation will issue a return authorization. The defective unit or parts shall be cleaned using disinfectant prior to return and must be returned for warranty inspection using the serial number, when applicable, as identification within thirty days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; DAMAGE CAUSED BY ANY EVENTS OUTSIDE THE CONTROL OF INVACARE INCLUDING, BUT NOT LIMITED TO: DAMAGE FROM CARELESSNESS, ACCIDENTAL DAMAGE, DAMAGE FROM SHARP OBJECTS, TEARS, NEGLIGENCE (WHETHER WILFUL OR NOT), TAMPERING, DAMAGE FROM FIRE, DAMAGE OCCURRING IN TRANSIT AFTER DELIVERY BY INVACARE OR EXCESSIVE WEAR AND TEAR,

INCLUDING BUT NOT LIMITED TO THE FAILURE TO FOLLOW CLEANING INSTRUCTIONS, USE OF INAPPROPRIATE CLEANING COMPOUNDS AND FAILURE TO ENSURE THAT ANY PRODUCT IS THOROUGHLY DRIED BEFORE TURNING OR FLIPPING IT OVER; DAMAGE RESULTING FROM CONTAMINATION FOLLOWING DAMAGE TO THE COVER AND FAILURE TO REPLACE IT, CONTAMINATION DUE TO INCORRECT STORAGE OR FAILURE TO INSPECT THE INTERIOR OF THE MATTRESS AS STATED HEREIN, OR DAMAGE DUE TO UNUSUAL CONTAMINATION OR INFECTION, SPILLAGE OR STAINING, OR USE OF ABRASIVE, CORROSIVE OR ANY SUCH SIMILAR DAMAGING SUBSTANCE; DAMAGE DUE TO UNDERTAKING ANY ACTION OR PROCEDURE USING THE PRODUCT OUTSIDE ITS NORMAL AND INTENDED USE; DAMAGE RESULTING FROM FAILURE TO REPLACE DAMAGED PARTS PROMPTLY OR DAMAGE DUE TO INTERIOR PARTS BEING USED WITHOUT THE COVER, BEING USED WITH A DAMAGED COVER, OR WITH THE COVER BEING INCORRECTLY FITTED; DAMAGE WHEN ANY PARTS OF THE MATTRESS ARE USED AS PART OF, OR IN CONJUNCTION WITH, ANY OTHER PRODUCT OR PIECE OF EQUIPMENT EXCEPT AS SPECIFICALLY PROVIDED FOR BY INVACARE; DAMAGE NOT PROVEN TO THE SATISFACTION OF INVACARE TO BE DUE TO DEFECTIVE MATERIALS, WORKMANSHIP OR ASSEMBLY; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS OR PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE.

IT IS A CONDITION OF THIS WARRANTY THAT THE FOLLOWING IS COMPLIED WITH:

1. ALL INSTRUCTIONS FOR CLEANING AND MAINTENANCE OF THE MATTRESS ARE ADHERED TO.
2. THE INTERNAL AND EXTERNAL PARTS OF ANY PRODUCT ARE INSPECTED AT LEAST ONCE EVERY THREE (3) MONTHS FOR ANY DAMAGE AND WEAR. ALL DAMAGED PARTS ARE REPLACED IMMEDIATELY WITH THE APPROPRIATE REPLACEMENT PART, OR THE MATTRESS IS IMMEDIATELY TAKEN OUT OF SERVICE UNTIL THE REPLACEMENT PART HAS BEEN OBTAINED AND THE DAMAGED PART HAS BEEN REPLACED.
3. THE MEASUREMENT OF COMPRESSION OF THE MATTRESS (THE "COMPRESSION TEST") BEYOND 25% WILL BE DETERMINED BY LAYING A STRAIGHT EDGE OVER THE COMPRESSED AREA AND MEASURING THE DEPTH OF THE AREA. IF THE FOAM IS DETERMINED TO BE COMPRESSED BEYOND 25%, THE MATTRESS WILL BE REPLACED.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR, REPLACEMENT, OR REFUND OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER AND DISCLAIMS ANY LIABILITY BEYOND THE INITIAL PURCHASE PRICE OF THE MATTRESS. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

I.4 Quality and Flame Retardancy

The foam and covers used to manufacture the Invacare mattresses are independently tested and certified in accordance with TB129 (Test procedure for mattresses for use in public buildings), 16 CFR 1632 and 1633 (Flammability).

I.5 Intended use

This pressure redistribution mattress is intended to be used in conjunction with an appropriately sized bed frame, as part of an overall pressure ulcer prevention program of care.

Any other use is prohibited.

This product has been designed to deliver effective pressure reduction to users, when the product is in normal use which is defined by Invacare Ltd as when the support surface is covered with a cotton, cotton combination or linen bed sheet, and any one of these would be the only item deployed between the support surface and the user.

I.6 Service life

We estimate a life expectancy of five years for these products, provided they are used in strict accordance with the intended use as set out in this document and all maintenance and service requirements are met. The estimated life expectancy can be exceeded if the product is carefully used and properly maintained, and provided technical and scientific advances do not result in technical limitations. The life expectancy can also be considerably reduced by extreme or incorrect usage.

The fact that we estimate a life expectancy for these products does not constitute an additional warranty.

2 Safety

2.1 Safety information

WARNING!

Risk of Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment.

WARNING!

Risk of developing pressure ulcers

Bed sheets must be loosely fitted, with creases smoothed out. Care must always be taken to ensure that the support surface in contact with the user is kept free from crumbs and other food debris, and that drip cables, stents, and other foreign objects do not become entrapped between the user and the pressure reducing surface of the mattress, as this may result in the development of pressure ulcers.

DANGER!

Risk of Injury Or Death

Bed accessories designed by other manufacturers have NOT been tested by Invacare. Use of NON-Invacare bed accessories may result in injury or death.

- Use ONLY Invacare rails, mattresses, bed extenders and other accessories with Invacare bed products.

WARNING!

Risk Of Injury Or Damage

Conditions such as restlessness, mental deterioration and dementia or seizure disorders (uncontrolled body movement), sleeping problems, and incontinence can significantly impact a patient's risk of entrapment. Pediatric patients or patients with small body size may also have an increased risk of entrapment.

- Monitor patients with these conditions frequently.

DANGER!

Risk Of Product Damage, Injury, Or Death

To avoid damage, injury, or death from explosion:

- Do not use with flammable anesthetics.

DANGER!

Risk Of Product Damage, Injury, Or Death

To avoid burning a hole in mattress/bedding or causing a fire, which could lead to further product damage, injury, and/or death:

- Do not smoke while using this device.
- Visitor smoking is not permitted. Smoking by visitors in the room will contaminate the mattress.

**DANGER!****Risk Of Product Damage, Injury, Or Death**

To avoid damage, injury, or death from fire:

- Only use nasal mask or half bed tent type with oxygen administering equipment.
- Do not allow oxygen tent to extend below mattress support level.

**WARNING!**

- Refer to the Bed Rail Entrapment Risk Notification Guide included with your mattress or at www.invacare.com for additional safety information.

**IMPORTANT!**

The information contained in this document is subject to change without notice.

- Check all parts for shipping damage and test before using.
- In case of damage, do not use.
- Contact *Invacare*/Carrier for further instructions.

3 Components

3.1 Product description


The *Invacare Glissando*®, *Softform*® Premier™, *Softform*® Excel™ and *Invacare Essential*® mattresses offers exceptional pressure reducing qualities, specific to the individuals needs. By promoting even weight distribution, they reduce tissue pressure at potentially vulnerable areas.

The water-resistant cover provides a vapour-permeable, multi stretch surface, to promote patient comfort and to maximise the effectiveness of the foam core.

3.2 Components


The following components are included within the scope of delivery:

Softform® Premier™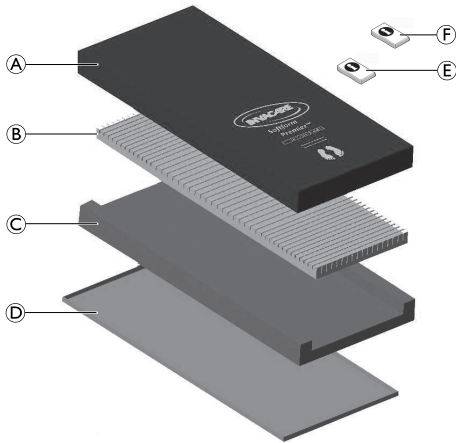


(A)	Two-way stretch waterproof polyurethane top cover
(B)	Castellated foam insert
(C)	Single piece foam "U" core
(D)	Toughened polyurethane coated base cover
(E)	User Manual
(F)	Bed Rail Entrapment Risk Notification Guide

Softform® Excel™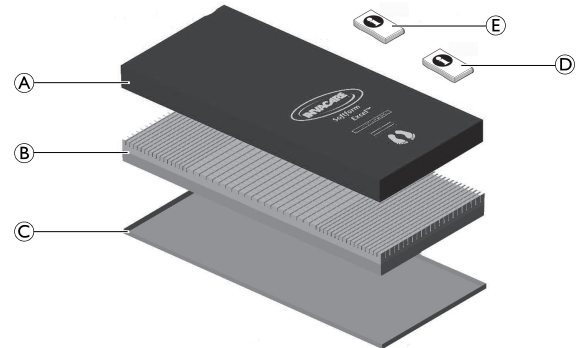


(A)	Two-way stretch waterproof polyurethane top cover
(B)	Foam core with 'keyhole cut' cells
(C)	Toughened polyurethane coated base cover
(D)	User Manual
(E)	Bed Rail Entrapment Risk Notification Guide

Glissando®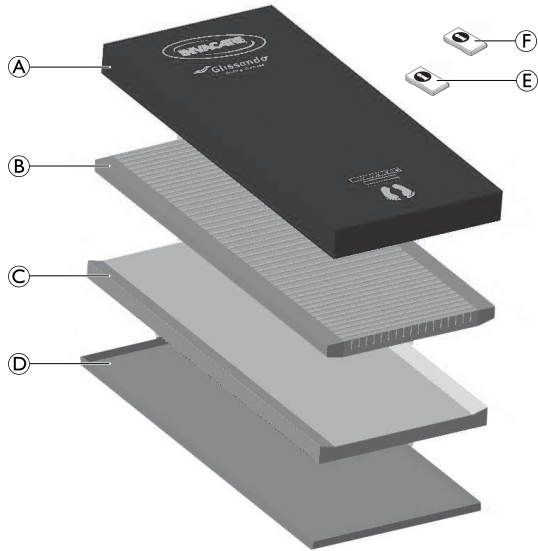

(A)	Two-way stretch waterproof polyurethane top cover
(B)	High density foam layer
(C)	Gliding layer
(D)	Toughened polyurethane coated base cover
(E)	User Manual
(F)	Bed Rail Entrapment Risk Notification Guide

Essential®

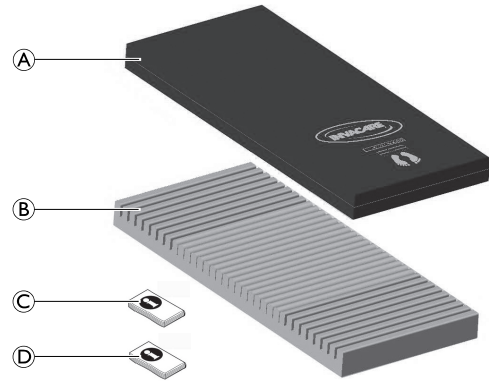(A)	Multi stretch, waterproof polyurethane cover
(B)	High density foam layer
(C)	User Manual
(D)	Bed Rail Entrapment Risk Notification Guide

4 Accessories

4.1 Raised Side Rail Cover (optional)

Optionally a replacement cover for bed frames with raised side rails can be fitted to the foam of the mattress. The RSR cover (which can be fitted to the Invacare Glissando®, Softform® Premier™ and Softform® Excel™) has welded seams. There are integrated foam bolsters within the Raised Side Rail cover which can be easily removed for cover laundering by means of individual zippers on the underside of the cover. The standard mattress cover can be replaced by the RSR covers without changing the foam.

WARNING!

Risk of injury or property damage

- Do not use Raised Side Rail Covers with integrated foam bolsters as a restraint or fall prevention in themselves.
- Do not use Raised Side Rail Covers outdoors.

The RSR Covers are intended to be used on Softform® Premier™ / Softform® Excel™ and Glissando® pressure redistributing mattresses in beds with raised side rails. The RSR Covers are intended to be used in an acute or long term care environment or in a home care situation. The bolsters are designed to warn the patient when they are nearing the edge and to give the cue to reposition themselves within the surface away from the edge.

5 Setup

5.1 Safety information

WARNING!

The mattresses are recommended to be installed on medical bed frames with bed sides or assist rails. It is preferred that the rails be in the raised position whenever the patient is on the bed.

- Operate all bed frame motorized functions through their full range of motion to be certain that there is no pulling, interference or pinching.

WARNING!

Risk Of Injury Or Death

To avoid entrapment and/or falling:

- Health care professionals assigned to each case should make the final determination whether side or assist rails are warranted after assessing patient risks of entrapment and falls in accordance with State patient restraint legislation or facility interpretation of such legislation.

WARNING!

- Do not modify this equipment without authorization of the manufacturer.
- If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of the equipment.

WARNING!

Risk of entrapment

Patient entrapment with the bed side rails may cause injury or death. Proper patient assessment and monitoring, and proper maintenance and use of equipment is required to reduce the risk of entrapment. Variations in bed rail dimensions, mattress thickness, size and density could increase the risk of entrapment.

- Visit the FDA website at <http://www.fda.gov> to learn about the risks of entrapment. Review “A Guide to Bed Safety”, published by the Hospital Bed Safety Workgroup, located at www.invacare.com. Use the link located under each bed rail product entry to access this bed safety guide.
- Refer to the Bed Rail Entrapment Risk Notification Guide included with your mattress for further information.

WARNING!

Risk of Injury Or Death

Patient entrapment from the use of bed side rails may cause injury or death. To avoid patient entrapment:

- Mattress must fit firmly against the bed frame and bed side rails to prevent patient entrapment. Follow the manufacturer’s instructions. Monitor patient frequently. Read and understand the User Manual prior to using the mattress and rails. User manuals are available from Invacare or your dealer.
- After any adjustments, repair or service and before use, make sure all attaching hardware is tightened securely.

WARNING!

Risk of Injury or Damage

To avoid product damage, shock/or personal injury:

- Keep all bed components and accessories a minimum of 12" / 305 mm away from hot or heated surfaces.

5.2 Fitting the Raised Side Rail Cover (optional)

1. Unzip the standard cover of the mattress and remove it carefully from the foam core.
2. Place the Raised Side Rail Cover onto the foam core of the mattress.

IMPORTANT!

- Ensure that the corners of the foam core are positioned correctly into the corners of the Raised Side Rail Cover.

3. Close the zippers of the Raised Side Rail Cover.

6 Usage

6.1 Safety information

1. Remove all packaging before use.
2. Place the mattress directly on the frame of the bed.

The mattress is designed for beds with adjustable lying surface.

WARNING!
Risk of Injury or Damage

Continued use of the bed with damaged parts could lead to the bed malfunctioning, causing injury to the user and/or caregiver.

- Check ALL bed components and carton for damage, and test components before use. In case of damage or if the bed is not working properly, contact a technician or Invacare for repair.
- Do not use the mattress outdoors.

WARNING!

It is very important for the patient to reposition themselves, or to be repositioned, on a regular basis. This must be based on the clinical judgement of a qualified health care professional. This relieves pressure which helps prevent both tissue compression and potential ulcer formation.

- Always consult a qualified health care professional before using the mattress.
- Monitor the patient frequently.

WARNING!

Risk of injury or damage

An increased risk of patient entrapment may occur over time due to mattress compression.

- Periodically monitor gaps between the bed, mattress, and/or bed rail. Where gaps occur, patient entrapment is possible and the mattress should be replaced.

CAUTION!

– Make sure that the printed side of the mattress cover always faces upwards.

- Make sure that the distance between the surface of the mattress and the top of the side rail is at least 8.7" / 220 mm.

IMPORTANT!

If holes are present in the mattress cover, there is a danger that liquids may ingress and contamination may occur.

- Medical equipment including infusion pumps and monitors must be attached to appropriate bed accessories.
- In domestic settings common causes of damage include cigarette burns and the claws of pets that puncture covers, allowing fluid ingress and staining.

IMPORTANT!

Risk of damage to the mattress covers

- To prevent accidental cover damage, do not place hypodermic needles, venflons, scalpels or other similarly sharp objects onto the mattress.
- Ensure that all venflons are taped down correctly with no sharp edges exposed.
- When using bridging boards or other patient transfer aids, care should be taken not to damage the mattress cover. All transfer aids should be checked for any sharp edges or burrs before use as these can damage the mattress cover.
- Make sure that the mattresses are not jammed or damaged by sharp edges when used on beds with an adjustable frame.
- When using the mattress on a profiling bed ensure that the knee break is used before the backrest.

7 Transport

7.1 Safety information

**IMPORTANT!**

- Take care when handling mattresses to ensure no damage to the cover. It is recommended that two people lift/carry mattresses.
- Avoid contact with jewellery, nails, abrasive surfaces etc.
- Do not drag mattresses.
- Avoid contact with wall, door frames, door catches or locks etc.
- Do not transport in roll cages unless completely protected from the sharp edges of the cage.

- I. Refer to the storage and shipping conditions in section Technical Data, ® 10.2 Environmental Parameters, page 25.

8 Maintenance

8.1 Inspection

It is recommended that the mattress is checked (foam and cover) for strike-through (this may include fluid ingress, stains, rips or damage) after the release of each patient or on a minimum monthly basis (depending on which occurs first) by a suitably qualified and competent person.

! IMPORTANT!

It is recommended that internal and external parts of the mattress be inspected at least every 3 months for damage and wear.

Check mattress

1. Unzip the cover completely.
2. Check for any staining on the white underside of the cover.
3. Check for any staining on the interior foam.
4. Replace any stained items and dispose of as per local authority procedure.

8.2 Cleaning and Care

WARNING! **Risk Of Injury Or Damage**

To avoid injury or damage from sharp objects:

- Before cleaning or disassembling, check the underside of the mattress folds for sharp objects such as scissors, needles, etc.
- Remove and discard sharp objects before proceeding with cleaning or disassembly.

WARNING! **Risk Of Injury Or Damage**

To avoid infectious exposure:

- Do not clean the bed when the patient is occupying the bed.
- Inspect all equipment. Properly clean or remove any item soiled with the patient's blood or other body fluids.
- Treat all soiled bedding as if it is contaminated with pathogenic microorganisms.

CAUTION!

Iodophor type disinfectants (Betadine®, etc.) are not recommended and will stain fabrics.

- To avoid fabric breakdown and loss of waterproofing, use only these types of properly diluted standard cleaning solutions (Quaternary wipe down, Phenolic wipe down or 8 – 10 % bleach/water wipe down).

IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.

- For further information on decontamination in health care environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG002 and your local infection control policy.

Wiping down covers

(Removal of contaminants such as dust and organic matter)

1. Remove the bedding.
2. Clean the top and bottom covers with diluted detergent solution or soapy water.

Cleaning covers

(Removal of contaminants such as dust and organic matter)

1. Remove all covers for laundering.
2. Launder the covers with the recommended temperature of 176 °F / 80 °C using a diluted detergent solution (Instructions on label).

IMPORTANT!

Washing at higher temperatures will cause shrinkage.

Drying covers

1. Hang mattress covers from a line or bar and drip dry in a clean indoor environment.
or
Tumble dry on a low heat setting.

IMPORTANT!

- Tumble dry setting must not exceed 104 °F / 40 °C.
- Do not tumble dry for longer than 10 minutes.
- Dry covers thoroughly before re-fitting to the mattress.

Disinfecting covers

(Reducing the number of microorganisms)

Please contact your hygiene specialist in the event of contamination.

1185740-A

IMPORTANT!

- Ensure that any residual detergent has been removed prior to disinfection.

Light soilage

1. Wipe down the cover with a 0.1% Chlorine Solution (1,000 ppm).
2. Rinse the cover with clean water using a single use nonabrasive cloth.
3. Dry the cover thoroughly.

Heavy soilage

Where the mattress is badly soiled, we recommend cleaning with a dilute cleaning solution at 176 °F / 80 °C in the washing machine.

Large spillages of blood should be absorbed and removed with paper towels first, followed by:

1. Clean up all spillages of bodily fluids i.e. blood, urine, faeces, sputum, wound exudate and all other bodily secretions as soon as possible using a 1% Chlorine Solution (10,000 ppm).
2. Rinse the cover with clean water using a single use nonabrasive cloth.

IMPORTANT!

Polyurethane coated fabrics can absorb liquids for short periods causing a temporary change to the polyurethane characteristics. The mattress cover swells temporarily and is more vulnerable to physical damage for a period after it is completely surface dried, by which time it will revert to its previous state.

3. Dry the cover thoroughly.

! **IMPORTANT!**

1% Chlorine Solution used on a regular basis can diminish the life of the cover if not rinsed and dried properly.
– Do not use granules.

WARNING!

– Remove contaminated foams from use.

CAUTION!

– Keep clear of open heat sources.

! **IMPORTANT!**

– Do not use phenols, alcohols, bleaches, or other abrasive materials.

Replacing covers

1. Unzip the cover and remove it carefully from the foam core.
2. Place new cover onto the foam core.
3. Then close the zipper.

! **IMPORTANT!**

– Ensure that the corners of the foam core are positioned correctly into the corners of the cover.
– Ensure that the profiled side of the foam is facing uppermost when packed into its cover.

8.3 Cleaning Raised Side Rail Covers

1. Unzip the Raised Side Rail Cover of the mattress and remove it carefully from the mattress foam core.
2. Open the four zippers underneath the integrated bolsters on the bottom of the RSR Cover.

3. Remove all four foam bolsters and store them in a dry and safe place.
4. Launder the RSR Cover in the washing machine at 176 °F / 80 °C.

! **IMPORTANT!**

– Washing at higher temperatures may cause shrinkage.

5. Hang mattress covers from a line or bar and drip dry in a clean indoor environment.
or
Tumble dry on a low heat setting.

! **IMPORTANT!**

– Tumble dry setting must not exceed 104 °F / 40 °C.
– Do not tumble dry for longer than 10 minutes.
– Dry covers thoroughly before re-fitting to the mattress.

6. Insert all four foam bolsters into the RSR cover.
7. Close all four zippers underneath the integrated bolsters.
8. Place the RSR cover onto the foam core of the mattress.

! **IMPORTANT!**


– Ensure that the corners of the foam core are positioned correctly into the corners of the RSR cover.

9. Close the zippers of the RSR cover.

Disinfecting Raised Side Rail Covers

1. Follow section “Disinfecting covers” in chapter 8.2 Cleaning and Care, page 18.

8.4 Compression test

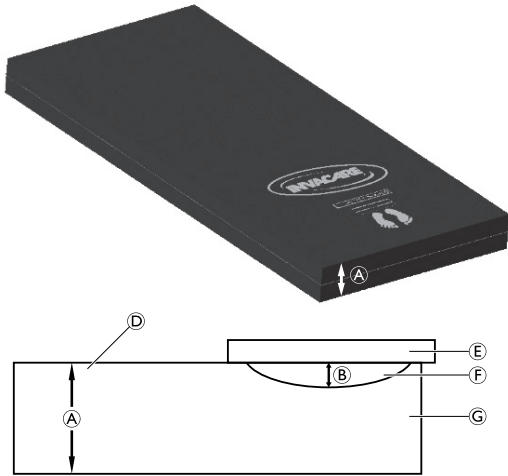1. Remove all bedding from the mattress ③.
2. Measure the thickness of the mattress (mattress depth ①) in an uncompressed area ④.
3. Lay a straight edge ⑤ over the compressed area ⑥ of the mattress so each end is roughly at the same height.
4. Measure from the bottom of the straight edge to the lowest point of the compressed area to find the depth of the compressed area (compression depth ②).
5. Put the measurements you have obtained into following formula (compression depth divided by mattress depth multiplied by 100) to determine the amount of compression (compress percentage): $⑦ = ② / ① \times 100$.

9 After Use

9.1 Storage

To ease storage and handling the mattresses are sometimes compact rolled depending on customer requirements. This in no way affects the performance and longevity of the product. When the mattress is unrolled for the first time, allow it to recover for a minimum of 1 hour before use to optimize performance.

! IMPORTANT!

- Store mattresses in a dry environment.
- Store mattresses within a protective cover.
- Store items on clean, dry, off-flooring free from sharp edges to avoid any possible damage.
- Never store other items on top of a mattress.
- Do not store mattresses next to radiators or other heating devices.
- Protect mattresses from direct sunlight.

! IMPORTANT! Risk of permanent deformation of raised side rail bolsters

- Do not store Raised Side Rail Covers (individually packed) for more than three months.

1. Refer to the storage and shipping conditions in section Technical Data, ® 10.2 Environmental Parameters, page 25.

9.2 Re-Use

A cleaning record must be kept as part of the cleaning system.

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used.

1. Before reuse, clean the product thoroughly,
® 8.2 Cleaning and Care, page 18.

9.3 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

10 Technical data

10.1 Mattress

	GLISSANDO®	SOFTFORM® PREMIER™	SOFTFORM® EXCEL™	ESSENTIAL®
Dimensions mattress (Length x Width x Height)	76 x 36 x 6" 1930 x 914 x 152 mm 80 x 36 x 6" 2032 x 914 x 152 mm 84 x 36 x 6" 2134 x 914 x 152 mm 80 x 39 x 6" 2032 x 991 x 152 mm 80 x 42 x 6" 2032 x 1067 x 152 mm 80 x 48 x 6" 2032 x 1219 x 152 mm	76 x 36 x 6" 1930 x 914 x 152 mm 80 x 36 x 6" 2032 x 914 x 152 mm 84 x 36 x 6" 2134 x 914 x 152 mm 80 x 39 x 6" 2032 x 991 x 152 mm 80 x 42 x 6" 2032 x 1067 x 152 mm 80 x 48 x 6" 2032 x 1219 x 152 mm	76 x 36 x 6" 1930 x 914 x 152 mm 80 x 36 x 6" 2032 x 914 x 152 mm 84 x 36 x 6" 2134 x 914 x 152 mm 80 x 39 x 6" 2032 x 991 x 152 mm 80 x 42 x 6" 2032 x 1067 x 152 mm 80 x 48 x 6" 2032 x 1219 x 152 mm	80 x 36 x 6" 2032 x 914 x 152 mm 84 x 36 x 6" 2134 x 914 x 152 mm
Height Raised Side Rail cover	3" / 76 mm	3" / 76 mm	3" / 76 mm	N/A
Maximum user weight	500 lbs / 227 kg ²⁾	500 lbs / 227 kg ²⁾ 650 lbs / 295 kg ³⁾	450 lbs / 204 kg	350 lbs / 159 kg
Weight of product with standard cover	28.7 lbs / 13 kg	28.7 lbs / 13 kg	22 lbs / 10 kg	16 lbs / 7.25 kg
Weight of product with Raised Side Rail cover	29.2 lbs / 13.25 kg	29.2 lbs / 13.25 kg	22.5 lbs / 10.2 kg	N/A
Warranty foam ¹⁾	8 years	8 years	6 years	2 years

	GLISSANDO®	SOFTFORM® PREMIER™	SOFTFORM® EXCEL™	ESSENTIAL®
Warranty standard cover ¹⁾	4 years	4 years	2 years	1 year
Warranty Raised Side Rail cover ¹⁾	2 years	2 years	2 years	N/A
Nominal density range	2.38 – 2.50 lbs 1.08 – 1.13 kg	2.38 – 2.50 lbs 1.08 – 1.13 kg	2.38 – 2.50 lbs 1.08 – 1.13 kg	1.44 – 1.56 lbs 0.65 – 0.71 kg
Nominal hardness range [N]	23.6 – 30.4 40.5 – 45	23.6 – 30.4 40.5 – 45	23.6 – 30.4 40.5 – 45	30.4 – 32.6
Firetesting	TB 129 16 CFR 1632 16 CFR 1633	TB 129 16 CFR 1632 16 CFR 1633	TB 129 16 CFR 1632 16 CFR 1633	TB 129 16 CFR 1632 16 CFR 1633

¹⁾ Warranty is against manufacturing defects.

²⁾ Mattress width 36" / 914 mm.

³⁾ Mattress widths 39", 42" and 48" / 991 mm, 1067 mm and 1219 mm.

10.2 Environmental Parameters

Operating conditions	
Ambient temperature	50 - 95 °F / 10 - 35 °C
Relative humidity	30% - 75% Non-Condensing
Atmospheric pressure	70 - 106 kPa
Storage and shipping conditions	
Ambient temperature	-40 - 158 °F / -40 - 70 °C
Relative humidity	10% - 100%
Atmospheric pressure	50 - 106 kPa

10.3 Materials

	GLISSANDO®	SOFTFORM® PREMIER™	SOFTFORM® EXCEL™	ESSENTIAL®
Cover	Multi-stretch, vapour permeable, water resistant polyurethane (PU)			
Foam	Combustion Modified High Resilience polyurethane foam (PU)			

Cover and Foam materials are Latex-free.

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Invacare UK Operations Limited,