*Jay Care Cushion*

*User Instruction Manual & Warranty*

*Jay Care Kussen*

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*Jay Care-dyna*

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*Jay Care -tyyny*

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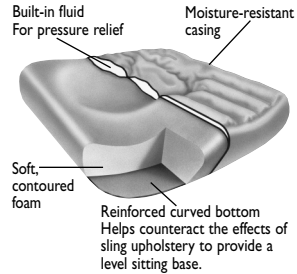
*Jay Care Pude*

*Brugsanvisning Manuel og Garanti*

### JAY CARE CUSHION - 3100 SERIES MODELS

The Jay Care Cushion has built-in positioning and maximum comfort in a moisture-resistant, easy-to-use cushion. The Jay Care Cushion is designed to help reduce the likelihood of pressure sores and provide good sitting posture without modifications or addition of accessories. It is designed primarily for geriatric and stroke patients, Multiple Sclerosis, Muscular Dystrophy, Polio, or other clients with sensation.

#### Jay Care Cushion components

### INSTRUCTIONS FOR USE FOR JAY CARE CUSHION

#### Caution

Your Jay Care Cushion is designed to help reduce the incidence of pressure sores. However, no cushion can completely eliminate sitting pressure or prevent pressure sores. The Jay Care Cushion is not a substitute for good skin care including: proper diet, cleanliness, and regular pressure reliefs.

In addition, prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist. Do not leave the cushion outside overnight at temperatures below 40° F (5° C). Allow the cushion to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage. Avoid sharp objects or exposure to excessive heat or open flame.

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
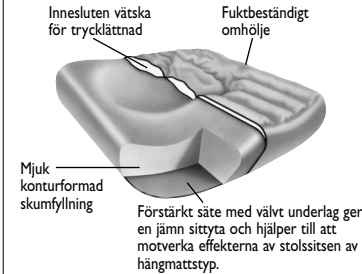
The Jay Care Cushion is designed to be used on sling upholstery or a dropped solid seat. Do not use on a solid seat that is level with the seat rails or on overslung upholstery. Overslung upholstery will encourage bowing of the cushion, detrimental to the cushion's design in support of the body. If the upholstery is overly slung (more than 2 1/2" 6cm) (see diagram A), replace the upholstery. Consult with your local authorized supplier.

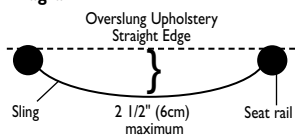Diagram A

The Jay Care Cushion is moisture-resistant under normal sitting conditions. However, it should not be used as a bath cushion. There are openings in the rear of the cushion that allow air to escape the cushion. Submersion in water could cause moisture to enter the cushion. Avoid bath or shower situations with the cushion.

#### Caution

Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the chair to tip backwards potentially resulting in injury. Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

#### Fitting a Jay Care Cushion

Select a cushion to match your hip width when in the seated position. The cushion length should extend to within 1" - 2" (2.5cm - 5cm) from the back of the knees. Place the cushion to the back of the wheelchair with the material identification tags toward the rear. When fit correctly, your hips should be to the back of the chair. Your ischials (seat bones) should be centered in the seat well of the cushion. Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures. Adjust the footrests so your legs rest firmly but comfortably on the cushion.

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The Jay Care Cushion is designed to position your legs 1/2" - 1" (1cm -2.5cm) above the level of the seat rails, minimizing the change in seating height. When sitting on the cushion, your hips should be as far to the back of the wheelchair seat as possible to allow your buttocks to sit in the deeper, fluid area of the cushion. Your legs should rest comfortably on the front half of the cushion.


The Jay Care Cushion will accommodate most sitting postures. The cushion cannot be modified or the incontinence-resistant aspects will be compromised. However, different sizes are available to accommodate a variety of width and length needs.

**Note:** Do not use the wheelchair's seat width as a guide in determining the appropriate cushion width. Improper cushion fit may be detrimental to the performance of a pressure distributing cushion and its support of the body.

To prevent the cushion from sliding on the wheelchair upholstery or during transfers, follow these instructions for the Fastener Kit.

1. Use the alcohol swabs to clean a 4" x 12" (10cm x 30cm) area on the wheelchair upholstery approximately 2" (5cm) from the front edge of the wheelchair upholstery.
2. Peel off the paper backing of the 4" x 12" (10cm x 30cm) Velcro® strip and attach it to the cleaned area of the wheelchair upholstery (see diagram B). Press firmly.
3. Place the cushion in the wheelchair. The Velcro® strip on the wheelchair upholstery should match up with the Velcro® strip on the bottom of the Care Cushion and secure the cushion in the wheelchair.

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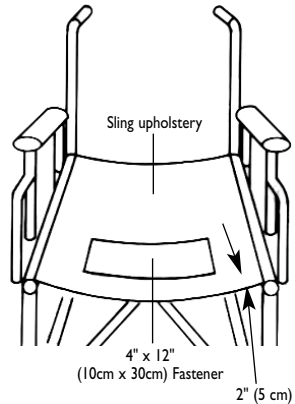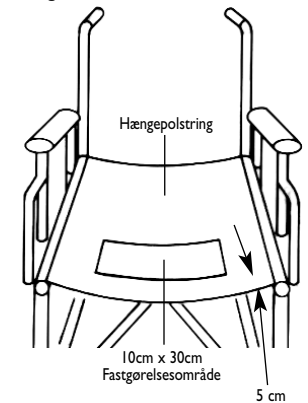
Diagram B

**WHEN USING A SOLID SEAT**

The Jay Care Cushion is designed to be used on a sling seat or a solid adjustable seat in the fully dropped position. It is not recommended for use on a solid seat that is level with the seat rails. If placed on such a seat, the curved bottom of the cushion will act as a "rocker", which may flatten out the cushion, allowing the legs to externally rotate (abduction). To install, simply place the cushion in the wheelchair and push it all the way to the back of the seat. Be sure that the front of the cushion faces the front of the wheelchair. Position the bump to the front. The curved edge of the back of the cushion will fit between the back posts of the wheelchair to "lock" it into place. Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended. Check the cover for tears and excessive wear and inspect the fluid pads for punctures or any other abnormalities. While checking the fluid pad if you ever feel the fluid is firmer in one area, simply knead the fluid back to its original consistency and/or call your local authorized supplier.

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**EASY MAINTENANCE, CLEANING, & ASSEMBLY****To clean cover**

Remove the cover from the base. Machine wash in warm water and drip or tumble dry on low heat. No ironing required.

**Note:** Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach. To refit cover after washing, stretch cover back over foam base. Ensure the back of the cover matches up to the back of the foam base.

**Note:** If utilized in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.

Use only Jay-designed covers on a Jay Care Cushion.

**To clean base**

Remove the cover from the base. Wipe with warm water and soap. Wipe off with a clean cloth.

**Note:** Avoid submersion of the base in water. Fit cover back over base. Ensure the back of the cover matches up to the back of the base.

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**JAY CARE CUSHION WARRANTY**

Your Jay Care Cushion is carefully inspected and tested to provide peak performance. Every Jay Care Cushion is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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---

**ENGLISH**

**Sunrise Medical  
Home Healthcare Group  
Mobility Products Division**  
7477 East Dry Creek Parkway  
Longmont, Colorado 80503  
USA  
(303) 218-4500

**DUTCH**

**Sunrise Medical BV**  
Importeur  
Pascalbaan 3  
3439 MP Nieuwegein  
tel: 030-6082100

**SWEDISH**

**Sunrise Medical AB**  
Britta Sahlgrens gata 8A  
SE-421 31 VÄSTRA FRÖLUNDA

**FINNISH**

**Merivaara Instrumentarium**  
Puusteinintie 2  
Lahti FINLAND  
SF 15150

**DANISH**

**G2 Wheelchair and Gear**  
Jegstrupvej 56  
DK 8361 Hasselager DENMARK  
DK 8361

This product is manufactured to comply with  
the 'Medical Device Directive' 93/42/EEC.

Manufactured under one or more of the following patents plus other patents pending: U.S. Patent  
# 4,588,229; 4,660,238; 4,726,624; 4,761,843; 4,842,330; 5,018,790; 5,255,404; 5,352,023; 5,362,543;  
5,369,829; 5,390,384; 5,395,162; 5,457,833; 5,490,299; 5,397,517; and other foreign patents.

© 1998, Sunrise Medical 6.98 Rev. A  
XT3105S