


# SKILLBUILDERS®

## ADJUSTABLE SWING SEAT

### INSTRUCTIONS TO ADJUST ROPE LENGTH

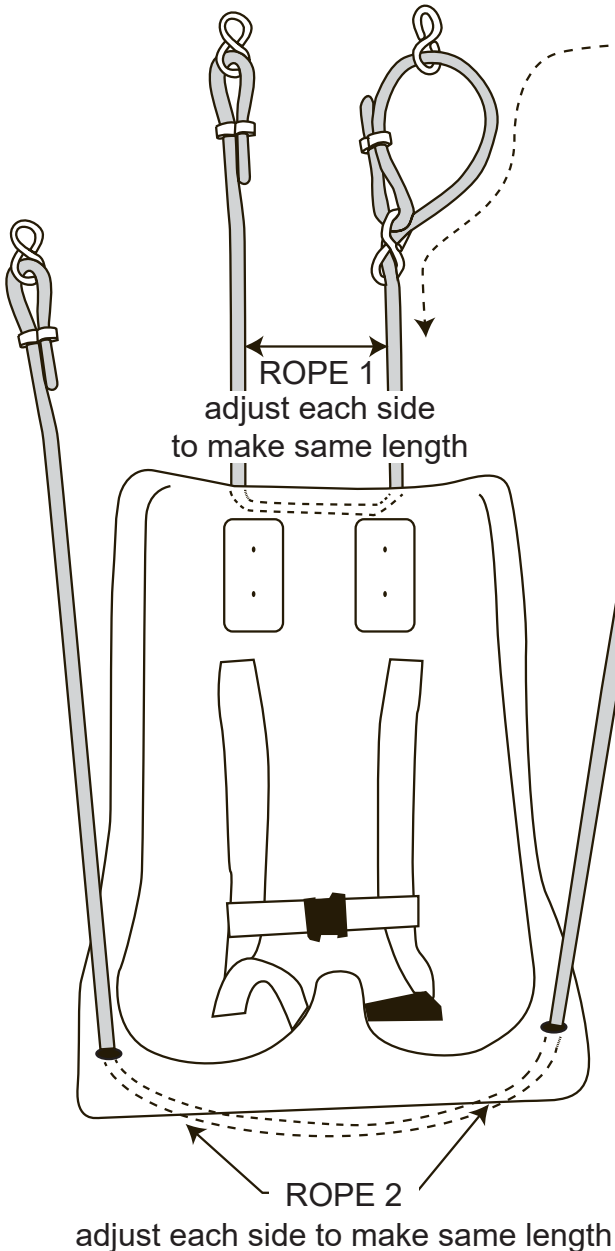

TO LENGTHEN rope:  
Pull rope DOWN as indicated  
to make loop SMALLER.  
Adjust lengths to make even.

TO SHORTEN rope:  
Pull rope UP as indicated  
to make loop LARGER.  
Adjust lengths to make even.



**IMPORTANT SAFETY NOTICE:**  
To ensure proper seat tilt, make sure  
each side of ROPE 1 is the same  
length and each side of ROPE 2  
is the same length.

seat	inner dimensions	maximum weight capacity
small	20"L x 12"W x 9"D 51cm x 30cm x 23cm	75 lbs / 34 kg
medium	23"L x 15"W x 10"D 58cm x 38cm x 25cm	150 lbs / 68 kg
large	31"L x 20"W x 14"D 79cm x 51cm x 36cm	200 lbs / 91 kg
full-body reclining	24"L x 14"W x 11"D 61cm x 36cm x 28cm	150 lbs / 68 kg



Fabrication Enterprises Inc  
PO Box 1500  
White Plains, NY 10602 (USA)  
tel: +1-914-345-9300 • 800-431-2830  
fax: +1-914-345-9800 • 800-634-5370  
FabEnt.com



Authorized CE representative:  
RMS UK Ltd.  
28 Trinity Road  
Nailsea, Somerset BS48 4NU (UK)

Skillbuilders is a trademark of Goldberg  
©2016 FEI, all rights reserved

ver 0716